

AccuWrap™

Two-layer compression bandage system



**Skin
Health**


Make compression accurate and comfortable

**Advanced
Wound Care**



Manage venous disease with accurate compression

Venous leg ulcers are caused by an obstruction of the venous system or a defective venous valve. The condition begins with leg swelling and skin inflammation, which can lead to a venous ulcer. These wounds heal slowly and 70% return after healing. Compression therapy is a primary strategy to treat recurrence.^{1,2}

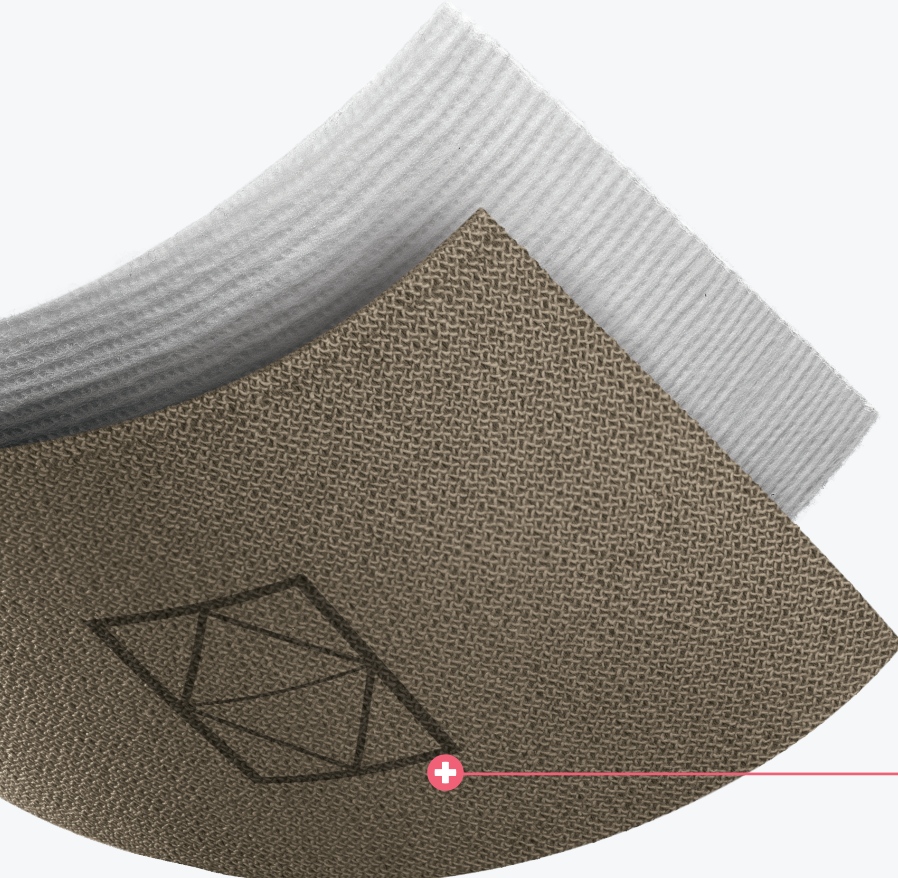

Key benefits

More comfortable for patients, encouraging compliance


Low-profile design easily fits under clothing and footwear

More accurate compression with accuracy indicator

Middle line indicates how to apply with 50% overlap



Accuracy indicator

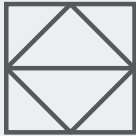

Indicator should be stretched to the size of this image

Put the comfort in compression

AccuWrap is a two-layer system designed to provide 30-40 mmHg³ of therapeutic compression to manage venous disease and associated edema for up to 7 days.

AccuWrap is more comfortable and breathable under clothing, making it easier for patients to wear throughout treatment.



Thinner than other leading brands³

Layer 1

Short stretch padding layer



100% polyester



Comfortable and breathable



Slight cohesive texture helps it stay in place

Layer 2

Woven short stretch compression layer



Sturdy, breathable woven fabric



Accuracy indicators confirm ideal compression level

AccuWrap compression system delivers therapeutic compression for up to 7 days

Apply layer 1 first. Repeat steps 1-5 for layer 2.



1. Flex the foot to 90 degrees and apply two anchor turns starting at the base of the toes.



2. Proceed up and around the ankle, covering the Achilles tendon.



3. Return over the top of the ankle and down to secure the heel. Cover all exposed skin of the heel and ankle.



4. Wrap in a spiral up the leg. While wrapping, overlap the bandage by half its width.



5. Finish at the notch just below the knee. Cut and discard any unused bandage.

Ordering information



AccuWrap kit

Item No.	Description	Compression	Pkg.
MSC7800	1. Absorbent padding: 4" x 4.4 yd. (10 cm x 4 m) 2. Short-stretch cohesive: 4" x 6 yd. (10 cm x 5.5 m) stretched	30-40 mmHg*	8 kits/cs



AccuWrap XL kit

Item No.	Description	Compression	Pkg.
MSC7800XL	1. Absorbent padding: 4" x 5.5 yd. (10 cm x 5 m) 2. Short-stretch cohesive: 4" x 7 yd. (10 cm x 7.5 m) stretched	30-40 mmHg*	8 kits/cs



AccuWrap Lite kit

Item No.	Description	Compression	Pkg.
MSC7802	1. Absorbent padding: 4" x 4.4 yd. (10 cm x 4 m) 2. Short-stretch cohesive: 4" x 6 yd. (10 cm x 5.5 m) stretched	20-30 mmHg*	8 kits/cs

*Data on file. Actual compression levels will depend on limb circumference & application technique, and therefore will vary between patients

Make skin health second nature™

Contact your Medline Skin Health Representative for a product sample or visit medline.com/skin-health today.

References: 1. Probst S, Allet L, Depeyre J, Colin S, Skinner MB. A targeted interprofessional educational intervention to address therapeutic adherence of venous leg ulcer persons (TIEVLU): study protocol for a randomized controlled trial. *Trials*.2019;20:243. Available at: <https://trialsjournal.biomedcentral.com/articles/10.1186/s13063-019-3333-4>. Accessed November 13, 2019. 2. Finlayson K, Edwards K, et al. (2010). The Impact of psychosocial factors on adherence to compression therapy to prevent recurrence of venous leg ulcers. *J. Clin Nurs.* 19(9-10): 1289-97. 3. Data on file.



We make healthcare run better™

Medline Industries, LP
Three Lakes Drive, Northfield, IL 60093
Medline United States | 1-800-MEDLINE (633-5463)
medline.com | info@medline.com

Medline Canada
1-800-268-2848 | medline.ca | canada@medline.com
Medline México
01-800-831-0898 | medlinemexico.com | mexico@medline.com

Follow us    

Some products may not be available for sale in Mexico or Canada. We reserve the right to correct any errors that may occur within this brochure.

© 2022 Medline Industries, LP. All rights reserved. AccuWrap and Medline are registered trademarks of Medline Industries, LP. MKT19W3577632 / LIT725R / 2.5M / SG / 88