

# Advin COVID-19 Antigen Test @Home



*Easy to Use, Visual Read Rapid SARS-CoV-2  
Antigen Test for Use Anywhere*


**Advin COVID-19 Antigen Test @Home is an FDA EUA<sup>+</sup> lateral flow immunoassay device intended for the qualitative detection of nucleocapsid protein antigens from SARS-CoV-2.**

*†Advin COVID-19 Antigen Test @Home is only for use under the Food and Drug Administration's Emergency Use Authorization. This product has not been FDA cleared or approved.*

## **Convenient & Simple**

- **Easy to Use** – Simple 4-step process from sample collection to results
- **Non-Invasive** – Anterior nasal swab specimens; no need to swab deep in the nasal cavity for sample collection
- **Fast Results** – in 10 minutes
- **Available in kits of 1, 2 and 25 tests** – Ideal for healthcare providers and at-home use
- Room Temperature Storage

# Test for SARS-CoV-2 in Four Simple Steps\*

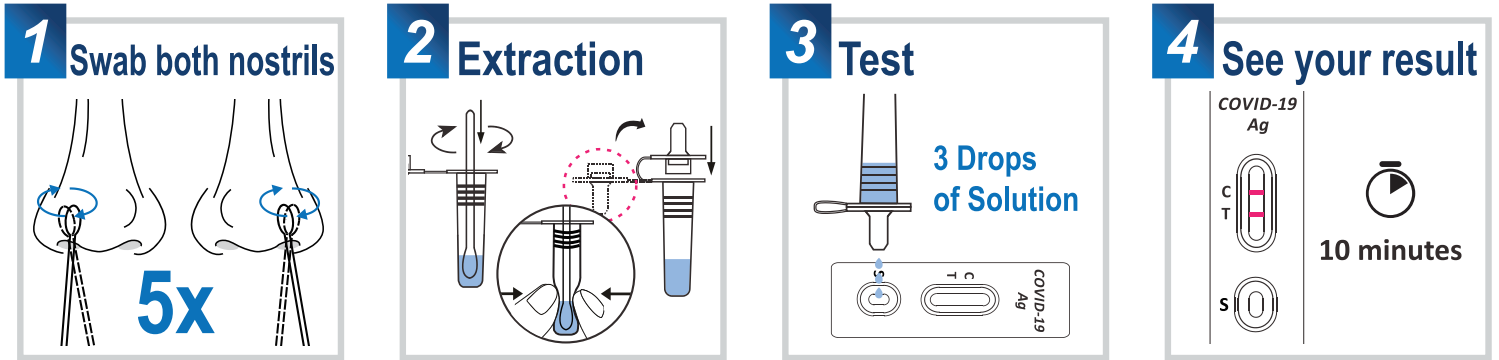

\*Refer to Product Insert for more details.

## Performance Characteristics\*

Compared to PCR

Test	PPA	NPA
Advin COVID-19 Antigen Test @Home	82.5% (95% CI:70.6%-90.2%)	100% (95% CI:99.2%-100%)

\*Refer to Product Insert for more details.

## Product Information

Information	Product Details
Time to Results	10 minutes
Sample	Anterior Nasal Swab Specimen
Result Interpretation	Visual Read
Storage Conditions	35.6° to 86° F (2° to 30° C)
Method	Lateral Flow

## Ordering Information

Product	# of Tests	Catalog Number
Advin COVID-19 Antigen Test @Home	1	66-9990-0
	2	66-9990-1
	25	66-9990-2



Chembio Diagnostics, Inc

www.chembio.com • +1.631.924.1135 • info@chembio.com