

# MDS89745RA/RAH Assembly Guide



Model number: MDS89745RA/RAH

Tools needed: None


## Guide to avoid or correct common assembly error for the MDS89745RA/RAH bath bench

The initial steps shown in the assembly instructions must be followed exactly or the assembly will not be able to be completed.

If this is not done correctly the padded armrests will not be able to be installed.




When installing the aluminum main frames, the imprinted A and B must be attached next to the corresponding A and B on the underside of the seat.

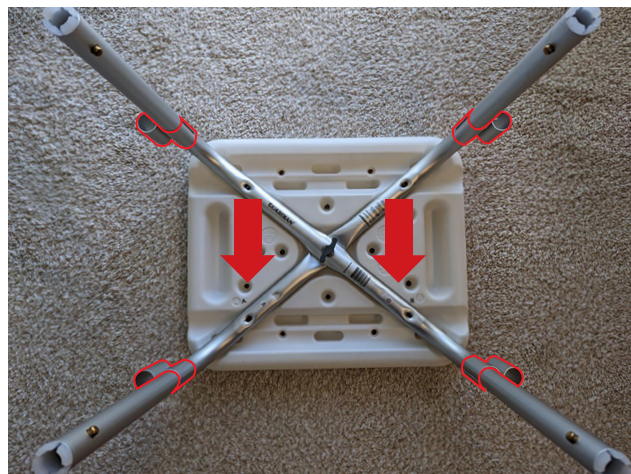

With the A/B lined up (shown by arrows) the arm rest holsters will be on the outside of the seat and legs.



Armrests installed correctly.

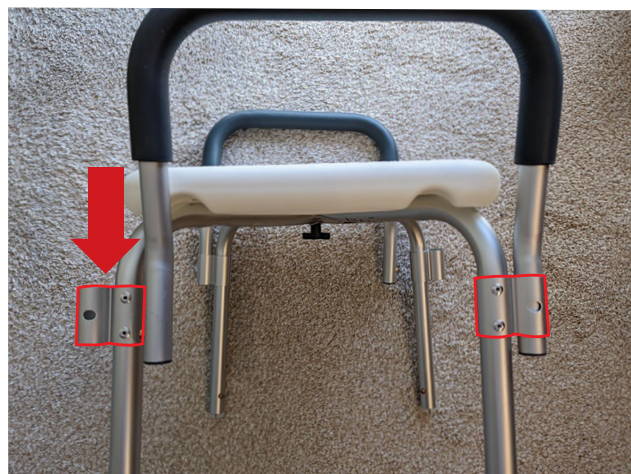

Installed incorrectly.