Manual Seated Scooter, Black
Trottinette manuelle avec siège, Noir
Escúter manual para uso sentado, Negro

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- A** Hand Brakes
- B** Hand Grips
- C** Brake Cable
- D** Handle Height Adjust Knob
- E** Basket
- F** Steering Column
- G** Locking Tiller Assembly Pin
- H** Seat Pad
- I** Rear Wheels
- J** Front Wheels
- K** Seat Cam Lock
- L** Seat Height Adjust Pin
- M** Allen Key
- N** Step-By-Step Assembly Guide
- O** Safety Guidelines

ATTENTION: This item does not come with a tire pump. A tire pump is needed to inflate tires. Check tire pressure routinely to ensure tires are inflated.

This guide applies to the following scooter:

Item Number	Description	Frame Color	Weight Capacity
MDS86000SS	Manual Seated Scooter	Black	300 lbs (136 kg)

First: Open and secure handle bar

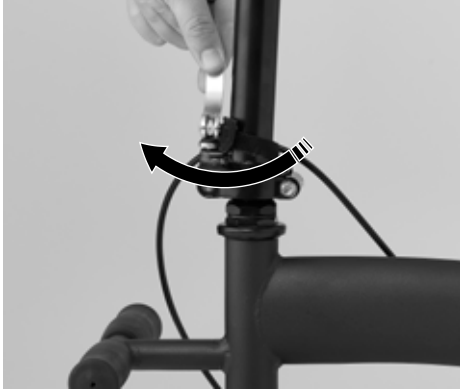
- 1** Remove contents from carton and lay parts next to frame.

- 2** Turn front wheels to face forward so that tiller is on front of scooter.

- 3** Raise tiller to the upright position. Push locking tiller assembly pin in until you hear a click. Make sure steering column is secure.

- 4** Rotate clockwise to tighten. Lift pin to upright position and push down tiller to secure.

- 5** Insert swivel lock screw into hole. Tighten using Allen key.

- 6** Insert handle bar into steering column with hand brake on front of scooter.

- 7** Adjust to desired height and insert Handle Height Adjust Knob. Turn clockwise until tightened.

Next: Set Desired Seat Pad Height

- 8** Lower the seat pad to desired height.

- 9** Insert the Seat Height Adjust Pin into the hole. Lock pin in place by sliding the loop over the protruding end of the pin.

- 10** Rotate the seat cam lock clockwise to tighten the lock pad into place. Fold close to frame.

Now: Using the Hand Brakes

- 11** Adjust brake cable to reduce loose slack and prevent from catching on objects.

- 12** To apply the brakes, simply pull the hand brake lever with your fingertips toward the hand grip.

- 13** To lock the brakes, push brake handle down until you hear a click.

Attaching Accessories

- 14** Attach basket to designated hook on steering column.

Adjusting The Seat Pad Height

15 Uninstall the locking seat height adjustment pin. Adjust seat pad to desired height.

16 Reinstall the locking seat height adjust pin and secure it in place. Next tighten the Seat Cam Lock to lock pad in place.

Finally: Adjust Handle Height

- 17** To adjust the handle height, loosen the handle height adjustment knob below the handle by turning it counterclockwise until it can be removed. Adjust the handlebar to the desired height and reinstall the handle height adjust pin by turning it clockwise until it is tightened.

To Fold the Scooter

- 18** Release the Seat Cam Lock and seat height adjust pin.

19 Lower the seat pad all the way.

20 Loosen Tiller Cam Lock.

21 To fold down the handlebars, slide assembly to the side until the tiller is disengaged.

- 22** Fold the handle bars down until it rests on the seat pad. To reduce wear on the seat pad, remove the handlebar height adjust knob and store in a safe place.

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Some products may not be available for sale in Mexico or Canada. We reserve the right to correct any errors that may occur within this brochure. © 2022 Medline Industries, LP. All rights reserved. Medline is a registered trademark of Medline Industries, LP.