Medline Transport Chair
Fauteuil de transport Medline
Silla de transporte Medline

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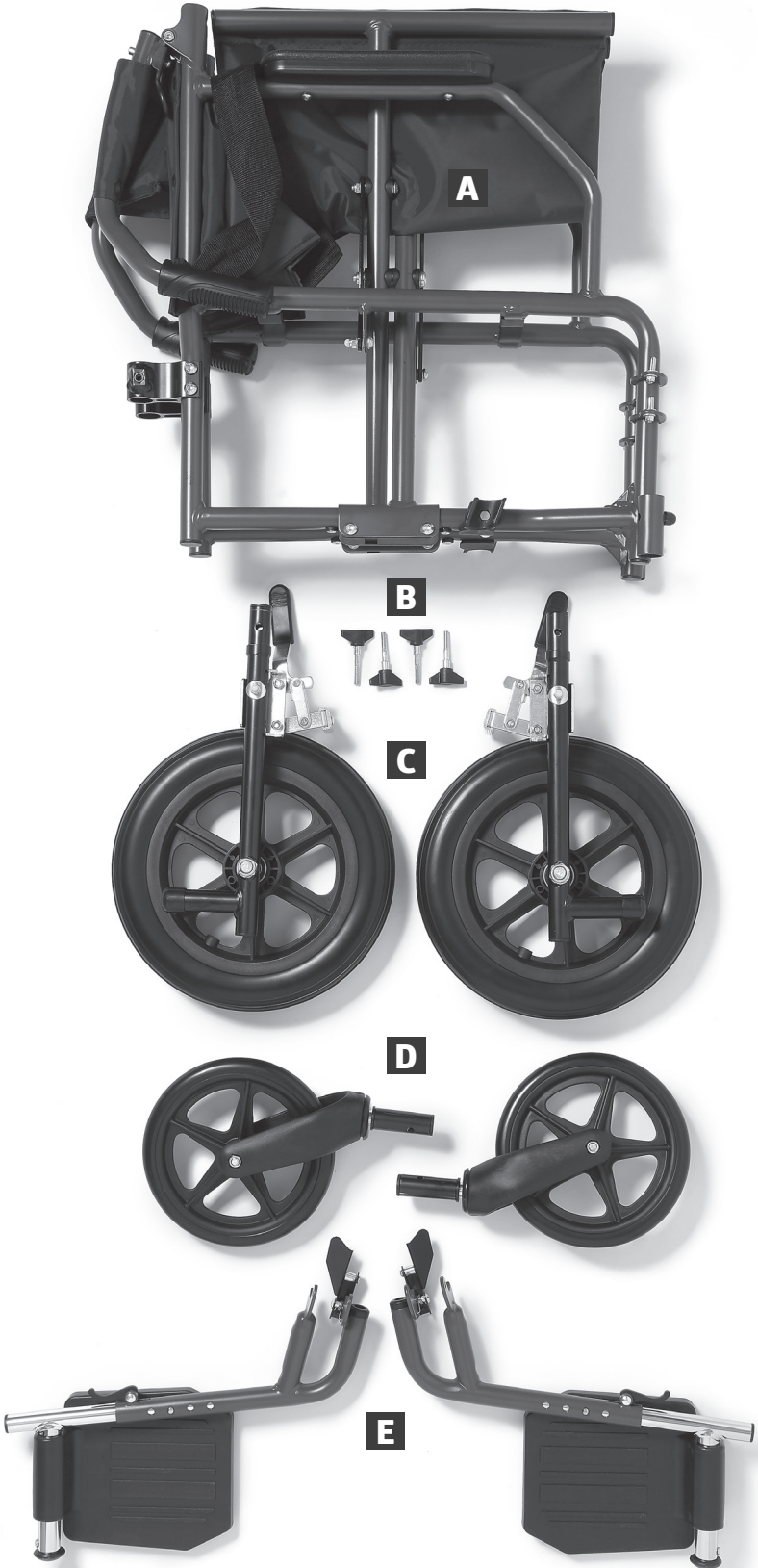
F Step-By-Step
Assembly Guide

G General Warnings

This guide applies to the following chair:

Item Number	Description	Color	Weight Capacity
MDS808210KDR	Transport Chair, Lightweight Steel, 19" x 16" (48.2 x 40.6 cm) Seat, Swing-Away Footrests, Desk Length Arms, and Stow-N-Go Clips	Grey & Burgundy	300 lb (136 kg)
MDS808200KDT	Transport Chair, Lightweight Steel, 19" x 16" (48.2 x 40.6 cm) Seat, Swing-Away Footrests, Desk Length Arms, and Stow-N-Go Clips	Grey & Teal	300 lb (136 kg)
MDS808200KWM	Transport Chair, Lightweight Steel, 19" x 16" (48.2 x 40.6 cm) Seat, Swing-Away Footrests, Desk Length Arms, and Stow-N-Go Clips	White & Light Tan	300 lb (136 kg)
MDS808200KNM	Transport Chair, Lightweight Steel, 19" x 16" (48.2 x 40.6 cm) Seat, Swing-Away Footrests, Desk Length Arms, and Stow-N-Go Clips	Navy & Blue	300 lb (136 kg)

First: Open the Chair

With the seat in an upright position, place hands on either side of the arms of the chair and gently pull outward.

As the chair begins to open, press down firmly on the outer edges of the seat with both hands, keeping fingers flat.

IMPORTANT NOTE:

Proceed with caution when opening the chair as there are several areas around the moving parts that could potentially pinch fingers.

Next: Install the Wheels

Front Wheels

1 Place the chair on its back.

2 Remove the black plastic plug from the outer tube and discard.

3 Unscrew the black plastic knob by turning it counterclockwise. Set aside.

Insert the wheel stem into the tube, making sure to align the hole in the wheel stem with the hole in the tube.

Re-insert the black plastic knob and turn it clockwise to tighten and secure the wheel to the chair.

6 Repeat these steps on the other side.

Rear Wheels

1

Place the chair on its front.

2

Unscrew the black knob from the chair base by turning it counterclockwise. Set aside.

3

Insert the wheel stem into the tube, making sure to align the hole in the wheel stem with the hole in the tube.

4

Re-insert the black plastic knob and turn it clockwise to tighten and secure the wheel to the chair.

5 Repeat these steps on the other side.

Engaging the Wheel Locks

The wheel locks are pre-installed on the rear wheels.

Push the lever towards the front of the chair to engage and lock the rear wheels.

Push the lever towards the back of the chair to unlock the rear wheels.

IMPORTANT NOTE:

Engage both wheel locks before entering or exiting the transport chair, when leaning forward, or while in an elevator or wheelchair lift.

Next: Install the Footrests

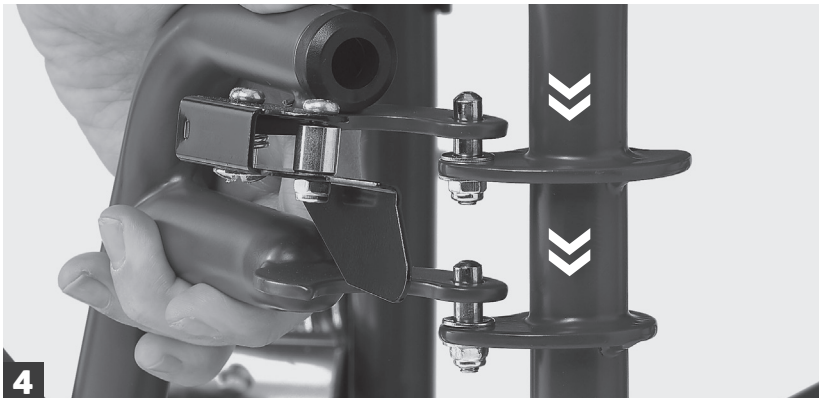
Engage the rear wheel locks by pushing the black lever towards the front of the chair.

Locate the two small silver pins on the chair above the front wheels.

Locate the two pre-drilled holes on the footrest.

Slide the footrest openings over the silver pins and place down.

5

Swing the footrest to the front of the chair until it locks into place.

6

Fold the foot plate down to use.

7 Repeat these steps on the other side.

IMPORTANT NOTE:

If the foot plate is facing outwards when attached to the chair, See section How To adjust the footrest height.

To Swing Away the Footrest:

- 1** Fold the foot plate up to the vertical position.

2 Release the footrest lock by pushing the black lever forward.

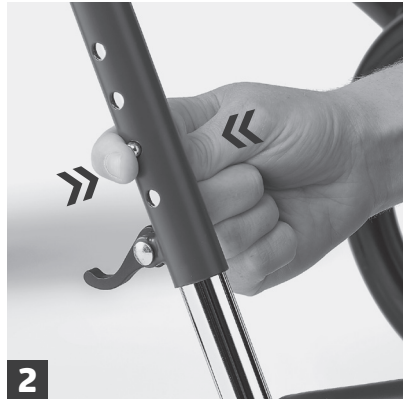
3 Snap the footrest into the Stow-N-Go Clips located behind the front wheel to secure it to the chair.

To Adjust the Footrest Height:


Flip up the black tab on the back of the leg rest.

Squeeze the push buttons on the stem of the leg rest to adjust the footrest height up or down.

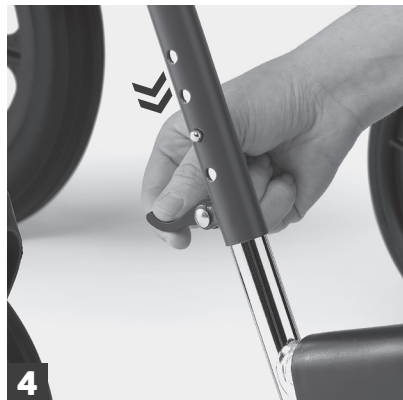When the desired height is achieved, release the pressure on the push buttons. The push buttons click securely into pre-drilled holes.

Flip down the black tab on the back of the footrest.

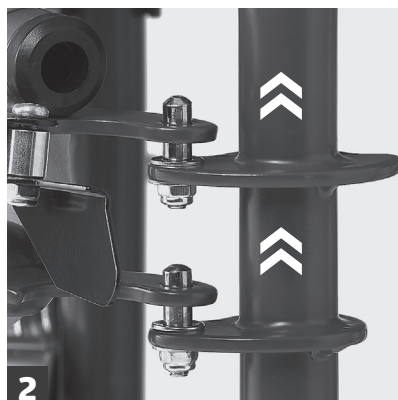
IMPORTANT NOTE:

If the foot plate is facing outwards when attached to the chair, follow instructions from figure 1 and figure 2, then rotate the foot plate so it is on the inside of the chair. Follow instructions for figure 3 and figure 4 to lock the footrest.

To Remove the Footrest From the Chair:

Disengage the tab lock.

Pull the footrest up and off of the silver pins.

Finally: Unfold the Seat Back

Using the handles, push the seat back up until the release assembly spring buttons click fully into the folding mechanism on both sides. (You should be able to see the spring buttons fully engaged from the front of the seat back canes.)

Ensure the frame/handles and seat back are locked securely into place before using the chair.

Close and Fold

Fold the footrest plates up to a vertical position, pull them back, and snap them into the Stow-N-Go holders on the side of the chair.

Turn the front wheels back. Unlock the back wheels.

Find the black release tabs on each side of the chair back canes and pull down. The seat back will collapse.

Using both hands, grasp the two fabric handles on the seat upholstery and pull up.

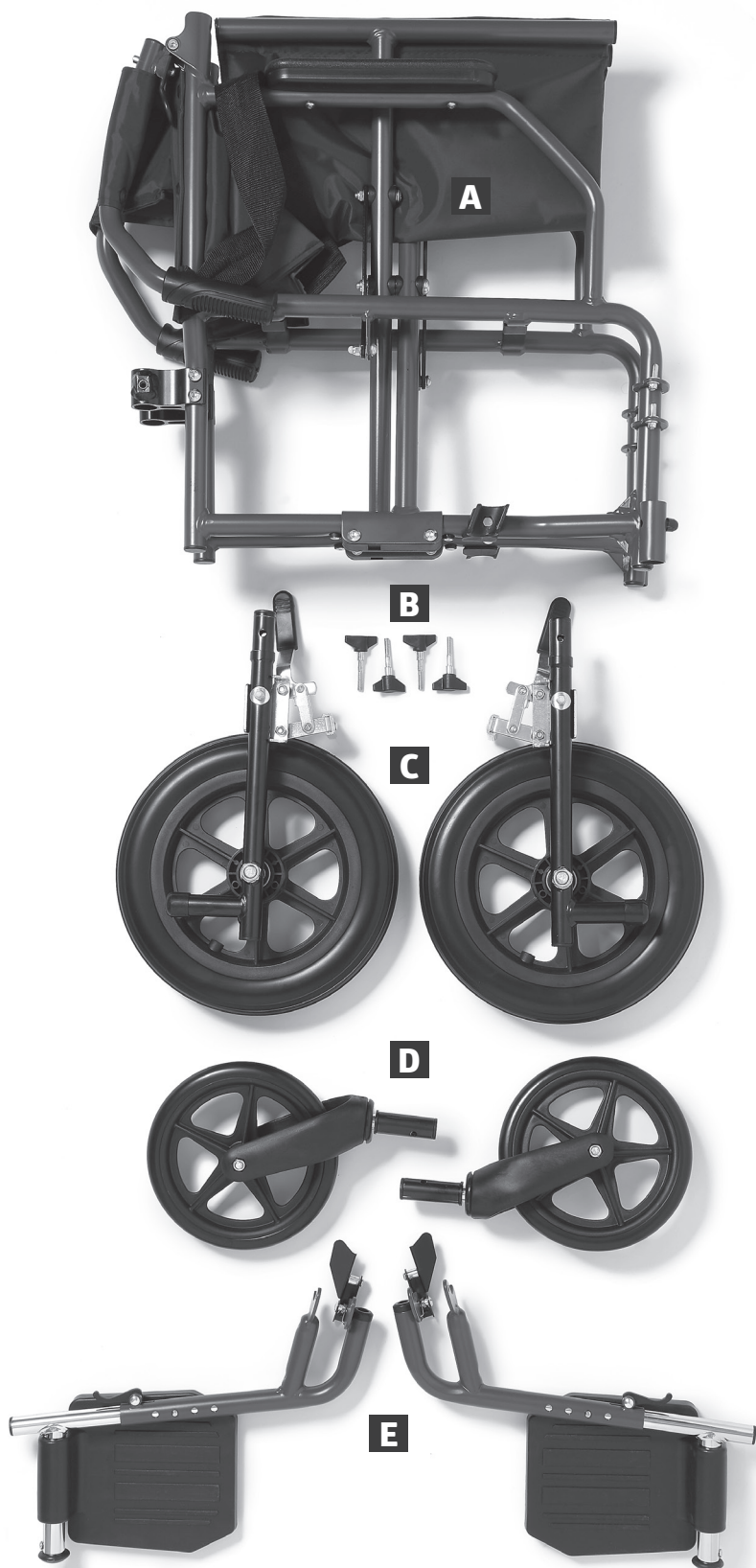
Now grasp the arms of the chair and push inward until it collapses further.

Chair folds down for easy storage and transport.

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ENTRAPMENT, STRANGULATION, SUFFOCATION AND FALL HAZARDS

Gaps in and around this product can entrap and kill people with Alzheimer's disease or dementia, or those who are sedated, confused, or frail, and are at increased risk of entrapment and strangulation. People attempting to climb over this product are at increased risk of injury or death from falls. Always make sure this product is properly secured to bed. If product can move away from bed or mattress, it can lead to entrapment and death.

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Microban® antimicrobial product protection is limited to the product itself and is not designed to protect users from disease causing microorganisms or as a substitute for normal cleaning practices.

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Follow us
Suivez-nous
Síguenos

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2023 Medline Industries, LP. Guardian and Medline are registered trademarks of Medline Industries, LP. Microban® is a registered trademark of Microban Products Company.

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