STEP-BY-STEP

ASSEMBLY GUIDE.

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PASO A PASO.

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I Step-by-Step Assembly Guide

J General Warnings

This guide applies to the following rollator:

Item Number	Description	Frame Color	Weight Capacity
MDS86850ESAR	Steel rollator with 6" wheels, knockdown. Handles are in 1" increments from 31" – 35". Includes nylon bag under seat	Burgundy	350 lbs. (159 kg)
MDS86850ESAB	Steel rollator with 6" wheels, knockdown. Handles are in 1" increments from 31" – 35". Includes nylon bag under seat	Blue	350 lbs. (159 kg)

First: Open the Rollator

Remove the rollator components from shipping carton. Then, in an upright position, unfold the rollator by pushing down on the folding bracket until the frame is in full extension.

Place the rollator on its side.

IMPORTANT NOTE:

Proceed with caution when opening the rollator as there are areas around the moving parts that could potentially pinch fingers.

Now: Install the Wheels

Front Wheels

Remove the four black plastic plugs from the outer tubes and discard.

Attach front legs without brakes to the front of the rollator. The front legs are marked 'Front Left' & 'Front Right'.

Use the triangular knob to screw the front legs in place. Repeat steps 1-3 for the other front leg extension without brakes.

Rear Wheels & Brakes

With the rollator still on its side, attach rear wheels with brakes. Make sure that the rear wheels and brake shoe are on the outside of the rollator and brake handle (See above). Use the triangular knobs to thread the rear legs in place.

3

On the same side insert the brake handle assembly into the frame and set to desired height. Brake cables should be on the outside of the handle bars and frame.

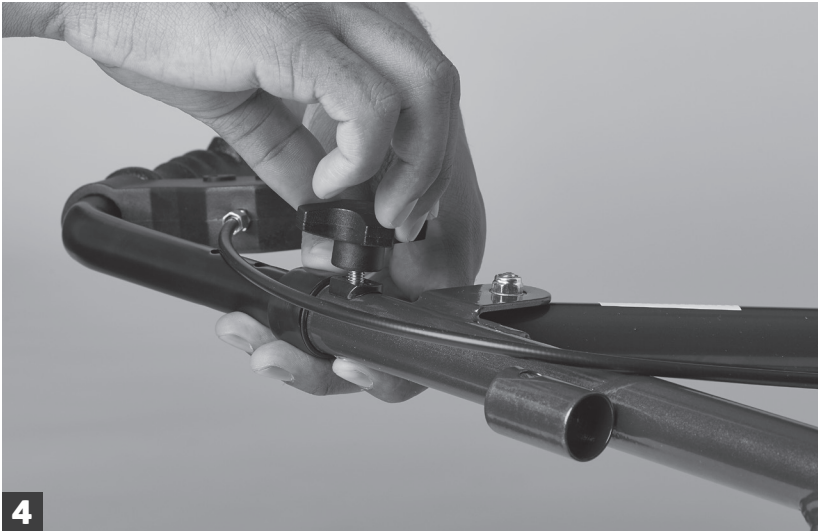
4

Insert the screw into the 6 sided hole in the frame. Make sure the screw head recesses into the frame.

3

Slide the curved washer onto the other end of the screw with the curved side facing the frame.

4

Screw on the four point knob.

5

Turn the rollator onto the other side so that the wheels that were just installed are now on the floor. Then, repeat steps 1-4. on the other side.

Finally Install Backrest

Stand the rollator up so all 4 wheels are on the floor.

Insert the backrest so that the curved portion faces the front of the unit. Make sure that the button snaps firmly into the hole so the back is secure.

Loop the cloth hooks onto the seat frame and make sure the bag is centered.

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Some products may not be available for sale in Mexico or Canada. We reserve the right to correct any errors that may occur within this brochure. © 2018 Medline Industries, Inc. All rights reserved. Medline is a registered trademark of Medline Industries, Inc. MICROBAN is a registered trademark of Microban Product Company Corp.
Medline Industries, Inc.