Bed Assist Bar

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User Manual

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REF BA6800

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Contents

Warnings	3-4
Assembly	5-7
Installation	8
Operating Instructions.....	9
Maintenance and Adjustments	9
Cleaning	9

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Warnings

- **DO NOT** install this product without first reading and understanding this instruction sheet. If you are unable to understand these warnings, cautions, and instructions, contact a healthcare professional or dealer before attempting to install this equipment. Failure to follow instructions can result in serious injury and death.
- Use only with non-articulating Twin, Full, Queen or King beds with mattresses ranging 6 in to 19 in.
- Before use ensure that all push buttons and hardware are securely in place and that the legs are leveled and are securely touching the ground.
- **DO NOT** use on beds without a box spring.
- This unit is not meant to support full body weight but to assist with stability when getting in or out of the bed. **DO NOT** use for any other purposes.
- **DO NOT** use this product as a restraint or to prevent users from rolling out of the bed.
- Ensure the product fits snug against the mattress and bed frame to prevent user entrapment. If the strap provided is not properly secured the product may move into an unsafe position which increases the danger of entrapment.
- **DO NOT** use two bed assist bars on the same side of the bed. This will create an entrapment hazard that can result in injury or death. A bed assist bar may be used on the opposite side of the bed, **ONLY** when there is **NO** interference of components or the straps when installed.
- **DO NOT** use this product if there is a risk of patient entrapment. Patient entrapment may cause injury or death!
- Stop using immediately if damaged or broken, or if parts are missing.
- Stop using immediately if product shifts out of its original position until it is readjusted into the correct position.
- In addition to contacting the manufacturer directly, consumers should report problems to the CPSC at its website [SaferProducts.gov](https://www.saferproducts.gov) or call 1-800-638-2772, or to the FDA at 1-800-332-1088.
- For further information, see: [cpsc.gov/en/Safety-Education/Neighborhood-Safety-Network/Posters/Adult-Bed-Rails/](https://www.cpsc.gov/en/Safety-Education/Neighborhood-Safety-Network/Posters/Adult-Bed-Rails/) and www.fda.gov/bedsafety.

 **WARNING:** If the strap provided is not properly secured the product may move into an unsafe position which increases the danger of entrapment. See assembly and installation instructions for proper use of the straps.

⚠️ WARNING – ENTRAPMENT, STRANGULATION, SUFFOCATION AND FALL HAZARDS

Gaps in and around this product can entrap and kill. People with Alzheimer's disease or dementia, or those who are sedated, confused, or frail, and are at increased risk of entrapment and strangulation. People attempting to climb over this product are at increased risk of injury or death from falls. Always make sure this product is properly secured to bed. If product can move away from bed or mattress, it can lead to entrapment and death.





⚠️ WARNING – SUFFOCATION/STRANGULATION/ENTRAPMENT HAZARD

If product is installed incorrectly or moves from its initial position, gaps can occur which can entrap and kill. People with Alzheimer's disease, dementia or other neurological conditions, or those who are sedated, confused, or frail, are at increased risk of entrapment, suffocation and strangulation.

- **NEVER** use unless product is tight against mattress, without gaps, and at least 12½ in. from headboard and footboard.
- **NEVER** use with children.
- **NEVER** use on toddler, bunk, water, or inflatable beds, or on beds with mattress toppers or soft compressible pads.

⚠️ WARNING – ENTRAPMENT HAZARD

NEVER use product without properly securing it to bed. Incorrect installation can allow product to move away from mattress, bed frame and/or head or foot boards, which can lead to entrapment and death.

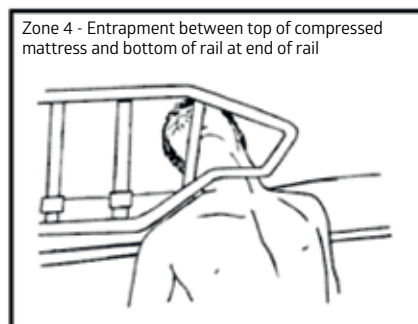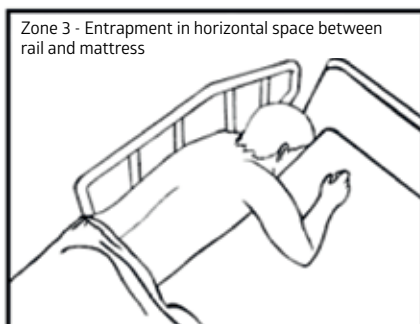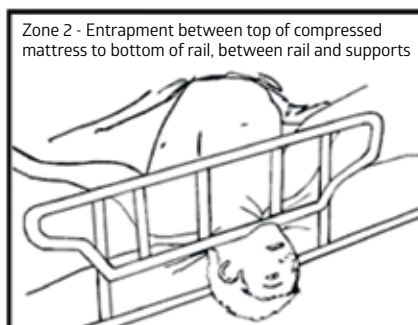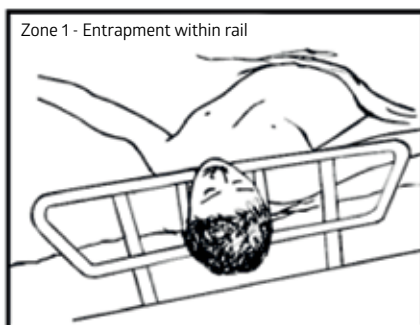Assembly and Installation Instructions

Assembly:

1. Open the box and remove the contents.

- A. Assist Bar
- B. Legs (2)
- C. Support Frames (2)
- D. Base Frame
- E. Brace Bar with Strap
- F. Space Bar with Push Buttons
- G. Height Adjust Bar with Caps
- H. Screws (2)

Note: screws are attached to Height Adjust Bar with Caps when shipped and must be removed prior to assembly

- I. Hex Key
- J. Storage Pocket
- K. Key

Figure 1

2. Gather parts C, D, E, and F. Please note there are 2 each of the Support Frames (C)- a left and a right.

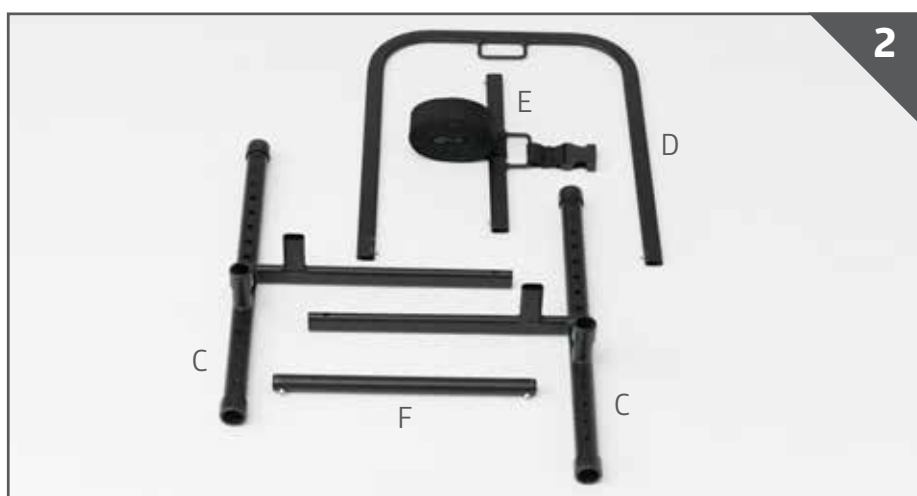
Figure 2

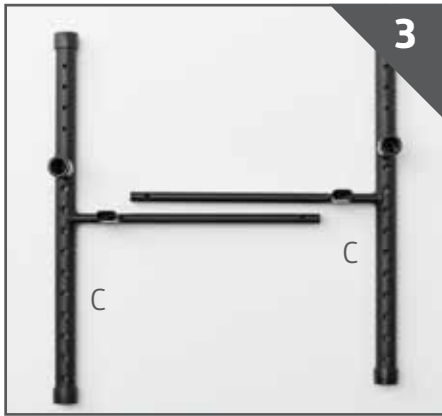
Figure 3

3. Look at the Support Frame (C) pieces.

Note: There are 4 smaller holes on the top of these pieces and 10 larger holes on the bottom. Keep this in mind for later.

Figure 4

4. Attach Brace Bar (E) in the oval shaped hole and Space Bar (F) in the circular hole to one of the Support Frames (C) using the push button. Ensure that push buttons align with adjustment hole when attached Brace Bar (E) and Space Bar (F).

Figure 5

5. Attach the other Support Frame (C) on the other side of the Brace Bar (E) and Space Bar (F). Press both push buttons on the bars at the same time for easiest assembly.

Figure 6

6. Place assembled 2 support frames flat on the ground. Assemble the Base Frame (D) to the 2 Support Frames (C) using the push buttons.

Figure 7

7. Identify which side of 2 Support Frames have 10 larger adjustment holes. Assemble both Legs (B) into the 2 Support Frame pieces (C), on the side with the 10 larger holes. Ensure that push buttons on Legs (B) are aligned with the adjustment holes on the Support Frame pieces. Push buttons must click into the adjustment hole.

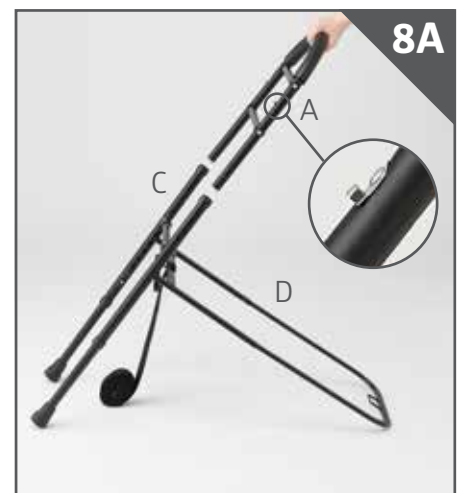
Figure 8A

- 8A. Slide the Assist Bar (A) to the side of the support Frame Pieces (C) with the 4 smaller holes. The bag clips must face outward away from Base Frame (D).
Note: Make sure the handle is straight when attaching for easiest assembly.

Figure 8B

8B. Select the handle height based off the mattress thickness range as stated below and on the product. Line up the holes of Assist Bar (A) with holes on both sides of Support Frames (C) ensuring they are the same on each side. Mattress Thickness Range and Horizontal bar position:

Lowest hole from bottom	6" to 16"
2nd Lowest from bottom	6" to 17"
2nd Highest from top	6" to 18"
Highest Height setting	6" to 19"

Note: The appropriate height selection **MUST** be chosen based off the mattress thickness or it could result in an entrapment risk.

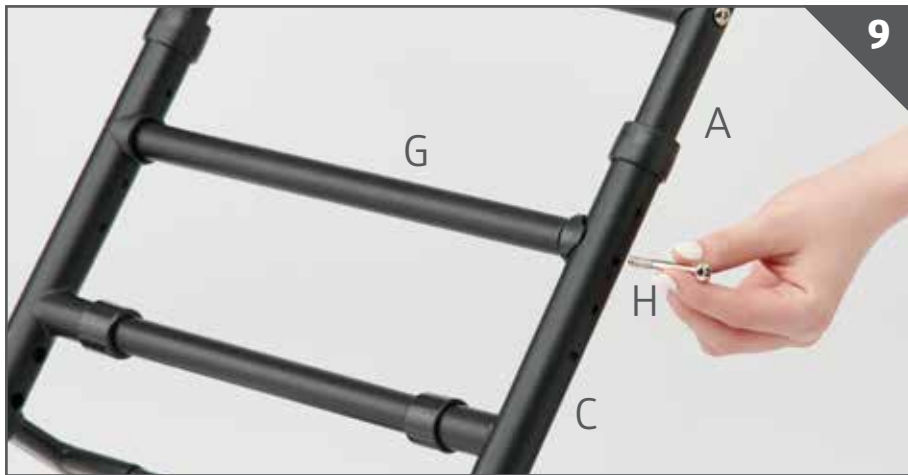
Figure 9

9. Insert the Height Adjust Bar with Caps (G) at an angle in the center of parts (A) and (C). The bar fits snug, tapping on the top of the bar may be required. The bar must be completely horizontal and lines up with the holes of part (A). Insert Screw (H) through parts (C) and (A) into part (G).

Figure 10

10. Tighten the Screw (H) using the Hex Key (I). Do not fully tighten until the screws and holes are lined up on both sides. Once aligned, fully tighten both sides. Ensure the Bar (G) is set at the same height on both sides and the Screw (H) goes through part (A) and (C). **NOTE:** The position of horizontal Bar (G) will change with each height setting.

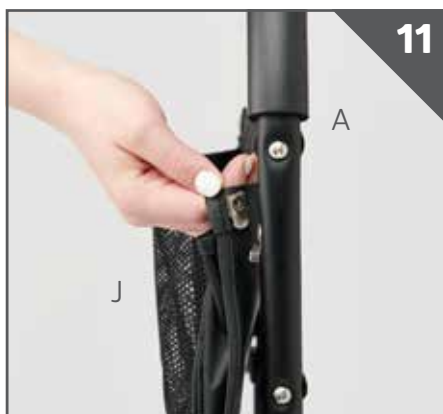
Figure 11

11A. If present, remove the disks from the metal clips on the Assist Bar (A).
11B. Attach the Storage Pocket (J) to the 2 clips on the outside of the Assist Bar (A).

Figure 12

12. Wrap the self fastening straps around the poles of the Assist Bar (A) to secure bag to the bar.

Installation:

Figure 13

13. Take the mattress off the bed. Place the assembled bed assist bar at the desired location on your box spring, at least 12½ in. from any headboard or footboard. The bed assist bar will be between the box spring and the mattress. The legs may need adjusting depending on the height of your bed frame and box spring. Make sure that the legs are firmly touching the ground.

Figure 14

14. Once in place, take strap attached to the Brace Bar (E) and thread it from the side it is attached under the loop on the far side of the Base Frame (D) and over the thick part of the frame (see Figure 13). Next, run the strap down the far side of the box spring, beneath the box spring, and then back up the near side and connect it to the buckle.

Figure 15

15. Fully tighten the strap so that the bed assist bar cannot be moved away from the bed. Make sure the strap is tight at **ALL TIMES**.

Figure 16A

16. After ensuring that the strap is flat and secure, place the mattress back on the bed. Take the key and lock the buckle by turning the key into the locked position. Make sure the buckle is **LOCKED** at all times. After buckle is locked, remove key and place in conspicuous location that cannot be accessed by individuals that should not have key access.

NOTE: If the key is lost, a flat head screwdriver can be used to lock or unlock the buckle.

Figure 16B

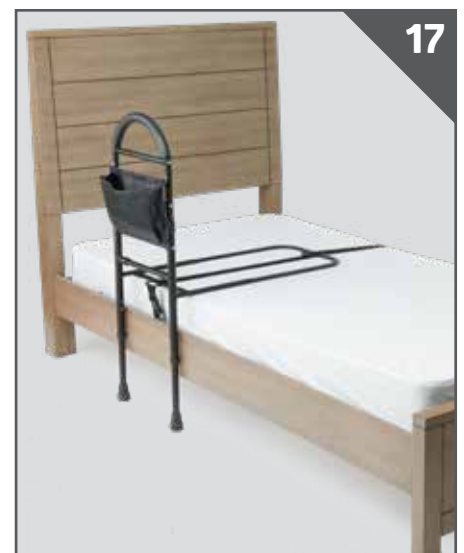
Figure 17

17. Depending on the width of your mattress the top of the assist bar might need height adjustment, adjust to desired height by removing both Screws (H) using Hex Key (I) and repeating steps 8, 9 and 10.

Operating Instructions

Getting in or out of bed:

- Grasp the handle bar on the padded section or near the top of the rail.
- Transfer in or out of the bed while grasping the handle or bar as an assist

Repositioning in the bed:

Grasp the rail by the padded section or horizontal bar to assist in repositioning.

During use:

- **DO NOT** grab the bag or tubing near the bag attachment
- **DO NOT** apply full body weight to the bar.
- Check that the rail and components did not come loose or require any adjustments after use.

Maintenance and Adjustments

- Check after each use for damaged or loose components and ensure that ALL parts are secure.
- Make sure that the strap remains tight around the box spring at ALL times. Re-adjust and tighten if the bar shifts.
- Make sure the buckle is locked at all times. If the key is lost, a flat head screwdriver can be used to lock or unlock the buckle.
- If any adjustments are necessary refer to the assembly and installation instruction section.
- Check that there are no obstructions to normal, safe operation.

NOTE: If any part or mechanism appears to be unsafe or damaged, **DO NOT USE THE PRODUCT**. If you experience problems, please contact Medline at 1-800-MEDLINE.

In addition to contacting the manufacturer directly, consumers should report problems to the CPSC at its website SaferProducts.gov or call 1-800-638-2772, or to the FDA at 1-800-332-1088.

Cleaning

1. Clean with regular household products. Use a non-abrasive cleaner with warm water. Wipe dry after cleaning. Leaving moisture on the device may cause it to rust.
2. **DO NOT** use abrasive powders, cleaners or scouring pads.

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Limited Warranty

Your Medline bed assist bar is warranted to be free of defects in material and workmanship for a lifetime for the original consumer. This device was built to precise standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the material and workmanship of our products. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse, negligence, or normal wear and tear. The warranty does not extend to non-durable components such as plastic accessories, tips and grips, which are subject to normal wear and replacement. Nondurable components are warranted for 6 months. If you have a question about your Medline product or this warranty, please call us at 1-800-MEDLINE. This warranty gives you specific legal rights, and you may have other rights, which vary from state to state.

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Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2024 Medline Industries, LP. Medline is a registered trademark of Medline Industries, LP.

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