

Bed Assist Bar Warning Notice



This notice is to supplement product references in the DME catalog or printed material and digital content, to bring awareness of inherent risks and warnings associated with Adult Portable Bed rails, referred to as Bed Assist Bars. This is in accordance with ASTM F3186-17, "Standard Specification for Adult Portable Bed Rails and Related Products" and 16 CFR Part 1270, "Safety Standard for Adult Portable Bed Rails".

Bed Assist Bars

Item No.	Description	Handle Material	HCPCS	Retail Pkg.	Pkg.
BA6800	Bed Assist Bar, Black	Foam	E0135	Y	1 ea
MDS6800BAM	M-Shape Handle, White	Foam	E0315	N	1 ea
MDS6800BAMMS	Martha Stewart Bed Assist bar	Foam	E0315	Y	1 ea
MDS6800SWIV	SwivAssist Bar, Black	Foam	E0315	N	3/cs
MDS6800SWIVRH	SwivAssist Bar, Retail	Foam	E0315	Y	1 ea



MDS6800BAMMS



MDS6800SWIV



MDS6800BAM



BA6800

⚠ WARNING

Entrapment, strangulation, suffocation and fall hazards

Gaps in and around this product can entrap and kill. People with Alzheimer's disease or dementia, or those who are sedated, confused, or frail, are at increased risk of entrapment and strangulation.

People attempting to climb over this product are at increased risk of injury or death from falls. Always make sure this product is properly secured to bed. If product can move away from bed or mattress, it can lead to entrapment and death.

⚠ WARNING

Suffocation/strangulation/entrapment hazard

If product is installed incorrectly or moves from its initial position, gaps can occur which can entrap and kill. People with Alzheimer's disease, dementia or other neurological conditions, or those who are sedated, confused, or frail, are at increased risk of entrapment, suffocation and strangulation.

- **NEVER** use unless product is tight against mattress, without gaps, and at least 12½ in. from headboard and footboard.
- **NEVER** use with children.
- **NEVER** use on toddler, bunk, water, or inflatable beds, or on beds with mattress toppers or soft compressible pads.

