MEDLINE ELEMENTS™

BEDSIDE COMMODE INODORO PORTÁTIL CHAISE D'AISANCE

PLEASE READ CAREFULLY BEFORE USING YOUR MEDLINE COMMUNE.

To ensure the safe installation, adjustment, and operation of Medline Elements Commode, these instructions **MUST** be followed.

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General Warnings

DO NOT install this product without first reading and understanding this instruction manual. If you are unable to understand these warnings, cautions, and instructions, contact a healthcare professional or dealer before attempting to install this equipment.

Instructions **MUST** be followed. **FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT.**

Warnings:

- ✓ Toilet seat must be in down position before use. Failure to do so may result in breakage of unit and possible injury to user.
- ✓ Inspect all parts for shipping damage. In case of shipping damage, do not use.
- ✓ Make certain that the snap buttons fully protrude through the same respective adjustment hole of each leg extension. This ensures that the leg extensions are securely locked in position and an even height adjustment is achieved.
- ✓ Ensure all screws, nuts and/or bolts are tight at all times.
- ✓ Check the rubber tips on the leg extensions for rips, wear or if they are missing. Replace them immediately if any of these conditions exist.
- ✓ All four leg extensions with rubber tips must touch the floor at all times.
- ✓ Users with limited physical capabilities should be supervised or assisted when using a commode.

DO NOT exceed the maximum Weight Capacity or serious injury could result: Maximum weight capacity: 400lbs (181kg).

During use, evenly distribute weight between the arm supports.

Cleaning

1. The bedside commode features built-in Microban® antimicrobial protection to help prevent the growth of stain and odor causing bacteria, mold and mildew to keep it cleaner for longer. *
2. The bedside commode can be easily cleaned with regular household cleaning products. Use a non-abrasive cleaner with warm water. **DO NOT** use abrasive powders or scouring pads.

Maintenance

- » Periodically inspect the bedside commode for damage or loose components.
- » Ensure all push buttons are protruded from the extension legs.
- » Check the rubber feet on the leg extensions for rips or wear. If any of these conditions exist, or if they are missing, replace them immediately.

Assembly Instructions

Ensure that you have all the necessary components before assembling your Medline Elements™ Bedside Commode.

- A** 1 Commode compartment (bucket, splash guard, and lid are all inside)
- B** 2 Leg Mainframes
- C** 8 hand screws with washer
- D** 4 leg extensions
- E** 2 armrest handles (one with toilet paper holder)
- F** 2 backrest extension bars
- G** 2 Small Hand screws
- H** 1 Backrest
- I** Bucket
- J** Bucket Lid
- K** Splash guard (not pictured)
- L** 4 Bronze braces (connected to commode compartment (A))

NOTE: Upon receipt of the package, check ALL parts for shipping damage. If shipping damage is noted, DO NOT USE. Call Medline to report the issue immediately at 1-800-MEDLINE.

PART ONE: Attaching Legs to main commode compartment

Below parts will be needed to assemble the legs to the main commode compartment and seat.

Step 1: Line up the leg mainframes (B) with the main commode compartment (A).

Start by flipping the commode compartment (A) upside down. Match up the holes on the leg mainframes (B) with the holes on the bronze brace (L) on the outside of the commode compartment (A). The bronze braces (L) will be on the inside of the leg mainframes (B). Ensure the tubular slot protruding from the back of the leg mainframe is placed toward the back end of the commode.

Step 1

Step 2

Step 2: Use hand screws (C) to connect the leg mainframes (B) to the commode compartment (A).

Insert hand screws (B) where holes align on the leg main frame (B) and bronze brace (L). Hand tighten. 4 hand screws (C) will be needed to fasten each leg main frame (B).

See below for additional pictures on how to apply hand screws.

Picture 1: Insert screw through the leg main frame first and then through the bronze brace.

Picture 2: Apply washer over screw and up against the bronze brace.

Picture 3: Take plastic screw piece and hand tighten over screw pressing washer up against bronze brace.

Picture 4: Hand Tighten till hand screw is secure.

PIC 1

PIC 2

PIC 3

PIC 4

NOTE: It is recommended that all four hand screws (C) for a single leg mainframe (B) be loosely crewed into place. Once all the screws are in place then fully tighten through the aligned holes.

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Depress the push-buttons on the mainframe and place feet leg extensions (D) over leg mainframe (B) until secured at desired height position. Repeat with three remaining legs. Make sure all four extension legs are installed at the same height.

Make certain that the snap buttons fully protrude through the same respective adjustment hole of each leg extension. This ensures that the leg extensions are securely locked in position and an even height adjustment is achieved.

Step 3

PART TWO: Assemble the Armrests

Step 1

Step 2

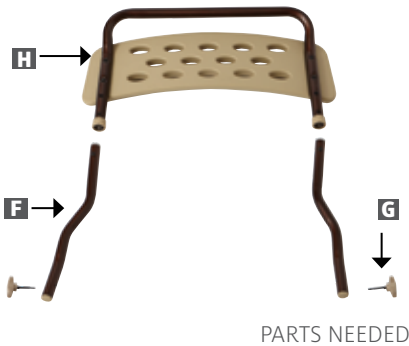
Step 1: Attach armrest (E) to the topside of the leg mainframes (B).

Flip commode over so it is resting with all four feet securely on the floor. Ensure that the legs are all set to the same height level. Insert one of the armrests (E) down into the upward facing tubes of the leg mainframes (B). Once inserted, press down the push buttons on both sides of the armrest (E) and continue to slide into the leg mainframe tube (B) until the push-buttons lock into place.

Make sure push-buttons are fully protruded before using. Repeat this step with the other armrest.

NOTE: Apply the armrests (E) so the handles (top portion) are angled outward creating more clearance between the two armrests (E).

PART THREE: Assemble the Backrest

STEP 1

STEP 2

Step 1: Attach the Backrest (H).

Slide the backrest (H) down over the backrest extension bars (F). Press down the push buttons on the backrest extension bars (F) and slide the backrest (H) down over the push buttons until the backrest locks into place. Make sure push-buttons are fully protruded before using.

Step 2: Attach the backrest extension bars (F) to the back of the leg mainframes (B).

Insert one of the backrest extension bars (F) into the tubular slot protruding from the back of the leg mainframes (B). Once inserted align the holes and tighten the small hand screw (G). HAND TIGHTEN ONLY.

Repeat this step with the second backrest extension bar (F) on the other side.

Medline Elements™ Bedside Commode has multiple adjustment points to create a custom fit that keeps you comfortable while bathing. Use the push-buttons on the various adjustable features to find the setting that you like best. Once you have made an adjustment please make sure push-buttons on both sides are fully protruded and secure before using.

Adjustment points:

.5" (1.27 cm) increment adjustable seat height.

Armrests can be removed if preferred.

Height adjustable seat.

Height adjustable backrest.

Height adjustable armrests.

Raise lid when commode is to be used

Close lid when not in use to keep toileting features hidden

The Medline ELEMENTS™ Bedside Commode has a large, flat seat cover that conceals the toilet seat and commode bucket when not in use. Concealing the commode function when not in use promotes dignity for the user and a less institutional look.

Cleaning a commode after it is used is an unpleasant job for caretakers. In addition, being cleaned up after can be a source of embarrassment for those using the commode. Medline Commode Liner with Absorbent Pad answers those concerns.

- Safe collection
- Solidifies liquids
- Eliminates Odor
- Easy disposal of body fluids

The Medline Commode liner will allow you to avoid unpleasant hand cleaning of the commode pail after use.

REF# MDS89664LINER

12 liners per Box, 6 boxes per case

Medline Commode Liners sold separately. Purchase Medline Commode Liners where ever the Medline Elements™ Bedside Commode is sold.

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LIFETIME LIMITED WARRANTY

Your Medline brand product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, which are subject to normal wear and replacement. Non-durable components are warranted for 6 months. If you have a question about your Medline product or this warranty, please call us at 1-800-MEDLINE. This warranty gives you specific legal rights, and you may have other rights, which vary from state to state.

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*This information is based upon standard laboratory tests and is provided for comparative purposes to substantiate antimicrobial activity for non-public health applications. Microban technology is not designed to replace normal cleaning practices. Antimicrobial action limited to product surface.

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Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual.

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© 2019 Medline Industries, LP. Medline Elements is a trademark of Medline Industries, LP. MICROBAN is a registered trademark of Microban Product Company Corp./Medline Elements es una marca comercial de Medline Industries LP. MICROBAN es una marca comercial registrada de Microban Product Company Corp./Medline Elements est une marque de commerce de Medline Industries LP. MICROBAN est une marque déposée de Microban Product Company Corp.