



COVID-19/FLU A&B

BINAXNOW™ COVID-19/FLU A&B

RAPID INSIGHTS, INFORMED ACTION

ONE SWAB. THREE RESULTS.

Respiratory symptoms often overlap, which is why clarity is crucial. BinaxNOW™ delivers clear, reliable results for COVID-19, Flu A, and Flu B in 15 minutes from a simple nasal swab, blending point-of-care speed, trusted quality, and cost-efficient performance for today's fast-paced practices.



NO EXTRA STEPS

Simple test procedure using a nasal swab, no instrument or serial testing required



RESULTS IN 15 MINUTES

Enabling you to diagnose and treat patients in a single visit



RESULTS YOU CAN TRUST

Provides three actionable insights right at the point of care




FAST, EASY TEST PROCEDURE*

Refer to complete Quick Reference Instructions before performing the test.


SWAB:

Rotate swab against inside of nostril, making at least 5 circles; repeat in other nostril.

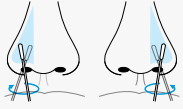


STIR:

Completely immerse swab in buffer tube solution, then vigorously mix by rotating 10 times.

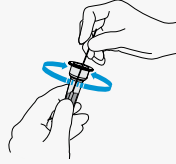


ADD DROPS:

Attach the cap onto the tube, then squeeze only 4 drops into the sample well.

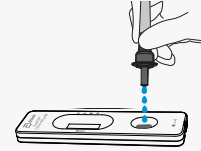


WAIT:

Read test result after 15 minutes to see COVID-19 and flu results.

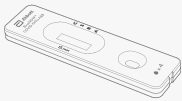

If the control line at C is visible, and any other line or multiple lines on S, B and/or A appear, the test is positive for that virus.

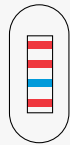

C = control (red line)
S = COVID-19 (SARS-CoV-2) (red line)
B = influenza B (blue line)
A = influenza A (red line)

PERFORMANCE*

	PPA	NPA
COVID-19 (SARS-CoV-2)	90.6%	99.4%
INFLUENZA A	89.7%	98.8%
INFLUENZA B	86.0%	99.7%

Direct nasal swabs taken within 4 days of symptom onset from individuals suspected of COVID-19, influenza A and /or influenza B.

NPA = negative percent agreement
PPA = positive percent agreement

KIT CONTENTS

- 25 test cassettes
- 25 sterile swabs
- 25 extraction buffer tubes and filter caps
- 1 Quick Reference Instructions

ORDERING INFORMATION

COVID-19/FLU A & B

TEST	UNIT OF MEASURE	PRODUCT CODE	CPT CODE*
BinaxNOW™ COVID-19/Flu A&B Test Kit	25 tests	20002054	Coming soon
BinaxNOW™ COVID-19/Flu A&B Test Control Swab Kit	10 positive, 10 negative	20002055	—

*www.codemap.com/abbottpoc/



FOR MORE INFORMATION,
CONTACT YOUR LOCAL ABBOTT
REPRESENTATIVE OR VISIT
GLOBALPOINTOF CARE.ABBOTT

*Refer to the product Instructions for Use or Quick Reference Instructions for detailed instructions and information on performance data.

© 2025 Abbott. All rights reserved. All trademarks referenced are trademarks of either the Abbott group of companies or their respective owners. Any photos displayed are for illustrative purposes only. RDx-25003957 10/25

