

# LiquiBand® Exceed™

## Simple, precise wound closure



# McKesson LiquiBand® Exceed™

McKesson LiquiBand® Exceed™ Topical Skin Adhesives are sterile, single-use octylcyanoacrylate tissue adhesives. Now offered in two sizes, both are designed for the closure of trauma and major surgical wounds. This liquid bonding agent aims to be as effective as stitches in holding many cuts, wounds and incisions together, and may also eliminate the need for dressings. LiquiBand is designed to seal the wound quickly as it forms a strong microbial shield to help protect wounds while they heal.

## LiquiBand® Exceed™ XS

- ▶ 7cm Yield
- ▶ No clog tip
- ▶ Ideal for small incisions, laparoscopic port sites



## LiquiBand® Exceed™

- ▶ 30cm Yield
- ▶ 90mins intraoperative reuse
- ▶ Ideal for medium to long surgical incisions



## Key Benefits

- ✓ **High Viscosity**<sup>3</sup>
  - ▶ Provides optimal flexibility
- ✓ **Controlled Expression of Adhesive**<sup>4</sup>
  - ▶ Applicator offers control during expression of adhesive
- ✓ **Microbial Barrier**<sup>5</sup>
  - ▶ Acts as a barrier against gram-positive, gram-negative and fungal microbes. Proven effectiveness against S. aureus, Paeruginosa, E.coli, Candida albicans and MRSA
- ✓ **No Clog Formulation**<sup>6</sup>
  - ▶ Allows users to work at their own pace
- ✓ **Maintains wound closure throughout healing process**<sup>9</sup>
  - ▶ The adhesive will stay on the skin for 5-10 days and sloughs off naturally



### Giving protection against:

Gram Positive Bacteria	Gram Negative Bacteria	Fungi
▶ S. Epidermidis	▶ Ps. Aeruginosa	▶ Asp. Brasiliensis (niger)
▶ S. Aureus & MRSA	▶ E. Coli	▶ C. Albicans
	▶ E. Cloacae	

## LiquiBand® Exceed™ XS

Size	Qty Per Box	Product Code
0.4mL	10	122-LBXS

## LiquiBand® Exceed™

Size	Qty Per Box	Product Code
0.8mL	10	122-LBX

The product information contained in this document, including the product images and additional product materials, was collected from various supplier sources. All product claims and specifications are those of the product suppliers and have not been independently verified by McKesson Medical-Surgical or its affiliates ("McKesson"). McKesson is not responsible for errors or omissions in the product information.

The properties of a product may change or be inaccurate following the posting or printing of the product information in the document, either in the print or online version. Caution should be exercised when using or purchasing any products from McKesson's online or print documents by closely examining the product packaging and the labeling prior to use.

Due to product changes, information listed in this document is subject to change without notice. This information is placed solely for your convenience in ordering and McKesson disclaims all responsibility for its completeness and accuracy, whether or not the inaccuracy or incompleteness is due to fault or error by McKesson.

## Application Guide *LiquiBand® Exceed™*

LiquiBand® Exceed™ Topical Skin Adhesive has a unique, felt applicator tip to facilitate even, consistent application in three simple steps.

### ● STEP ONE

- ▶ While holding upright, squeeze the applicator wings until a cracking sound is heard.



### ● STEP TWO

- ▶ Invert the tip and gently squeeze wings to prime the device. When the tip turns violet, LiquiBand® Exceed™ is ready to use.
- ▶ Ensure that wound is thoroughly clean and dry prior to closure.



### ● STEP THREE

- ▶ While holding the wound edges together, apply an even coat of LiquiBand® Exceed™ over the length of the wound.
- ▶ Continue to hold the wound edges together for 60 seconds.



- ❗ If using on the eye area, adjust the patient's position. Use petroleum jelly to create a barrier close to the wound, and use surgical gauze to protect the patient's eyes.

Unless otherwise noted, the recommendations in this document were obtained from: Advanced Medical Solutions. Be advised that information contained herein is intended to serve as a useful reference for informational purposes only and is not complete clinical information. This information is intended for use only by competent healthcare professionals exercising judgement in providing care. McKesson cannot be held responsible for the continued currency of or for any errors or omissions on the information.  
© 2018. LiquiBand® and LiquiBand® Exceed™ are registered trademarks of Advanced Medical Solutions Limited

START ORDERING TODAY! VISIT [MMS.MCKESSON.COM](http://MMS.MCKESSON.COM) OR CONTACT YOUR ACCOUNT MANAGER.

McKesson Medical-Surgical  
9954 Mayland Drive  
Richmond, VA 23233  
[mms.mckesson.com/mckesson-brands](http://mms.mckesson.com/mckesson-brands)

Post-Acute Care and Home Health  
800.347.2456

Home Medical Equipment  
888.822.8111

Primary Care  
866.625.2679

McKesson Brands Support  
800.777.4908

With McKesson Brands, you'll never have to compromise. We deliver quality products at competitive prices, backed by our 100% satisfaction guarantee. We have decades of experience in the market, and we offer you support and work to help ensure you have the products you need, when you need them—making it even easier for you to provide the best care for your patients.

**■ Satisfaction Guaranteed**

If you are not completely satisfied with any McKesson Brands product, you may return it for a full refund or credit.

The product information contained in this document, including the product images and additional product materials, was collected from various supplier sources. All product claims and specifications are those of the product suppliers and have not been independently verified by McKesson Medical-Surgical ("McKesson"). McKesson is not responsible for errors or omissions in the product information.

The properties of a product may change or be inaccurate following the posting or printing of the product information in the document, either in the print or online version. Caution should be exercised when using or purchasing any products from McKesson's online or print documents by closely examining the product packaging and the labeling prior to use.

Due to product changes, information listed in this document is subject to change without notice. This information is placed solely for your convenience in ordering and McKesson disclaims all responsibility for its completeness and accuracy, whether or not the inaccuracy or incompleteness is due to fault or error by McKesson.