

*Designed to be more sensitive than traditional guaiac-based methods.
Increased specificity to lower GI bleeding makes it an ideal tool to
screen for colorectal cancer.*

Benefits

- Ability to screen two specimens using a single test cassette & buffer tube
- Meets the American Cancer Society multiple-day testing recommendation
- No diet or drug restrictions
- Patient-friendly collection procedure
- Favorable reimbursement



Reliability

- Analytical Sensitivity: 50 ng Hb/ml buffer or ug Hb/g feces
- Specific for human hemoglobin
- Manufactured in the USA

Convenience


- 30-day room temperature specimen stability
- 24-month shelf life from date of manufacture
- Physician Office (4487) or Patient Take Home Kit (4485)

Reimbursement Codes


- G0328
Colorectal cancer screening with immunochemical FOBT
- 82274
Blood, occult, by fecal hemoglobin determination with immunochemical FOBT

Exemplar CPT and reimbursement provided. Refer to regional CMS fee schedule for relevant/current CPT codes and reimbursement rates. All reimbursement codes should be accompanied by an ICD9 code.


Patient Instructions

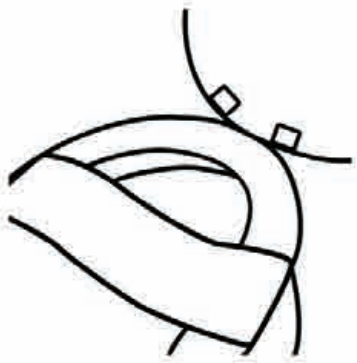


1. Lift toilet seat and position provided sample collection paper across rim of toilet bowl.

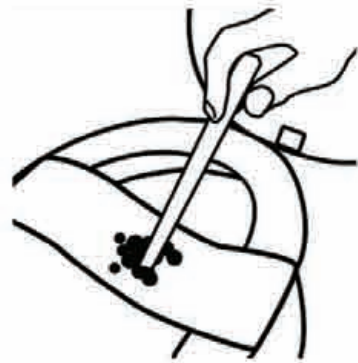


2. Deposit bowel movement onto collection paper. Collect sample from 4 different sites of the stool using one of the provided applicator sticks.

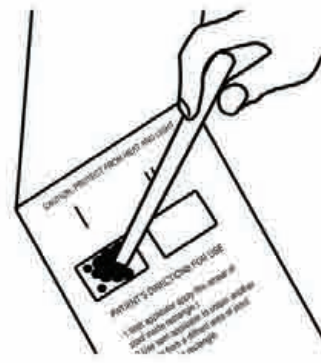

3. Place sample card on a flat surface, open flap of Consult sample card. Apply sample to window 1. Spread evenly and fill window with stool sample. Close flap, flush tissue.

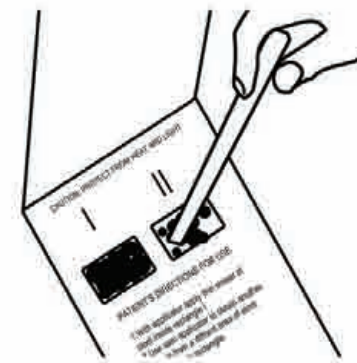

4. Repeat steps 2 thru 4 for second bowel movement and apply to collection window 2. Close the flap.

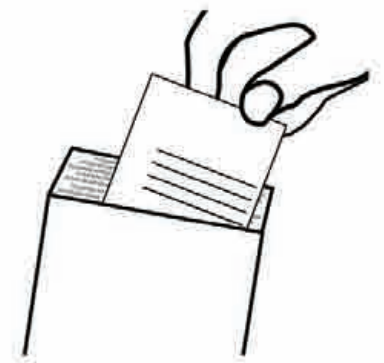

5. Insert completed Consult sample card into enclosed mailing pouch and return to healthcare provider.


Physician Instructions



Slowly peel back the sample tab.



Place sample strip into preparation buffer tube, replace cap and shake tube vigorously.



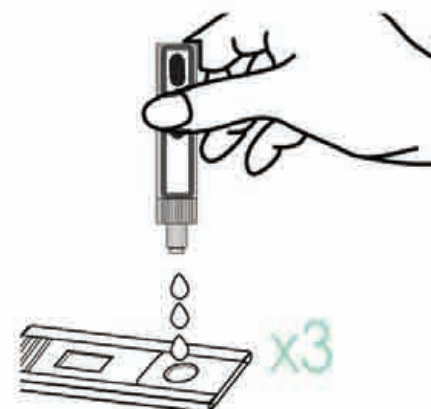


Squeezing the tube slightly, dispense three (3) drops into cassette well.



Read result at five (5) minutes.

RESULT READING

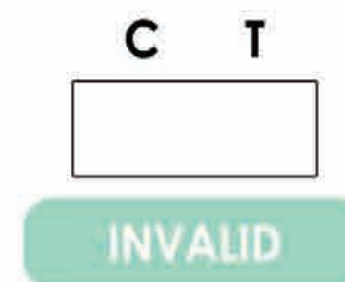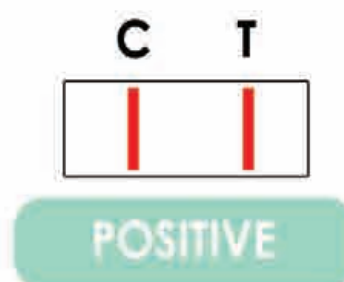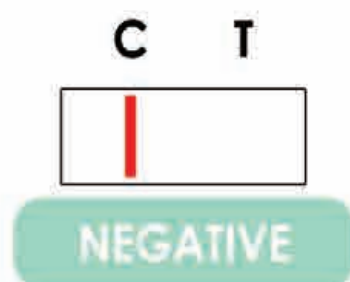

Ordering Information

iFOBT (4485)
Includes: 35 Mailing Envelopes,
35 Sample Slides w/ 2 day collection,
25 Test Cassettes and 25 Buffer Tubes

iFOBT (4487)
Includes: 25 Test Cassettes
and 25 Buffer Tubes

iFOBT (4486)
Includes: 10 Test Cassettes
and 10 Buffer Tubes

iFOBT Mailers (4488)
Includes: 20 Mailing Envelopes
and 20 Sample Slides