

Cleaning & Disinfection Instructions



For use with MFR#s:

134-108394 ErgoCart Storage Work Cart With E-Lock
134-108395 ErgoCart Storage Work Cart With Key Lock
134-108396 Deluxe Mobile Workstation

WARNING!

- Always comply with facility cleaning standards and guidelines.
- Always follow directions on cleaning product label and heed warnings.
- Before using any cleaner, do a spot test on a small, inconspicuous area on your product to ensure it will not cause damage to the finish.
- Solvents, acidic, or abrasive cleansers can damage surfaces and are not recommended.
- Wipe spills away promptly and rinse several times with water.
- Steel wool and other abrasive cleaning pads will damage the surface.
- Do not mix different cleaning agents to clean or disinfect.
- Avoid rust remover, drain cleaners, and toilet bowl cleaners. These contain harsh chemicals and will quickly cause permanent damage.
- Avoid hair, textile, and food dyes, as they can cause permanent stains.
- Disinfectants with high concentrations of isopropyl alcohol and/or bleach can be corrosive to plastic with prolonged use.

PRODUCT FOR GENERAL CLEANING

Soap and water are recommended for general cleaning needs.

PRODUCT FOR DISINFECTION

When disinfection is necessary the following products may be used:

- McKesson Disposable Germicidal Surface Wipes, MFR# 50-66160
- McKesson Disposable Germicidal Surface Wipes, MFR# 50-66170

RECOMMENDED CLEANING PROCESS

1. Wipe up any spills and remove any debris before applying cleaner.
2. Apply cleaner/disinfectant according to directions on the cleaning agent label.
3. Avoid using sprays on products with electronic components (e.g., electronic locks).
4. Ensure that cleaning solution is wiped away promptly, adhering to the label instructions.
5. Dispose used cleaning cloths and solutions.

134-108394 ErgoCart Storage Work Cart With E-Lock
134-108395 ErgoCart Storage Work Cart With Key Lock

⚠ WARNING: Risk of reproductive harm from exposure to Bisphenol A. See P65Warnings.ca.gov.