



# Clearview<sup>®</sup> RSV

Fast, accurate RSV detection  
in children under 6 and adults  
over 60



# Clearview<sup>®</sup> RSV

Clearview<sup>®</sup> RSV is a lateral flow immunogold assay for the direct visual, qualitative detection of RSV protein F in clinical samples with:



Rapid results in 15 minutes – while the patient waits.



Easy-to-use, CLIA-waived technology.




High sensitivity and specificity so you can test and treat with confidence.


## NASOPHARYNGEAL SWAB TEST PROCEDURE

Refer to product package insert for full instructions.

**1** Agitate the swab for 15–20 seconds, then roll the swab against the inside of the container as you remove it.

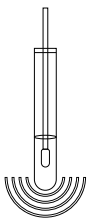


**2** Draw sample to the premeasured line on the disposable pipette.

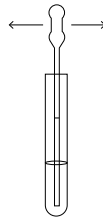

**3** Empty the pipette into the extraction tube.




**4** Add 4 drops of extraction reagent and swirl to mix.

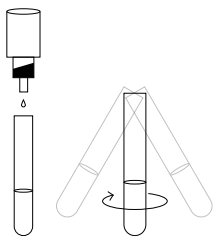

**5** Draw all liquid into a clean, disposable pipette, then squeeze onto the test device.



**6** Read the test results at 15 minutes. Do not read the test results after 25 minutes.

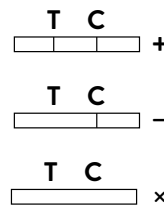

## PRODUCT INFORMATION

Information Type	Product Detail
Format	Cassette
CLIA complexity	Waived
Sample types	Nasopharyngeal swab, nasal aspirate (wash)
Time to results	15 minutes
Age limitations	Children under 6/ adults over 60
Shelf life*	12 months
Storage conditions	2°C–30°C (36°F–86°F)
External controls	Included in test kit

\*From date of manufacture.

## REIMBURSEMENT

Reimbursement codes, coverage determinations and other resources are available at:

[codemap.com/abbottpoc](http://codemap.com/abbottpoc)

## ORDER INFORMATION

Clearview <sup>®</sup> RSV	Order Number
20 Cassettes – CLIA Waived	135-060

To learn more, contact your local sales representative or visit [globalpointofcare.abbott](http://globalpointofcare.abbott)

