REF MDSC40018

# ComfortGlide® Sling | Toile | Eslinga

## Disposable Repositioning Sling

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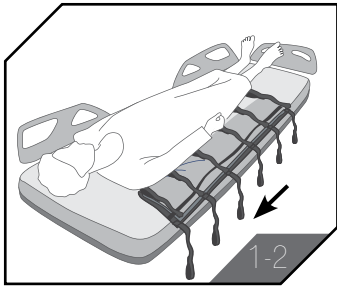
## Instructions for Using the ComfortGlide® Sling to Reposition:

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**1. Unfold and position the ComfortGlide® Sling next to patient, align the top of the sheet with the patient's shoulders.**

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**2. Place a ComfortGlide® Drypad (if applicable) over the ComfortGlide® Sling and fold the edges of both longways to tuck under the patient.**

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**3. Roll patient onto his/her side following patient handling policies.**

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**5. On the opposite side of the bed, roll patient away from you and unfold the ComfortGlide® Sling and pad.**

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**6. Using the handles along the perimeter of the ComfortGlide® Sling, gently slide the patient as needed to center them on the bed.**

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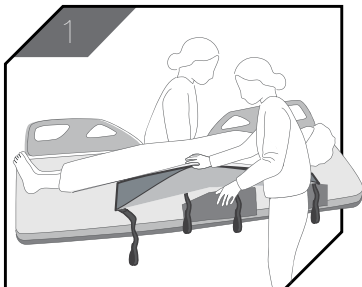
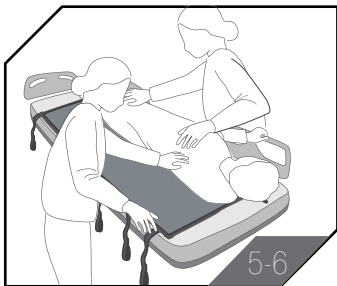
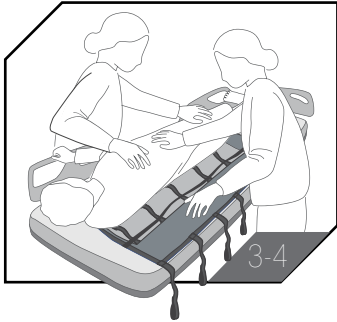
**\*If utilizing Fitted sheet, apply prior to step 1**

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**1. Log-roll patient onto his/her side following patient handling policies. Place ComfortGlide® Wedges underneath : the ComfortGlide® Sling, with "DOWN" arrows pointing towards the mattress. One wedge should be above the sacrum and the other below as shown.**

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**2. Slowly roll patient back on top of the wedges. Sacrum should be fully off-loaded between both wedges, leaving about a "fist" width of space between them.**

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**3. Check that there are no folds and wrinkles under the patient and smooth out as needed.**

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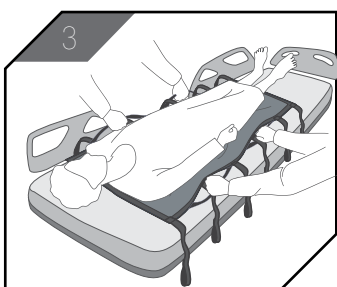
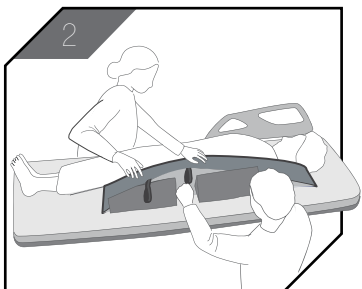
**4. Adjust the bed to the desired angle and ensure the patient is in the proper position.**

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**CAUTION: This sling must be inspected prior to each use for any rips, tears, frayed, or bleached areas which are unsafe and could result in injury. Do not exceed max weight specified. Refer to owner's manual for operation of your lift. Do not launder!**

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**The ComfortGlide® Sling has 6 sling straps on each side of the glide sheet. Intended for use with loop-style cradles. This sling cannot be used with clip style cradles.**

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**1. Fold the sling in half, and log-roll patient onto his or her side following patient handling policies. Tuck sheet beneath the patient. Check that the top of the sling is aligned above the patient's head to support the head during lifting.**

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**2. On the opposite side of the bed, log-roll the patient away from you and pull the rest of the sheet through.**

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**3. Attach sling loops to the lift and raise the lift until the patient is clear of the bed. If using a 6-point cradle, attach two sling straps per hook. If using a 4-point cradle, attach 3 sling straps per hook. Ensure the same color loop is used on both left and right sides, to keep the patient balanced in the sling. Note that attaching the shorter loops near the patient's head and longer loops near the patient's legs will achieve a more upright position for the patient.**

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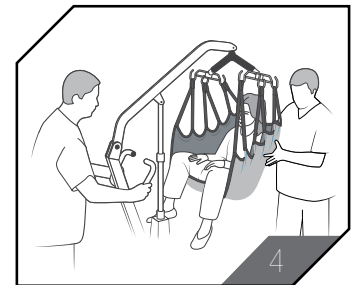
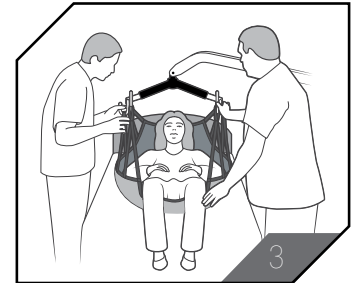
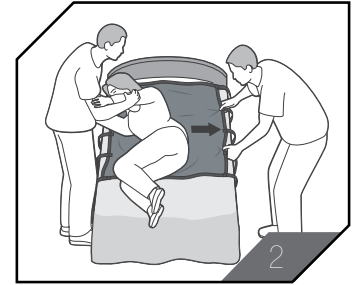
**4. Perform patient transfer following patient handling policies.**

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**Do not attach side handles to a mechanical lift, only use sling straps to attach this product to a mechanical lift.**

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**1. For best outcomes, only use in combination with ComfortGlide® Sheets, Wedges and Drypads.**

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**2. ATTENTION: If using wedges with ComfortGlide, do not use a ComfortGlide® sheet to slide the patient onto the wedges. Use log-roll method to place wedges under the patient.**

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**3. Always follow your facility's guidelines for proper patient handling.**

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(Fr) Capacité de 453 kg (1,000 lb)

**Medline Industries, LP**

Three Lakes Drive, Northfield, IL 60093 USA

1-800-MEDLINE

medline.com | info@medline.com

FOLLOW US | SÍGANO | SSUIVEZ-NOUS

### Medline United States

1-800-MEDLINE (633-5463)

www.medline.com | info@medline.com

### Medline Canada

1-800-268-2848

www.medline.ca | canada@medline.com

### Medline Mexico

01-800-831-0898

www.medlinemexico.com | mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this instruction sheet. ©2023 Medline Industries, LP. Medline and Excel Translator are registered trademarks of Medline Industries, LP.

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