# ComfortGlide® TRP Repositioning Sheet

## User's Manual

Optional Retention Straps

Microfiber Top

Breathable Material

Ergonomic Handles

Low-Friction Nylon


**REF** CGSHEETXLD

### Available Components:

MSC50024 – ComfortGlide XL Wedges

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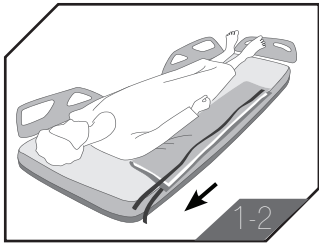## Instructions for Use / Instrucciones de Uso / Mode d'emploi :

**1. Unfold and position the ComfortGlide® Sheet next to patient, align the top of the sheet with the patient's shoulders. The 2 retention straps(if applicable) should be towards the head of the bed.**

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**2. Place a ComfortGlide® Drypad (if applicable) over the glide sheet and fold the edges of both long-ways to tuck under the patient.**

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**3. Roll patient onto his/her side following patient handling policies.**

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**4. Tuck the glide sheet under patient's side.**

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**5. On the opposite side of the bed, roll patient away from you and unfold the glide sheet and pad.**

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**6. Using the glide sheet handles, gently slide the patient as needed to center them on the bed.**

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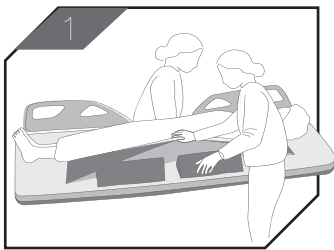
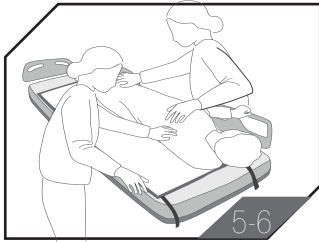
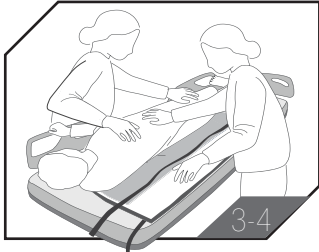
**\*if utilizing Fitted sheet, apply prior to step 1**

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**1. Log-roll patient onto his/her side following patient handling policies. Place ComfortGlide® Wedges underneath the ComfortGlide® Sheet, with "DOWN" arrows pointing towards the mattress. One wedge should be above the sacrum and the other below as shown.**

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**2. Slowly roll patient back on top of the wedges. Sacrum should be fully off-loaded between both wedges, leaving about a "fist" width of space between them.**

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**3. Check that there are no folds and wrinkles under the patient and smooth out as needed.**

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**4. Adjust the bed to the desired angle and use the dual retention straps(if applicable) to secure the top of the glide sheet to the bed. There are two ways to properly secure the straps.**

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**(1). Secure them to an opening in the bed frame, or**

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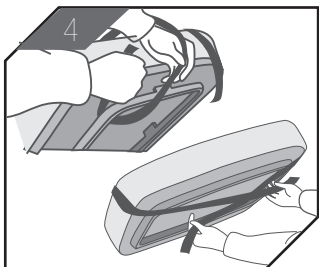
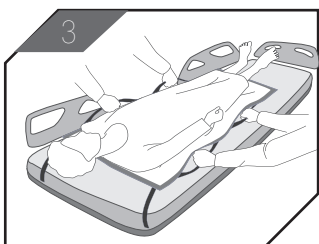
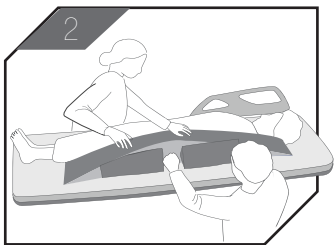
**(2). If there is no bed structure within reach, attach the straps to each other around the bottom of the mattress.**

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## To Clean / Para Limpiar / Nettoyage :

**ComfortGlide® Sheet:** Wipe to spot clean with mild detergent as needed. Discard if soiled or damaged.

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**ComfortGlide® Reusable Glide Sheet:** Refer to particular product for washing instructions.

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**ComfortGlide® Wedges:** Wedges may be cleaned according to your infection prevention policies. Discard if soiled or damaged.

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**ComfortGlide® Fitted Sheets:** Refer to particular product for washing instructions.

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### 1. ComfortGlide components are not designed for patient lifting.

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### 2. Not to be used in combination with other lifting or positioning devices.

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### 3. For best outcomes, only use in combination with ComfortGlide® Sheets, Wedges and Drypads.

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### 4. ATTENTION: If using wedges with ComfortGlide, do not use a ComfortGlide® sheet to slide the patient onto the wedges. Use log-roll method to place wedges under the patient.

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### 5. Always follow your facility's guidelines for proper patient handling.

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