# ComfortGlide® TRP Repositioning Sheet

## User's Manual


**REF** CGLIDESHEETNS

### Available Accessories:

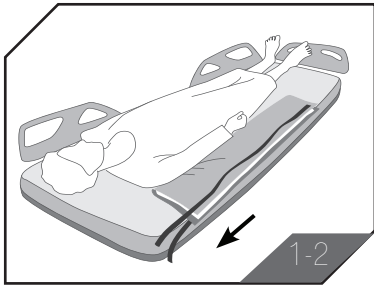MSC50023 - ComfortGlide Wedges  
CGLIDEPAD - ComfortGlide Drypad

## Instructions for Use / Instrucciones de Uso / Mode d'emploi :

**1. Unfold and position the ComfortGlide® Sheet next to patient, align the top of the sheet with the patient's shoulders.**

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**2. Place a ComfortGlide® Drypad (if applicable) over the glide sheet and fold the edges of both long-ways to tuck under the patient.**

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**3. Roll patient onto his/her side following patient handling policies.**

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**4. Tuck the glide sheet under patient's side.**

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**5. On the opposite side of the bed, roll patient away from you and unfold the glide sheet and pad.**

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**6. Using the glide sheet handles, gently slide the patient as needed to center them on the bed.**

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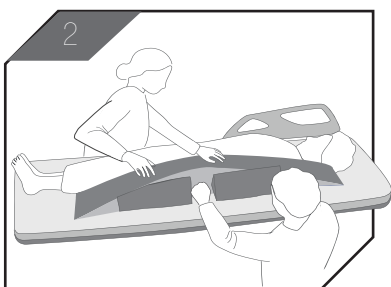
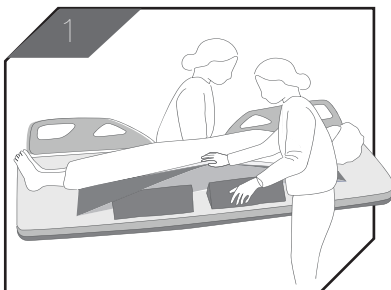
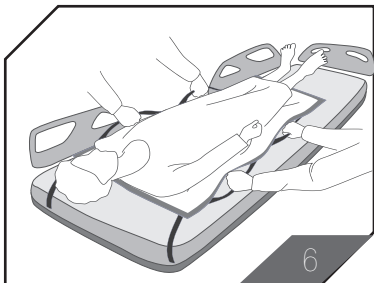
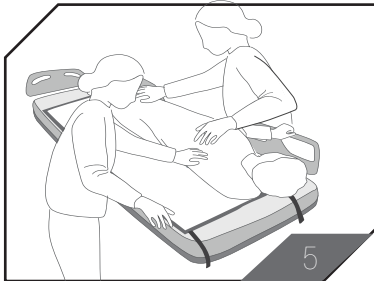
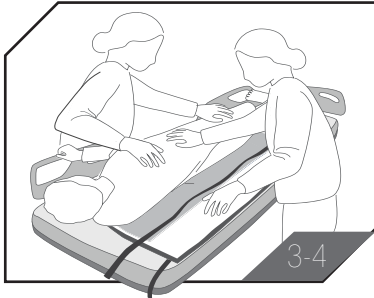
**\*if utilizing Fitted sheet, apply prior to step 1**

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**1. Log-roll patient onto his/her side following patient handling policies. Place ComfortGlide® Wedges underneath the ComfortGlide® Sheet, with "DOWN" arrows pointing towards the mattress. One wedge should be above the sacrum and the other below as shown.**

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**2. Slowly roll patient back on top of the wedges. Sacrum should be fully off-loaded between both wedges, leaving about a "fist" width of space between them.**

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**3. Check that there are no folds and wrinkles under the patient and smooth out as needed.**

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### 1. Ensure the transfer surfaces are as close together as possible.

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### 3. Lower the receiving surface slightly below sending surface.

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### 4. Ensure the patient is in the supine position, both surfaces are completely horizontal, and the necessary bed rails are lowered.

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### 5. Bridge the gap between the two surfaces with an additional bed sheet or blanket.

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### 6. Grab the glide sheet handles and slide the patient onto the receiving surface.

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### 7. Confirm the patient is center on the receiving surface once transfer is complete.

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### 8. Separate surfaces and raise bed rails, if applicable.

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## To Clean / Para Limpiar / Nettoyage :

**ComfortGlide® Sheet:** Wipe to spot clean with mild detergent as needed. Discard if soiled or damaged.

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**ComfortGlide® Reusable Glide Sheet:** Refer to particular product for washing instructions.

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**ComfortGlide® Wedges:** Wedges may be cleaned according to your infection prevention policies. Discard if soiled or damaged.

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**ComfortGlide® Fitted Sheets:** Refer to particular product for washing instructions.

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### 1. ComfortGlide components are not designed for patient lifting.

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### 2. Not to be used in combination with other lifting or positioning devices.

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### 3. For best outcomes, only use in combination with ComfortGlide® Sheets, Wedges and Drypads.

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### 4. ATTENTION: If using wedges with ComfortGlide, do not use a ComfortGlide® sheet to slide the patient onto the wedges. Use log-roll method to place wedges under the patient.

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### 5. Always follow your facility's guidelines for proper patient handling.

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