

# McKesson Comprehensive Wound Care



# Table of Contents

**02**

**McKesson Educational Resources**

**03**

**Wound Care Algorithm**

**04**

**Instructional Packaging**

**05**

**Skin Cleansing and Preparation**

- Antiseptics
- Pads and Swabsticks
- Wipes
- Wound Cleansers
- Sundries

**11**

**General Wound Care**

- Gauze and Sponges
- Adhesive Bandages
- Cohesive Bandages
- Elastic Bandages
- Fluff Bandage Rolls
- Packing Strips
- Retainer Dressings
- Tapes
- Moisture Barriers

**28**

**Clinical Guide to Assessing Wounds**

**31**

**Advanced Wound Care**

- Calcium Alginate Dressings
- Collagen
- Compression Bandages
- Contact Layers
- Cover Dressings and Composites
- Super Absorbent Dressings
- Film Dressings
- Foam Dressings
- Gelling Fiber
- Honey Dressings
- Hydrocolloids
- Hydrogels
- Hydrophilic Dressings
- Impregnated Dressings
- Skin Fold Management
- Wound Measuring Devices

**49**

**Wound Closure**

- Skin Closure Strips
- Topical Skin Adhesive

**51**

**Kits and Trays**

- Dressing Change Trays



# Skin and wound care present a variety of business and caregiving challenges.

## Here's your opportunity...

- To help save money, without compromising quality or performance.
- To access exclusive skin care products and clinical support.
- To excel skin and wound care solutions that your staff and patients deserve.

### **Satisfaction Guaranteed**

If you are not completely satisfied with any McKesson Brands product, you may return it for a full refund or credit within 12 months from the purchase date. McKesson Brands satisfaction guarantee is limited to products purchased directly from McKesson. If a McKesson Brands product is not purchased directly from McKesson, any return request or request for refund must be directed to the selling party.

## Never compromise on care<sup>SM</sup>

With McKesson Brands, you'll never have to compromise. We deliver quality products at competitive prices, backed by our 100% satisfaction guarantee. With our decades of experience and reliable, fast distribution, we help ensure you have the products you need, when you need them — making it even easier for you to provide the best care for your patients.



# Solutions for Advanced Skin and Wound Care

## EDUCATIONAL RESOURCES

### Clinical Support

At McKesson Medical-Surgical, we offer a variety of educational resources to assist you in training your staff on the challenges of wound care.

#### Online education



Our online education portal provides access to courses for CNAs and licensed professionals. Contact your McKesson Medical-Surgical Account Manager for access to our e-learning portal.

#### Product selection guides



We offer product selection guides that provide information on selecting products to meet your advanced wound care needs. Customizable versions are also available.

#### Webinars



Presentations provide educational training and can be tailored to meet your needs.

#### Assessment tools




- Skin assessment
- Wound assessment algorithm
- Healthy and necrotic tissue
- Wound infection
- Types of tissue breakdown
- Pressure injuries

**At McKesson Medical-Surgical, we offer a variety of educational resources to assist you in training your staff on the challenges of wound care. Contact our Clinical Resource Team at 877.611.0081.**

# Assessing Wounds

## Wound Care Algorithm

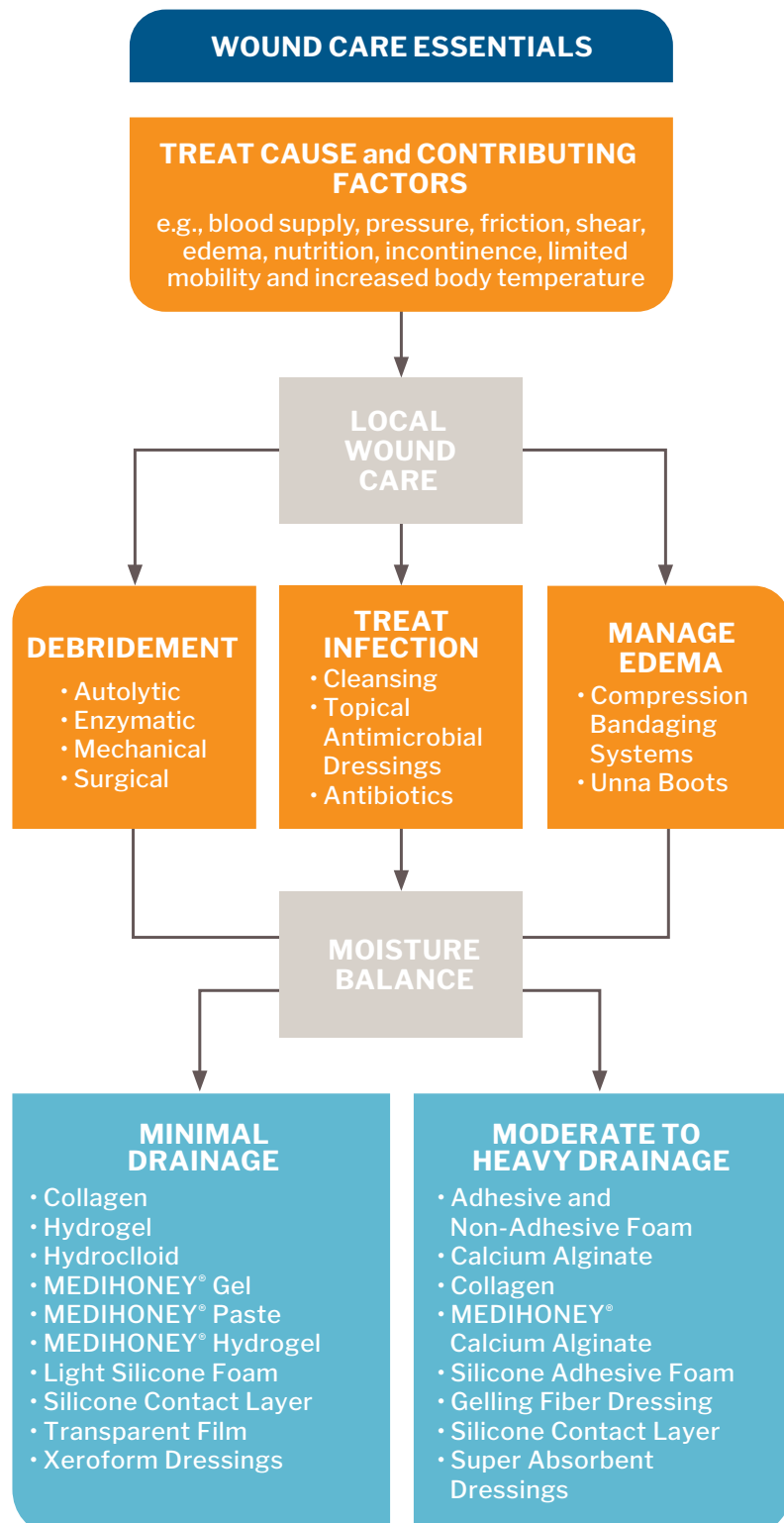

### Elements of Wound Assessment

**Location:** Refers to specific body part or bony prominence over which the wound is located.

**Etiology:** Pressure, venous, diabetic, arterial, surgical, traumatic.

**Classification:** Stage (for pressure injuries); full or partial thickness.

**Measurement:** Length x width x depth in centimeters. Assess for undermining and tunneling.

**Wound bed:** Cleanse prior to assessment. Determine tissue type (granulation, slough, eschar, epithelialization, anatomical structures) and the percentage of each tissue type.

**Exudate amount and character:** None, light, moderate, heavy. Serous, serosanguineous, sanguineous, purulent, color and consistency.

**Surrounding skin:** Normal, red/erythematous, blanchable, non-blanchable, hyperpigmented, macerated, rash, indurated, edematous.

**Wound edges:** Healthy, diffuse, well defined, undermined, rolled under (epibole), fibrotic, calloused.

**Signs of infection:** Pain, erythema, edema, heat/warmth, increased drainage, purulence, foul odor, delayed healing.

**Pain:** Assess for pain related to the wound in general, as well as how it relates to the treatment or dressing change. Use a pain scale to get a more quantitative assessment.

**Accurate documentation:** Document according to facility policy and procedure so progress to goals can be determined. Accurately document findings.

# Instructional Packaging

Based on direct feedback from clinicians, we know that properly selecting and using the right wound care products can be an ongoing challenge. That's why we've made it easier to identify the products and have provided instructions for use. We are committed to finding ways to make the treatment of wounds less challenging for you.

INSTRUCTIONS FOR USE

**MCKESSON**

Hydrocolloid Dressings create a moist wound environment, conducive to healing and autolytic debridement. These dressings also protect the wound from external moisture and contaminants such as urine and feces and are designed to minimize bunching.

**CHANGING THE DRESSING (Frequency of Change: up to 7 Days)**



- 1** Clear and assess the wound bed as appropriate. Select a dressing size 1" larger than the maximum wound dimension (length or width) to allow a minimum of 1/2" of skin contact beyond the wound margin.
- 2** Protect peri-wound skin as needed with liquid barrier film. Remove one side of the paper liner and, without tension, place dressing over wound and surrounding skin.
- 3** Remove remaining film liner in the direction of the blue arrow and, without tension, smooth dressing over wound and surrounding skin.

Avoid storage in direct sunlight/fluorescent lighting and keep area cool, dry, and well-ventilated.

Contents: **STERILE** in unopened, undamaged inner package.  
 natural rubber latex.  
 one? Call 1-800-777-4968  
 ?? Call 1-877-611-0081

**in Guaranteed**

McKesson Medical-Surgical Inc.  
 33228

**STERILE R**

**MFR # 1888**



(01)20612479192984

PRODUCT NAME

**MCKESSON**


**Hydrocolloid Dressings**  
**FILM BACKING**

PRODUCT ATTRIBUTES

SACRAL | STERILE

PRODUCT DESCRIPTOR

PRODUCT PHOTO

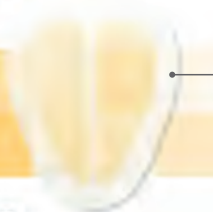

PRODUCT CALLOUTS

**6" x 7"**  
 (15.2 cm x 17.8 cm)

**1**  
 PER PACK

**5**  
 PACKS PER BOX

**DO NOT REUSE**

**MFR # 1888**

MANUFACTURER NUMBER

# Skin Cleansing and Preparation



For clinical information about our products, call 877.611.0081.  
To order, visit [mms.mckesson.com](https://mms.mckesson.com) or call your McKesson Medical-Surgical Account Manager.

## SKIN CLEANSING AND PREPARATION

### Antiseptics

#### McKesson Antiseptic Skin Cleanser

- 4% CHG surgical scrub attacks a wide range of microorganisms and is clinically proven to help reduce bacteria on the skin that can cause disease
- For use as a patient preoperative skin prep, wound cleansing and healthcare personnel hand wash
- Can be used for general skin cleansing, stronger than regular soaps
- Dye free and triclosan free
- Fresh scent



| Item Description  | MFR #    | ITEM #  | QTY             |
|-------------------|----------|---------|-----------------|
| 15 mL Packet      | 16-CHG15 | 1094352 | 50/bx, 10 bx/cs |
| 4 fl. oz. Bottle  | 16-CHG4  | 1055586 | 1/ea, 48/cs     |
| 8 fl. oz. Bottle  | 16-CHG8  | 1055587 | 1/ea, 24/cs     |
| 16 fl. oz. Bottle | 16-CHG16 | 1055588 | 1/ea, 12/cs     |
| 32 fl. oz. Bottle | 16-CHG32 | 1055590 | 1/ea, 12/cs     |
| 1 Gallon Jug      | 16-CHGGL | 1055591 | 1/ea, 4/cs      |

#### McKesson Isopropyl Rubbing Alcohol 70%

- Antiseptic



| Item Description  | MFR #    | ITEM # | QTY         |
|-------------------|----------|--------|-------------|
| 16 fl. oz. Bottle | 23-D0022 | 49176  | 1/ea, 12/cs |
| 32 fl. oz. Bottle | 23-D0024 | 556517 | 1/ea, 12/cs |
| 1 Gallon Jug      | 23-A0023 | 350600 | 1/ea, 4/cs  |

#### McKesson Hydrogen Peroxide 3%


- Topical solution USP
- Antiseptic
- For treatment of minor cuts and abrasions



| Item Description  | MFR #    | ITEM # | QTY         |
|-------------------|----------|--------|-------------|
| 4 fl. oz. Bottle  | 23-F0010 | 141564 | 1/ea, 24/cs |
| 16 fl. oz. Bottle | 23-D0012 | 142779 | 1/ea, 12/cs |
| 1 Gallon Jug      | 23-A0013 | 139307 | 1/ea, 4/cs  |

#### McKesson PVP Prep Solution

- Contains 10% povidone iodine, USP weight/volume
- Ideal as an antiseptic for use on minor skin wounds, cuts or burns
- Ideal for preparing the skin prior to surgery
- Flip-top cap

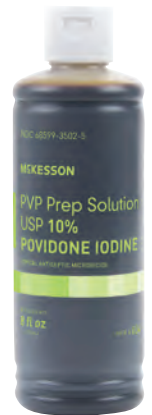

| Item Description  | MFR # | ITEM #  | QTY         |
|-------------------|-------|---------|-------------|
| 4 fl. oz. Bottle  | 039   | 1043539 | 1/ea, 36/cs |
| 8 fl. oz. Bottle  | 034   | 1043538 | 1/ea, 24/cs |
| 16 fl. oz. Bottle | 035   | 911740  | 1/ea, 12/cs |
| 1 Gallon Jug      | 036   | 854301  | 1/ea, 4/cs  |

## Pads and Swabsticks



### McKesson Isopropyl Alcohol Swabsticks

- Saturated with 70% v/v isopropyl alcohol
- For topical cleansing prior to injection prep or venipuncture

| Item Description | MFR # | ITEM # | QTY                      |
|------------------|-------|--------|--------------------------|
| Sterile 3's      | 988   | 854754 | 3/pk, 25 pk/bx, 10 bx/cs |



### McKesson Alcohol Prep Pads

- Saturated with 70% v/v isopropyl alcohol
- For topical cleansing prior to injection prep or venipuncture

| Item Description | MFR #  | ITEM # | QTY                       | HCPCS |
|------------------|--------|--------|---------------------------|-------|
| Medium, Sterile  | 58-204 | 191089 | 1/pk, 200 pk/bx, 20 bx/cs | A4245 |
| Large, Sterile   | 58-404 | 225940 | 1/pk, 100 pk/bx, 10 bx/cs | A4245 |



### McKesson Povidone Iodine, USP Swabsticks

- Acts as a topical antiseptic and germicide for preparation of the skin prior to surgery or injections
- Contains 10% povidone iodine, USP weight/volume (equivalent to 1% titrable iodine)

| Item Description | MFR # | ITEM # | QTY                      | HCPCS |
|------------------|-------|--------|--------------------------|-------|
| Sterile 1's      | 986   | 854752 | 1/pk, 50 pk/bx, 10 bx/cs | A4247 |
| Sterile 3's      | 987   | 854753 | 3/pk, 25 pk/bx, 10 bx/cs | A4247 |



### McKesson Calcium Alginate Swabsticks

- Aluminum shaft
- Specially designed for small cavities and wound care
- Ideal specimen collection tool for testing and screening
- Miniature calcium alginate tip is biodegradable

| Item Description | MFR # | ITEM # | QTY                   |
|------------------|-------|--------|-----------------------|
| Sterile          | 1338  | 883891 | 1/ea, 50/bx, 10 bx/cs |

## SKIN CLEANSING AND PREPARATION

### Wipes

#### McKesson Adhesive Remover Wipes

- Helps to remove adhesive tape, adhesive residue and adhesive ostomy products from skin
- Cleanses skin and removes adhesives and debris from peristomal skin
- Gentle, alcohol-based formula
- Packaged in convenient, individually wrapped packets



| Item Description | MFR #    | ITEM #  | QTY                   | HCPCS |
|------------------|----------|---------|-----------------------|-------|
| 2.4" x 2.4"      | 176-5729 | 1088821 | 1/ea, 50/bx, 50 bx/cs | A4456 |



#### McKesson Benzalkonium Chloride Towelettes

- McKesson BZK Towelettes are designed for general antiseptic cleansing for both patients and clinicians
- Each towelette contains 0.13% w/v benzalkonium chloride to help disinfect
- Single-use, individually packaged antiseptic towelettes are a convenient, cost-effective cleansing solution for a variety of disinfecting needs

| Item Description | MFR # | ITEM # | QTY                    |
|------------------|-------|--------|------------------------|
| 5" x 7"          | 269   | 865276 | 1/ea, 100/bx, 10 bx/cs |



#### McKesson No-Sting Skin Barrier Film

- Forms a barrier to protect the skin from adhesives, friction and body fluids
- Alcohol-free formula for sting-free comfort
- Leaves a thin, breathable and transparent film on skin with no offensive odor
- Increases adhesion of tapes and wafers for added security and extended wear time
- Helps avoid adhesive buildup

| Item Description | MFR #    | ITEM #  | QTY                    | HCPCS |
|------------------|----------|---------|------------------------|-------|
| 2.4" x 2.4"      | 176-5728 | 1088820 | 1/ea, 25/bx, 100 bx/cs | A5120 |

## Wound Cleaners

### McKesson Dermal Wound Cleanser

- Non-sterile, preserved wound cleanser is for multiple use on chronic and acute wounds
- Non-cytotoxic
- Convenient rinse-free formulation



| Item Description                      | MFR # | ITEM # | QTY         | HCPCS |
|---------------------------------------|-------|--------|-------------|-------|
| 4 fl. oz. Squeeze Bottle, Non-Sterile | 1718  | 976352 | 1/ea, 12/cs | A6260 |
| 8 fl. oz. Spray Bottle, Non-Sterile   | 1719  | 949420 | 1/ea, 6/cs  | A6260 |
| 16 fl. oz. Spray Bottle, Non-Sterile  | 1720  | 949421 | 1/ea, 6/cs  | A6260 |

### McKesson Saline Wound Flush

- For moistening absorbent wound dressings and cleaning minor cuts, minor burns, superficial abrasions and minor irritations of the skin
- 0.9% USP sodium chloride solution
- Saline stream is 4 to 13 psi
- Drug and preservative free



| Item Description     | MFR #   | ITEM #  | QTY         | HCPCS |
|----------------------|---------|---------|-------------|-------|
| 3.0 fl. oz., Sterile | 37-6503 | 1103071 | 1/ea, 12/cs | A6260 |
| 7.1 fl. oz., Sterile | 37-6507 | 636352  | 1/ea, 12/cs | A6260 |



### McKesson Puracyn® Plus Professional Formula Wound Irrigation Solution

- Designed to improve the most essential part of the wound treatment process: preparation of the wound bed
- Provides clinicians with an ideal solution for proper wound bed preparation
- Contains hypochlorous acid, a known antimicrobial, which serves as a preservative to inhibit the growth of microorganisms within the product
- Non-cytotoxic and non-irritating
- Won't harm healthy tissue
- Contains no steroids or antibiotics
- Contains no alcohol or iodine

| Item Description        | MFR #    | ITEM #  | QTY        | HCPCS |
|-------------------------|----------|---------|------------|-------|
| 4 fl. oz. Pump          | 186-6505 | 1113210 | 1/ea, 6/cs | A6260 |
| 8.5 fl. oz. Pump        | 186-6509 | 1113211 | 1/ea, 6/cs | A6260 |
| 16.9 fl. oz. Septum Cap | 186-6516 | 1113212 | 1/ea, 6/cs | A6260 |
| 16.9 fl. oz. Trigger    | 186-6517 | 1113213 | 1/ea, 6/cs | A6260 |
| 500 mL Instill          | 186-6525 | 1113214 | 1/ea, 6/cs | A6260 |

## SKIN CLEANSING AND PREPARATION

### Wound Cleansers

#### McKesson Puracyn® Plus Wound and Skin Care Solution

- An over-the-counter formula intended for use at home, as a synergistic wound care management system
- Contains hypochlorous acid, a known antimicrobial, which serves as a preservative to inhibit the growth of microorganisms within the product
- Non-cytotoxic and non-irritating
- Won't harm healthy tissue
- Contains no steroids or antibiotics



| Item Description | MFR #    | ITEM #  | QTY        | HCPCS |
|------------------|----------|---------|------------|-------|
| 8 fl. oz.        | 186-6002 | 1113216 | 1/ea, 6/cs | A6260 |
| 16 fl. oz.       | 186-6004 | 1113217 | 1/ea, 6/cs | A6260 |

### Sundries



#### McKesson Cotton Balls

- 100% cotton
- Ideal for prepping, wound cleansing and perineal care
- Soft, offering maximum absorbency
- Retains shape wet or dry

| Item Description    | MFR #   | ITEM # | QTY              | HCPCS |
|---------------------|---------|--------|------------------|-------|
| Medium, Non-Sterile | 16-9153 | 980221 | 2000/bg, 2 bg/cs | A9270 |
| Large, Non-Sterile  | 16-9152 | 980220 | 1000/bg, 2 bg/cs | A9270 |



#### McKesson Lister Bandage Scissors

- Quality instrument for routine use

| Item Description | MFR #    | ITEM # | QTY  |
|------------------|----------|--------|------|
| 5.5", Extra Fine | 43-2-231 | 487480 | 1/ea |



#### McKesson Cotton-Tipped Applicators

- Wood shaft and absorbent cotton tip
- Ideal for application of prepping solutions
- Convenient ruler for measurement of site application

| Item Description | MFR #     | ITEM # | QTY                      |
|------------------|-----------|--------|--------------------------|
| 6", Sterile      | 24-106-1S | 999736 | 1pk, 100 pk/bx, 10 bx/cs |

# General Wound Care



For clinical information about our products, call 877.611.0081.  
To order, visit [mms.mckesson.com](https://mms.mckesson.com) or call your McKesson Medical-Surgical Account Manager.

## GENERAL WOUND CARE

# Gauze and Sponges



## McKesson All-Purpose Sponges

- Non-woven 100% cotton material
- Non-woven sponges are designed to resist fraying and residual lint
- Ideal for many applications such as cleaning, prepping, scrubbing and wound dressing
- More absorbent than traditional woven gauze
- Compare to Mirasorb®

| Item Description   | MFR #     | ITEM # | QTY                      | HCPCS |
|--------------------|-----------|--------|--------------------------|-------|
| <b>Non-Sterile</b> |           |        |                          |       |
| 2" x 2", 4-Ply     | 16-607613 | 481051 | 200/pk, 40 pk/cs         | A6216 |
| 4" x 4", 4-Ply     | 16-607623 | 481053 | 200/pk, 20 pk/cs         | A6216 |
| <b>Sterile</b>     |           |        |                          |       |
| 2" x 2", 4-Ply     | 16-602318 | 481052 | 2/pk, 25 pk/bx, 60 bx/cs | A6402 |
| 4" x 4", 4-Ply     | 16-602317 | 481054 | 2/pk, 25 pk/bx, 24 bx/cs | A6402 |
| 4" x 4", 4-Ply     | 16-602326 | 481055 | 10/tr, 96 tr/cs          | A6402 |



## McKesson Non-Woven Sponges, Poly-Rayon Blend

- Poly/rayon blend offers excellent absorption with virtually no linting
- High-quality alternative to cotton gauze with greater wicking properties than cotton gauze
- Appropriate for wound dressing, cleaning, prepping, and scrubbing

| Item Description                    | MFR #    | ITEM # | QTY                      | HCPCS |
|-------------------------------------|----------|--------|--------------------------|-------|
| <b>Non-Sterile</b>                  |          |        |                          |       |
| 2" x 2", 4-Ply                      | 92242000 | 446034 | 200/pk, 20 pk/cs         | A6216 |
| 3" x 3", 4-Ply                      | 93342000 | 476359 | 200/pk, 20 pk/cs         | A6216 |
| 4" x 4", 4-Ply                      | 94442000 | 446036 | 200/pk, 10 pk/cs         | A6216 |
| <b>High Absorbency, Non-Sterile</b> |          |        |                          |       |
| 2" x 2", 4-Ply                      | 16-40224 | 482412 | 200/pk, 20 pk/cs         | A6216 |
| 3" x 3", 4-Ply                      | 16-40334 | 482413 | 200/pk, 20 pk/cs         | A6216 |
| 4" x 4", 4-Ply                      | 16-40444 | 476709 | 200/pk, 10 pk/cs         | A6216 |
| <b>Sterile</b>                      |          |        |                          |       |
| 2" x 2", 4-Ply                      | 16-4224  | 446032 | 2/pk, 50 pk/bx, 30 bx/cs | A6402 |
| 2" x 2", 4-Ply                      | 42-24    | 939875 | 2/pk, 50 pk/bx, 30 bx/cs | A6402 |
| 3" x 3", 4-Ply                      | 16-4234  | 446054 | 2/pk, 40 pk/bx, 30 bx/cs | A6402 |
| 4" x 4", 4-Ply                      | 16-4244  | 446033 | 2/pk, 25 pk/bx, 24 bx/cs | A6402 |
| 4" x 4", 4-Ply                      | 42-44    | 946738 | 2/pk, 25 pk/bx, 24 bx/cs | A6402 |
| 4" x 4", 6-Ply                      | 16-4246  | 466523 | 2/pk, 25 pk/bx, 12 bx/cs | A6402 |
| <b>High Absorbency, Sterile</b>     |          |        |                          |       |
| 2" x 2", 4-Ply                      | 16-42224 | 482409 | 2/pk, 50 pk/bx, 30 bx/cs | A6402 |
| 4" x 4", 4-Ply                      | 16-42444 | 482411 | 2/pk, 25 pk/bx, 24 bx/cs | A6402 |

# Gauze and Sponges



## McKesson Split Sponges

- Non-woven poly/rayon blend offers excellent absorption with reduced linting
- Offers easy application around drains, chest tubes, catheters, tracheotomies, and IVs
- Fast wicking and superior absorbency

| Item Description                | MFR #    | ITEM # | QTY                      |
|---------------------------------|----------|--------|--------------------------|
| <b>Sterile</b>                  |          |        |                          |
| 2" x 2", 6-Ply                  | 16-4226  | 446055 | 2/pk, 35 pk/bx, 20 bx/cs |
| 4" x 4", 6-Ply                  | 16-42426 | 446056 | 2/pk, 25 pk/bx, 12 bx/cs |
| <b>High Absorbency, Sterile</b> |          |        |                          |
| 2" x 2", 6-Ply                  | 16-42026 | 482414 | 2/pk, 35 pk/bx, 20 bx/cs |
| 4" x 4", 6-Ply                  | 16-42046 | 482415 | 2/pk, 25 pk/bx, 12 bx/cs |



## McKesson Non-Adherent Pads

- Absorbent non-woven pad with non-stick formulation
- For use with lightly draining wounds
- Individually wrapped in easy-to-peel packaging

| Item Description | MFR #   | ITEM # | QTY                       |
|------------------|---------|--------|---------------------------|
| <b>Sterile</b>   |         |        |                           |
| 2" x 3"          | 16-4292 | 373768 | 1/pk, 100 pk/bx, 12 bx/cs |
| 3" x 4"          | 16-4293 | 373777 | 1/pk, 100 pk/bx, 12 bx/cs |
| 3" x 8"          | 16-4290 | 471997 | 1/pk, 75 pk/bx, 8 bx/cs   |



## McKesson Woven Gauze Sponges, Cotton

- 100% cotton
- Appropriate for wound dressing, cleaning, prepping, or packing
- Reduced linting

| Item Description            | MFR #    | ITEM # | QTY                       |
|-----------------------------|----------|--------|---------------------------|
| <b>Non-Sterile</b>          |          |        |                           |
| 2" x 2", 8-Ply              | 22082000 | 373772 | 200/pk, 25 pk/cs          |
| 2" x 2", 12-Ply             | 22122000 | 446029 | 200/pk, 40 pk/cs          |
| 3" x 3", 8-Ply              | 33082000 | 446030 | 200/pk, 20 pk/cs          |
| 3" x 3", 12-Ply             | 33122000 | 373776 | 200/pk, 20 pk/cs          |
| 4" x 4", 8-Ply              | 44082000 | 440028 | 200/pk, 20 pk/cs          |
| 4" x 4", 12-Ply             | 44122000 | 440343 | 200/pk, 10 pk/cs          |
| 4" x 4", 16-Ply             | 44162000 | 446031 | 200/pk, 10 pk/cs          |
| <b>Sterile</b>              |          |        |                           |
| 2" x 2", 12-Ply             | 16-4221  | 501657 | 1/pk, 100 pk/bx, 24 bx/cs |
| 4" x 4", 12-Ply             | 16-4241  | 446027 | 1/pk, 50 pk/bx, 24 bx/cs  |
| <b>Premium, Non-Sterile</b> |          |        |                           |
| 2" x 2", 8-Ply              | 22802000 | 446037 | 200/pk, 25 pk/cs          |
| 2" x 2", 12-Ply             | 12222000 | 446038 | 200/pk, 40 pk/cs          |
| 3" x 3", 12-Ply             | 12332000 | 446040 | 200/pk, 20 pk/cs          |
| 4" x 4", 8-Ply              | 44802000 | 446041 | 200/pk, 20 pk/cs          |
| 4" x 4", 12-Ply             | 12442000 | 446042 | 200/pk, 10 pk/cs          |
| 4" x 4", 16-Ply             | 16442000 | 446043 | 200/pk, 10 pk/cs          |

## GENERAL WOUND CARE

# Gauze and Sponges



## McKesson Premium Woven Gauze Sponges, Type VII

- 100% cotton
- Appropriate for wound dressing, cleaning, prepping, or packing
- Reduced linting

| Item Description   | MFR #    | ITEM #  | QTY                      |
|--------------------|----------|---------|--------------------------|
| <b>Non-Sterile</b> |          |         |                          |
| 2" x 2", 12-Ply    | 42222000 | 1043628 | 200/pk, 40 pk/cs         |
| 3" x 3", 12-Ply    | 42332000 | 1043629 | 200/pk, 20 pk/cs         |
| 4" x 4", 8-Ply     | 42448000 | 1043631 | 200/pk, 20 pk/cs         |
| 4" x 4", 12-Ply    | 42422000 | 1043630 | 200/pk, 10 pk/cs         |
| <b>Sterile</b>     |          |         |                          |
| 2" x 2", 8-Ply     | 16-42228 | 460859  | 2/pk, 50 pk/bx, 30 bx/cs |
| 3" x 3", 12-Ply    | 16-42332 | 447084  | 2/pk, 40 pk/bx, 30 bx/cs |
| 4" x 4", 8-Ply     | 16-42448 | 447085  | 2/pk, 50 pk/bx, 12 bx/cs |
| 4" x 4", 12-Ply    | 16-42442 | 446047  | 2/pk, 25 pk/bx, 24 bx/cs |
| 4" x 4", 12-Ply    | 16-42412 | 446049  | 1/pk, 50 pk/bx, 24 bx/cs |
| 4" x 4", 12-Ply    | 16-42441 | 446028  | 10/tr, 128 tr/cs         |
| 4" x 4", 16-Ply    | 16-4276  | 474706  | 10/tr, 72 tr/cs          |



## McKesson Woven Gauze Sponges, Poly-Rayon Blend

- Poly/rayon blend offers excellent absorption with reduced linting
- High-quality alternative to cotton gauze with greater wicking properties than cotton gauze
- Appropriate for wound dressing, cleaning, prepping, and scrubbing

| Item Description | MFR #   | ITEM # | QTY                      |
|------------------|---------|--------|--------------------------|
| <b>Sterile</b>   |         |        |                          |
| 2" x 2", 8-Ply   | 16-4228 | 635942 | 2/pk, 50 pk/bx, 30 bx/cs |
| 3" x 3", 12-Ply  | 16-4232 | 384816 | 2/pk, 40 pk/bx, 30 bx/cs |
| 4" x 4", 8-Ply   | 16-4248 | 762704 | 2/pk, 50 pk/bx, 12 bx/cs |
| 4" x 4", 12-Ply  | 16-4242 | 762703 | 2/pk, 25 pk/bx, 24 bx/cs |



## McKesson Super Sponges

- 100% cotton
- Bulky for added cushioning and wound protection
- Fast-wicking and highly absorbent
- Use as a primary or secondary dressing
- Diagonal fold
- Conformable

| Item Description          | MFR #    | ITEM # | QTY                      |
|---------------------------|----------|--------|--------------------------|
| <b>Sterile, Stretched</b> |          |        |                          |
| 6" x 6-3/4"               | 16-42626 | 446053 | 2/pk, 20 pk/bx, 12 bx/cs |
| 6" x 6-3/4"               | 16-4265  | 473032 | 5/pk, 10 pk/bx, 9 bx/cs  |

# Gauze and Sponges



## McKesson Oval Eye Pad

- Constructed of a soft blend of polyethylene and rayon for maximum absorption
- Oval shape design helps protect the entire eye

| Item Description      | MFR #   | ITEM # | QTY                      |
|-----------------------|---------|--------|--------------------------|
| <b>Sterile</b>        |         |        |                          |
| Small 1-5/8" x 2-5/8" | 16-4281 | 467977 | 1/pk, 50 pk/bx, 12 bx/cs |
| Large 2-1/8" x 2-5/8" | 16-4282 | 467978 | 1/pk, 50 pk/bx, 12 bx/cs |



## McKesson Combine ABD Pads

- Non-woven outer fabric with inner cellulose tissue construction that helps provide maximum moisture barrier
- Sealed edges to help prevent leaking
- Absorption with virtually no linting
- Faster wicking for drier wound sites

| Item Description | MFR #   | ITEM # | QTY                      | HCPCS |
|------------------|---------|--------|--------------------------|-------|
| <b>Sterile</b>   |         |        |                          |       |
| 5" x 9"          | 16-4250 | 446057 | 1/pk, 20 pk/bx, 20 bx/cs | A6252 |
| 7.5" x 8"        | 16-4252 | 446058 | 1/pk, 12 pk/bx, 15 bx/cs | A6253 |
| 8" x 10"         | 16-4254 | 446059 | 1/pk, 24 pk/bx, 20 bx/cs | A6253 |



## McKesson Trauma Dressing

- For first aid applications: lacerations, fractures, abrasions/ burns, penetration wounds
- Absorbent

| Item Description | MFR #   | ITEM # | QTY            |
|------------------|---------|--------|----------------|
| <b>Sterile</b>   |         |        |                |
| 10" x 30"        | 16-4255 | 474702 | 1/pk, 50 pk/cs |



## McKesson Cover Dressing Sponges

- Non-woven facing
- Absorbent and fast wicking
- Compare to Topper®

| Item Description | MFR #   | ITEM # | QTY                      |
|------------------|---------|--------|--------------------------|
| <b>Sterile</b>   |         |        |                          |
| 4" x 4"          | 16-2436 | 482272 | 2/pk, 25 pk/bx, 24 bx/cs |

## GENERAL WOUND CARE

# Gauze and Sponges



## McKesson Conforming Stretch Gauze Bandages

- Polyester blend provides elasticity while retaining its shape and reducing slippage
- Breathable and absorbent for added comfort
- Secures and stretches to conform to the body's contours
- Finished edges to help prevent unraveling
- Needs minimal taping for ease of use
- Low-linting formulation helps minimize the risk of fibers being left behind in the wound bed

| Item Description             | MFR #  | ITEM # | QTY                     |
|------------------------------|--------|--------|-------------------------|
| <b>Stretched Non-Sterile</b> |        |        |                         |
| 1" x 1.7 yds.                | 16-010 | 993036 | 24 rl/bg, 4 bg/cs       |
| 2" x 4.1 yds.                | 16-011 | 993032 | 12 rl/bg, 8 bg/cs       |
| 3" x 4.1 yds.                | 16-012 | 993033 | 12 rl/bg, 8 bg/cs       |
| 4" x 4.1 yds.                | 16-013 | 993034 | 12 rl/bg, 8 bg/cs       |
| 6" x 4.1 yds.                | 16-014 | 993035 | 6 rl/bg, 8 bg/cs        |
| <b>Stretched, Sterile</b>    |        |        |                         |
| 1" x 1.7 yds.                | 16-016 | 993031 | 1/pk, 24 pk/bg, 4 bg/cs |
| 2" x 4.1 yds.                | 16-017 | 999367 | 1/pk, 12 pk/bg, 8 bg/cs |
| 3" x 4.1 yds.                | 16-018 | 999364 | 1/pk, 12 pk/bg, 8 bg/cs |
| 4" x 4.1 yds.                | 16-019 | 999366 | 1/pk, 12 pk/bg, 8 bg/cs |
| 6" x 4.1 yds.                | 16-020 | 999365 | 1/pk, 6 pk/bg, 8 bg/cs  |

# Adhesive Bandages



## McKesson Adhesive Bandages, Fabric

- Flexible, durable fabric that stretches for added comfort
- Helps protect cuts and scrapes by conforming to skin with an absorbent, non-stick pad
- Features comfortable fit, long-lasting adhesive

| Item Description          | MFR #   | ITEM # | QTY              | HCPCS |
|---------------------------|---------|--------|------------------|-------|
| <b>Sterile</b>            |         |        |                  |       |
| Digit, Small, Tan         | 16-4818 | 466875 | 100/bx, 24 bx/cs | A6413 |
| Knuckle, 1-1/2" x 3", Tan | 16-4814 | 466873 | 100/bx, 24 bx/cs | A6413 |
| Patch, 2" x 3", Tan       | 16-4816 | 466874 | 50/bx, 24 bx/cs  | A6413 |
| Patch, 2" x 4", Tan       | 16-4817 | 514534 | 50/bx, 24 bx/cs  | A6413 |
| Spot, 1", Tan             | 16-4812 | 466870 | 100/bx, 24 bx/cs | A6413 |
| Strip, 3/4" x 3", Tan     | 16-4813 | 466871 | 100/bx, 24 bx/cs | A6413 |
| Strip, 1" x 3", Tan       | 16-4811 | 466872 | 100/bx, 24 bx/cs | A6413 |

# Adhesive Bandages



## McKesson Pink Adhesive Bandages, Plastic

- Flexible, durable plastic that stretches for added comfort
- Features comfortable fit, long-lasting adhesive

| Item Description       | MFR #   | ITEM #  | QTY              | HCPCS |
|------------------------|---------|---------|------------------|-------|
| <b>Sterile</b>         |         |         |                  |       |
| Strip, 3/4" x 3", Pink | 16-4835 | 1055594 | 100/bx, 24 bx/cs | A6413 |



## McKesson Adhesive Bandages, Plastic

- The breathable material helps facilitate the healing process
- Helps protect cuts and scrapes by conforming to skin with an absorbent, non-stick pad
- Absorption with virtually no linting
- Faster wicking for drier wound sites

| Item Description      | MFR #   | ITEM # | QTY              | HCPCS |
|-----------------------|---------|--------|------------------|-------|
| <b>Sterile</b>        |         |        |                  |       |
| Spot, 1" Round, Tan   | 16-4822 | 466877 | 100/bx, 24 bx/cs | A6413 |
| Strip, 3/4" x 3", Tan | 16-4823 | 464083 | 100/bx, 24 bx/cs | A6413 |
| Strip, 1" x 3", Tan   | 16-4821 | 466878 | 100/bx, 24 bx/cs | A6413 |
| Patch, 2" x 4", Tan   | 16-4825 | 511333 | 50/bx, 24 bx/cs  | A6413 |



## McKesson Kids™ Adhesive Bandages, Plastic

- Assorted designs
- Breathable plastic
- Helps protect cuts and scrapes by conforming to skin with an absorbent, non-stick pad

| Item Description                   | MFR #      | ITEM #  | QTY              | HCPCS |
|------------------------------------|------------|---------|------------------|-------|
| <b>Sterile</b>                     |            |         |                  |       |
| Spot, 1", Assorted Print           | 16-4832-17 | 1110981 | 100/bx, 24 bx/cs | A6413 |
| Strip, 3/4" x 3", Assorted Print   | 16-4838-17 | 1110980 | 100/bx, 24 bx/cs | A6413 |
| Strip, 3/4" x 3", Multi-Color Neon | 16-4837    | 1055592 | 100/bx, 24 bx/cs | A6413 |
| Strip, 3/4" x 3", Blue/Pink Camo   | 16-4836    | 1055593 | 100/bx, 24 bx/cs | A6413 |

## GENERAL WOUND CARE

# Adhesive Bandages

## McKesson Oval Adhesive Bandages

- Designed with flexible, durable fabric and long-lasting adhesive to provide comfort through extended use time
- Helps protect cuts and scrapes by conforming to skin with an absorbent, non-stick pad
- Capable of absorbing 10 times its own weight in water



| Item Description | MFR #   | ITEM #  | QTY              |
|------------------|---------|---------|------------------|
| <b>Sterile</b>   |         |         |                  |
| 1" x 1.25"       | 16-4800 | 1103072 | 100/bx, 24 bx/cs |



## McKesson Adhesive Dressings

- Highly absorbent
- Non-adherent dressing prevents wound pad from sticking to wound and disrupting healing tissue
- Acrylic adhesive strips on both sides

| Item Description | MFR #   | ITEM #  | QTY              |
|------------------|---------|---------|------------------|
| <b>Sterile</b>   |         |         |                  |
| 2" x 3"          | 16-4272 | 1088663 | 100/bx, 24 bx/cs |
| 3" x 4"          | 16-4273 | 1086175 | 100/bx, 12 bx/cs |



## McKesson Silicone Adhesive Bandages

- Help protect cuts and scrapes by conforming to skin with an absorbent, non-stick pad
- Waterproof, transparent strips are easy to re-apply

| Item Description      | MFR #   | ITEM #  | QTY             |
|-----------------------|---------|---------|-----------------|
| <b>Sterile</b>        |         |         |                 |
| Strip, 1" x 3", Sheer | 16-4840 | 1083097 | 10/bx, 32 bx/cs |

# Cohesive Bandages



## McKesson Latex Cohesive Bandages

- CAUTION: This product contains natural rubber latex which may cause allergic reactions
- Self-adhering; no need for fasteners or clips
- Ideal for holding dressings in place without use of adhesives
- Provides light compression and soft tissue support for minor strains and sprains
- Easily conforms to the body's natural contours and movements

| Item Description               | MFR #   | ITEM # | QTY               | HCPCS |
|--------------------------------|---------|--------|-------------------|-------|
| <b>Latex, Tan, Non-Sterile</b> |         |        |                   |       |
| 1" x 5 yds.                    | 16-3101 | 464151 | 1 rl/pk, 30 pk/cs | A6453 |
| 2" x 5 yds.                    | 16-3202 | 464152 | 1 rl/pk, 36 pk/cs | A6453 |
| 3" x 5 yds.                    | 16-3303 | 464153 | 1 rl/pk, 24 pk/cs | A6454 |
| 4" x 5 yds.                    | 16-3404 | 464154 | 1 rl/pk, 18 pk/cs | A6454 |
| 6" x 5 yds.                    | 16-3606 | 464155 | 1 rl/pk, 12 pk/cs | A6455 |

|  |         |        |                   |       |
|--|---------|--------|-------------------|-------|
| <b>Latex, Multi-Color, Non-Sterile</b> |         |        |                   |       |
| 1" x 5 yds.                            | 16-4201 | 944355 | 1 rl/pk, 30 pk/cs | A6453 |
| 1.5" x 5 yds.                          | 16-4304 | 987335 | 1 rl/pk, 48 pk/cs | A6453 |
| 2" x 5 yds.                            | 16-4202 | 464156 | 1 rl/pk, 36 pk/cs | A6454 |
| 3" x 5 yds.                            | 16-4303 | 464157 | 1 rl/pk, 24 pk/cs | A6454 |
| 4" x 5 yds.                            | 16-4476 | 992533 | 1 rl/pk, 18 pk/cs | A6454 |

|                            |          |        |                   |       |
|----------------------------|----------|--------|-------------------|-------|
| <b>Latex, Tan, Sterile</b> |          |        |                   |       |
| 3" x 5 yds.                | 16-53333 | 520558 | 1 rl/pk, 24 pk/cs | A6454 |
| 4" x 5 yds.                | 16-53434 | 520556 | 1 rl/pk, 18 pk/cs | A6454 |
| 6" x 5 yds.                | 16-53636 | 520557 | 1 rl/pk, 12 pk/cs | A6455 |



## McKesson Stretched Cohesive Bandages

- Not made with natural rubber latex
- Self-adhering; no need for fasteners or clips
- Ideal for holding dressings in place without use of adhesives
- Provides light compression and soft tissue support for minor strains and sprains
- Easily conforms to the body's natural contours and movements

| Item Description                   | MFR #    | ITEM # | QTY               | HCPCS |
|------------------------------------|----------|--------|-------------------|-------|
| <b>Stretched, Tan, Non-Sterile</b> |          |        |                   |       |
| 1" x 5 yds.                        | 16-53111 | 520550 | 1 rl/pk, 30 pk/cs | A6453 |
| 2" x 5 yds.                        | 16-53212 | 520551 | 1 rl/pk, 36 pk/cs | A6453 |
| 3" x 5 yds.                        | 16-53313 | 520552 | 1 rl/pk, 24 pk/cs | A6454 |
| 4" x 5 yds.                        | 16-53414 | 520549 | 1 rl/pk, 18 pk/cs | A6454 |
| 6" x 5 yds.                        | 16-53616 | 520548 | 1 rl/pk, 12 pk/cs | A6455 |

|  |          |         |                   |       |
|--|----------|---------|-------------------|-------|
| <b>Stretched, Multi-Color, Non-Sterile</b> |          |         |                   |       |
| 1" x 5 yds.                                | 16-53121 | 1032956 | 1 rl/pk, 30 pk/cs | A6453 |
| 2" x 5 yds.                                | 16-53222 | 1032955 | 1 rl/pk, 36 pk/cs | A6453 |

|                                |          |        |                   |       |
|--------------------------------|----------|--------|-------------------|-------|
| <b>Stretched, Tan, Sterile</b> |          |        |                   |       |
| 3" x 5 yds.                    | 16-53343 | 520555 | 1 rl/pk, 24 pk/cs | A6454 |
| 4" x 5 yds.                    | 16-53444 | 520553 | 1 rl/pk, 18 pk/cs | A6454 |
| 6" x 5 yds.                    | 16-53646 | 520554 | 1 rl/pk, 12 pk/cs | A6455 |

## GENERAL WOUND CARE

# Elastic Bandages

## McKesson Elastic Bandages

- Not made with natural rubber latex
- Designed to provide comfortable, controlled and consistent compression for strains and sprains
- Reusable design retains elasticity, even after repeated use and washing
- Intended to provide comfort and support for an extended period of time
- Hook-and-loop closure allows bandage to be easily wrapped and secured without clips
- Front closure anchors the bandage on the first wrap quickly and easily
- End closure conveniently secures the bandage at any given point without pins, clips or tape
- Intended to offer firm support for stress injuries and to reduce strain on muscles, tendons and joints
- Can be used for securing splints, dressings and hot and cold packs

| Item Description            | MFR #     | ITEM #  | QTY            | HCPCS |
|-----------------------------|-----------|---------|----------------|-------|
| <b>Non-Sterile</b>          |           |         |                |       |
| 2" x 5 yds., Stretched, Tan | 16-2033-2 | 1086527 | 10/bx, 5 bx/cs | A6448 |
| 3" x 5 yds., Stretched, Tan | 16-2033-3 | 1086528 | 10/bx, 5 bx/cs | A6449 |
| 4" x 5 yds., Stretched, Tan | 16-2033-4 | 1086529 | 10/bx, 5 bx/cs | A6449 |
| 6" x 5 yds., Stretched, Tan | 16-2033-6 | 1086530 | 10/bx, 5 bx/cs | A6450 |



## McKesson Knit-Woven Blend Elastic Bandages

- Not made with natural rubber latex
- Designed to provide comfortable, controlled and consistent compression for strains and sprains
- Intended to offer firm support to injured parts of the body and to help reduce strain on muscles, tendons and joints
- Can also be used for securing splints, dressings and ice packs
- Feature a hook-and-loop closure intended to replace the need for traditional metal clips

| Item Description            | MFR # | ITEM # | QTY            | HCPCS |
|-----------------------------|-------|--------|----------------|-------|
| <b>Non-Sterile</b>          |       |        |                |       |
| 2" x 5 yds., Stretched, Tan | 052   | 911846 | 10/bx, 5 bx/cs | A6448 |
| 3" x 5 yds., Stretched, Tan | 053   | 911814 | 10/bx, 5 bx/cs | A6449 |
| 4" x 5 yds., Stretched, Tan | 054   | 911815 | 10/bx, 5 bx/cs | A6449 |
| 6" x 5 yds., Stretched, Tan | 056   | 911816 | 10/bx, 5 bx/cs | A6450 |



# Elastic Bandages

## McKesson Knit Elastic Bandages

- Not made with natural rubber latex
- Hand washable
- Hook and loop closure
- Single patient use

| Item Description            | MFR #         | ITEM # | QTY                  | HCPCS |
|-----------------------------|---------------|--------|----------------------|-------|
| <b>Non-Sterile</b>          |               |        |                      |       |
| 2" x 5 yds., Stretched, Tan | 16-1033-2     | 454618 | 1/ea, 10/bx, 5 bx/cs | A6448 |
| 3" x 5 yds., Stretched, Tan | 16-1033-3     | 454619 | 1/ea, 10/bx, 5 bx/cs | A6449 |
| 4" x 5 yds., Stretched, Tan | 16-1033-4     | 454620 | 1/ea, 10/bx, 5 bx/cs | A6449 |
| 6" x 5 yds., Stretched, Tan | 16-1033-6     | 454621 | 1/ea, 10/bx, 5 bx/cs | A6450 |
| <b>Sterile</b>              |               |        |                      |       |
| 3" x 5 yds., Stretched, Tan | 16-1033-3-STR | 471792 | 36/cs                | A6449 |
| 4" x 5 yds., Stretched, Tan | 16-1033-4-STR | 471793 | 36/cs                | A6449 |



# Fluff Bandage Rolls

## McKesson Conforming Stretch Gauze Bandages

- Polyester design provides elasticity while retaining its shape and reducing slippage
- Breathable and absorbent for added comfort
- Secures and stretches to conform to the body's contours.
- Finished edges to help prevent unraveling
- Needs minimal taping for ease of use
- Low-linting formulation helps minimize the risk of fibers being left behind in the wound bed



| Item Description   | MFR #  | ITEM # | QTY              |
|--------------------|--------|--------|------------------|
| <b>Non-Sterile</b> |        |        |                  |
| 6" Stretched       | 16-014 | 993035 | 6 rl/bg, 8 bg/cs |

## McKesson Gauze Bandage Roll

- Made of 100% prewashed, fluff-dried woven cotton to provide maximum absorbency
- Crinkle-weave pattern increases bulk to cushion
- 6-ply construction with finished edges reduce risk of wound contamination and granulomas
- Open-weave design provides fast wicking action and superior aeration to help reduce risk of maceration
- Bulk cushioning and conformability improve wound site protection



| Item Description   | MFR #   | ITEM #  | QTY               |
|--------------------|---------|---------|-------------------|
| <b>Non-Sterile</b> |         |         |                   |
| 2.25" x 3 yds.     | 16-4262 | 1055582 | 12 rl/bg, 8 bg/cs |
| <b>Sterile</b>     |         |         |                   |
| 2.25" x 3 yds.     | 16-4062 | 1056324 | 1 rl/pk, 96 pk/cs |

# Fluff Bandage Rolls

## McKesson Stretched Gauze Bandage Rolls

- 100% cotton
- Crinkled cotton pattern adds loft and bulk for added cushioning and wound protection
- Fast-wicking and absorbent
- Conformable
- Use as a primary or secondary dressing
- Comparable to Kerlix™ gauze bandage rolls



| Item Description           | MFR #    | ITEM # | QTY          | HCPCS |
|----------------------------|----------|--------|--------------|-------|
| <b>Non-Sterile</b>         |          |        |              |       |
| 3.4" x 3.6 yds., Stretched | 16-4063  | 649420 | 1/rl, 96/cs  | A6446 |
| 4.5" x 4.1 yds., Stretched | 30642000 | 446050 | 1/rl, 100/cs | A6446 |
| <b>Sterile</b>             |          |        |              |       |
| 3.4" x 3.6 yds., Stretched | 16-4263  | 470749 | 1/rl, 96/cs  | A6446 |
| 4.5" x 4.1 yds., Stretched | 16-4264  | 446051 | 1/rl, 100/cs | A6446 |

## Packing Strips



### McKesson Plain Packing Strips

- Designed for nasal packing, dental packing or as packing for open or infected wounds
- Easy dispensing packaging (patent pending) helps facilitate caregiver convenience
- Made of 100% cotton, ravel-resistant gauze designed to resist linting or fraying

| Item Description         | MFR #    | ITEM # | QTY         | HCPCS |
|--------------------------|----------|--------|-------------|-------|
| <b>Sterile</b>           |          |        |             |       |
| 1/4" x 5 yds. Per Bottle | 61-59120 | 470436 | 1/bt, 12/cs | A6407 |
| 1/2" x 5 yds. Per Bottle | 61-59220 | 632342 | 1/bt, 12/cs | A6407 |
| 1" x 5 yds. Per Bottle   | 61-59320 | 470438 | 1/bt, 12/cs | A6407 |



### McKesson Iodoform Packing Strips 5% w/w

- Designed for nasal packing, dental packing or as packing for open or infected wounds
- Provides an additional antiseptic property for minor cuts and wounds
- Easy dispensing packaging (patent pending) helps facilitate caregiver convenience
- Made of 100% cotton, ravel-resistant gauze designed to resist linting or fraying

| Item Description         | MFR #    | ITEM # | QTY         | HCPCS |
|--------------------------|----------|--------|-------------|-------|
| <b>Sterile</b>           |          |        |             |       |
| 1/4" x 5 yds. Per Bottle | 61-59145 | 470440 | 1/bt, 12/cs | A6266 |
| 1/2" x 5 yds. Per Bottle | 61-59245 | 470441 | 1/bt, 12/cs | A6266 |
| 1" x 5 yds. Per Bottle   | 61-59345 | 470442 | 1/bt, 12/cs | A6266 |

# Retainer Dressings




## McKesson Tubular Elastic Retainer Net

- Stretched length 25 yards
- Designed to serve as a secondary dressing
- Applies gentle pressure to keep bandages and medical devices securely in place without adhesives
- Easy to customize and conforms to body contours
- Allows flexibility and maximum air flow
- Available in a wide range of sizes to suit a variety of patient needs
- Single use

| Item Description   | MFR #      | ITEM #  | QTY            | HCPCS |
|--------------------|------------|---------|----------------|-------|
| <b>Non-Sterile</b> |            |         |                |       |
| Size 1             | MSVP114701 | 1113345 | 1/bx, 10 bx/cs | A6457 |
| Size 2             | MSVP114702 | 1113346 | 1/bx, 10 bx/cs | A6457 |
| Size 3             | MSVP114703 | 1113347 | 1/bx, 10 bx/cs | A6457 |
| Size 4             | MSVP114704 | 1113348 | 1/bx, 10 bx/cs | A6457 |
| Size 5             | MSVP114705 | 1113349 | 1/bx, 10 bx/cs | A6457 |
| Size 6             | MSVP114706 | 1113350 | 1/bx, 10 bx/cs | A6457 |
| Size 7             | MSVP114707 | 1113351 | 1/bx, 10 bx/cs | A6457 |
| Size 8             | MSVP114708 | 1113352 | 1/bx, 10 bx/cs | A6457 |
| Size 9             | MSVP114709 | 1113353 | 1/bx, 10 bx/cs | A6457 |
| Size 10            | MSVP114710 | 1113354 | 1/bx, 10 bx/cs | A6457 |
| Size 11            | MSVP114711 | 1113355 | 1/bx, 10 bx/cs | A6457 |

## Sizing Guide

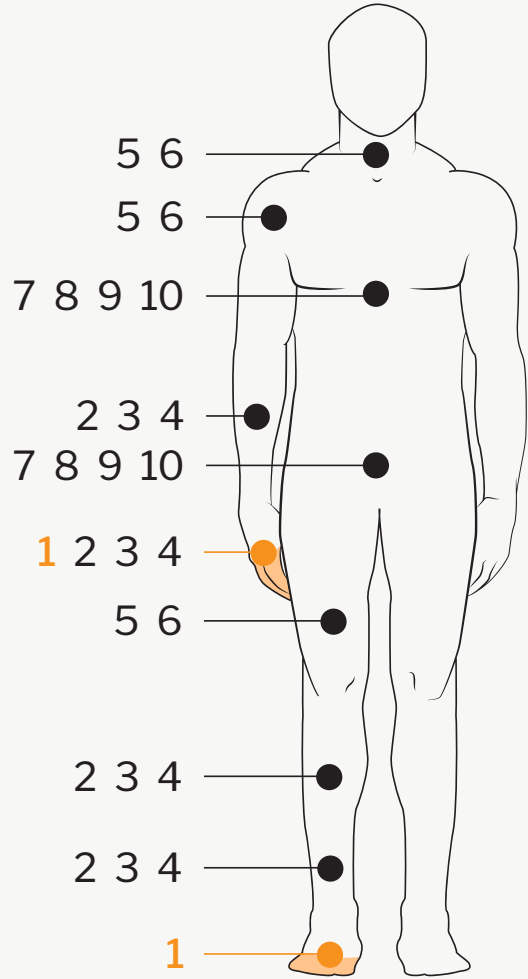

| SIZE     | SUGGESTED APPLICATION  |
|----------|--|
| <b>1</b> | <b>SMALL FINGER, TOES</b>  |
| 2        | SMALL HAND   X-SMALL ARM, LEG, FOOT<br>MEDIUM FINGER, TOES                 |
| 3        | MEDIUM HAND   SMALL ARM, LEG, FOOT   |
| 4        | LARGE HAND   MEDIUM ARM, LEG, FOOT   |
| 5        | X-SMALL HEAD   SMALL SHOULDER, THIGH<br>LARGE ARM, LEG, FOOT               |
| 6        | SMALL HEAD   MEDIUM SHOULDER, THIGH  |
| 7        | MEDIUM HEAD   LARGE SHOULDER, THIGH<br>SMALL CHEST, BACK, AXILLA, PERINEUM |
| 8        | LARGE HEAD   X-LARGE THIGH<br>MEDIUM CHEST, BACK, AXILLA, PERINEUM         |
| 9        | LARGE CHEST, BACK, AXILLA, PERINEUM  |
| 10       | X-LARGE CHEST, BACK, AXILLA, PERINEUM                                      |
| 11       | XX-LARGE CHEST, BACK, AXILLA, PERINEUM                                     |

# Tapes



## McKesson Paper Surgical Tape

- Gently removes from skin
- Air and moisture permeable
- Hypoallergenic

| Item Description          | MFR #     | ITEM #  | QTY                   | HCPCS |
|---------------------------|-----------|---------|-----------------------|-------|
| <b>White, Non-Sterile</b> |           |         |                       |       |
| 1/2" x 10 yds.            | 16-47305  | 455530  | 1/rl, 24/bx, 12 bx/cs | A4452 |
| 1" x 10 yds.              | 16-47310  | 455531  | 1/rl, 12/bx, 12 bx/cs | A4452 |
| 1" x 1.5 yds.             | 16-47310S | 1055583 | 1/rl, 100/bx, 8 bx/cs | A4452 |
| 2" x 10 yds.              | 16-47320  | 455532  | 1/rl, 6/bx, 12 bx/cs  | A4452 |
| 3" x 10 yds.              | 16-47330  | 466889  | 4 rl/bx, 12 bx/cs     | A4452 |
| <b>Tan, Sterile</b>       |           |         |                       |       |
| 1/2" x 10 yds.            | 16-47305T | 1055585 | 1/rl, 24/bx, 12 bx/cs | A4452 |
| 1" x 10 yds.              | 16-47310T | 1055584 | 1/rl, 12/bx, 12 bx/cs | A4452 |

## McKesson Cloth Surgical Tape

- Gently removes from skin
- Air and moisture permeable
- Hypoallergenic

| Item Description          | MFR #    | ITEM # | QTY                   | HCPCS |
|---------------------------|----------|--------|-----------------------|-------|
| <b>White, Non-Sterile</b> |          |        |                       |       |
| 1" x 10 yds.              | 16-47110 | 455534 | 1/rl, 12/bx, 12 bx/cs | A4452 |
| 2" x 10 yds.              | 16-47120 | 455535 | 1/rl, 6/bx, 12 bx/cs  | A4452 |
| 3" x 10 yds.              | 16-47130 | 466891 | 1/rl, 4/bx, 12 bx/cs  | A4452 |



## McKesson Silicone Adhesive Surgical Tape

- Removes cleanly without disrupting fragile skin layers or causing patients undue pain
- Silicone adhesive offers reliable and pliable fixation
- Gentle, repositionable tape maintains effectiveness when repeated taping is necessary
- Can be neatly torn by hand
- Ideal for patients with sensitive, compromised or at-risk skin
- Ideal for patients concerned with maintaining skin integrity and minimizing pain

| Item Description                | MFR #    | ITEM #  | QTY                   | HCPCS |
|---------------------------------|----------|---------|-----------------------|-------|
| <b>Transparent, Non-Sterile</b> |          |         |                       |       |
| 1" x 5.5 yds.                   | 16-48410 | 1073806 | 1/rl, 12/bx, 10 bx/cs | A4452 |
| 2" x 5.5 yds.                   | 16-48420 | 1073807 | 1/rl, 6/bx, 10 bx/cs  | A4452 |



## GENERAL WOUND CARE

# Tapes



## McKesson Transparent Surgical Tape

- Gently removes from skin
- Air and moisture permeable
- Hypoallergenic

| Item Description   | MFR #    | ITEM # | QTY                   | HCPCS |
|--------------------|----------|--------|-----------------------|-------|
| <b>Non-Sterile</b> |          |        |                       |       |
| 1" x 10 yds.       | 16-47210 | 455537 | 1/rl, 12/bx, 12 bx/cs | A4452 |
| 2" x 10 yds.       | 16-47220 | 455538 | 1/rl, 6/bx, 12 bx/cs  | A4452 |



## McKesson Waterproof Surgical Tape

- Hypoallergenic
- Waterproof backcloth
- Strong adhesive with minimal residue

| Item Description   | MFR #    | ITEM # | QTY                  |
|--------------------|----------|--------|----------------------|
| <b>Non-Sterile</b> |          |        |                      |
| 2" x 10 yds.       | 16-47520 | 520284 | 1/rl, 6/bx, 12 bx/cs |

## McKesson Dressing Retention Tape

- Retention tapes are used to secure dressings for a conformable fit that is built for patient comfort
- Breathable, wear resistant fabric
- Conformable to body contours
- Non-irritating adhesive
- Grid guidelines help facilitate easy measurement
- Released backing

| Item Description          | MFR #   | ITEM #  | QTY            | HCPCS |
|---------------------------|---------|---------|----------------|-------|
| <b>White, Non-Sterile</b> |         |         |                |       |
| 2" x 10 yds.              | 16-4802 | 1087971 | 1/bx, 24 bx/cs | A4452 |
| 4" x 10 yds.              | 16-4804 | 1087972 | 1/bx, 24 bx/cs | A4452 |
| 6" x 10 yds.              | 16-4806 | 1087973 | 1/bx, 12 bx/cs | A4452 |



# Tapes

## McKesson Cloth Surgical Tape, Perforated

- Breathable, spun-lace, non-woven, soft cloth surgical tape
- 2" perforations for easy tearing
- Bidirectional tear for convenient sizing
- Gently removes from skin leaving virtually no residue
- Medical-grade acrylic adhesive
- Flexible design helps maintain shape to enhance security and accommodate swelling

| Item Description          | MFR #     | ITEM #  | QTY               | HCPCS |
|---------------------------|-----------|---------|-------------------|-------|
| <b>White, Non-Sterile</b> |           |         |                   |       |
| 1" x 10 yds.              | 172-49210 | 1084091 | 2 rl/pk, 12 pk/cs | A4452 |
| 2" x 10 yds.              | 172-49220 | 1084092 | 1 rl/pk, 12 pk/cs | A4452 |
| 3" x 10 yds.              | 172-49230 | 1084093 | 1 rl/pk, 12 pk/cs | A4452 |
| 4" x 10 yds.              | 172-49240 | 1084094 | 1 rl/pk, 12 pk/cs | A4452 |
| 6" x 10 yds.              | 172-49260 | 1084095 | 1 rl/pk, 6 pk/cs  | A4452 |



## Moisture Barriers

### McKesson Moisture Barrier Film Dressings


- Shower protection dressing measuring 7" x 7" intended to cover PICC lines, IV catheters, Hickman catheters, dialysis lines, and stomas
- Water-resistant, PE-film forms barrier reduces the risks of infection from water-borne bacteria
- Transparent dressing allows uninhibited view of wound site
- Helps keep sites, wounds, and dressings clean and dry when patient showering
- Generally protects surgical incisions, intravenous (IV) sites, catheters, PICC lines, and other primary dressings from water and moisture while showering
- Strong adhesive made of synthetic rubber, tack, and acrylic/acrylate secures protection during use and gently releases from the skin during removal

| Item Description   | MFR #   | ITEM # | QTY             |
|--------------------|---------|--------|-----------------|
| <b>Non-Sterile</b> |         |        |                 |
| 7" x 7"            | 1162466 | 11077  | 5/pk, 80 pk/cs  |
| 9" x 9"            | 1162467 | 11099  | 5/pk, 140 pk/cs |
| 10" x 12"          | 1162488 | 11012  | 5/pk, 50 pk/cs  |



# Clinical Guide to Assessing Wounds

## Healthy Tissue

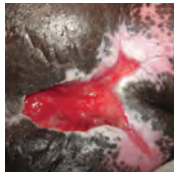

### Granulation Tissue:<sup>1</sup>

The pink/red moist tissue composed of new blood vessels, connective tissue, fibroblasts, and inflammatory cells, which fills an open wound when it starts to heal; typically appears deep pink or red with an irregular, granular surface.



### Epithelialization:<sup>1</sup>

The process of becoming covered with or converted to epithelium. New epithelial cells advance across the wound until they eventually meet epithelial cells moving in from the opposite direction.

NPUAP copyright and used with permission.

## Necrotic Tissue/Debridement

Debridement is the removal of necrotic or devitalized tissue from the wound bed. The presence of necrotic tissue increases the risk for bacterial growth and infection, and its presence generally impairs the natural healing process. The most common methods of debridement include autolytic, enzymatic, mechanical and surgical. Necrotic tissue in a wound is typically referred to as either slough or eschar.



### Slough:<sup>1</sup>

Soft moist avascular (devitalized) tissue; may be white, yellow, tan, or green; may be loose or firmly adherent.



### Eschar:<sup>1</sup>

Black or brown necrotic, devitalized tissue; tissue may be loose or firmly adherent, hard, soft or soggy.

### Autolytic debridement:<sup>1</sup>

Removal of devitalized tissue accomplished by the use of moisture retentive dressings. Maintaining a moist wound environment and moisture balance will facilitate the body's own enzymes and white blood cells in debridement of necrotic tissue.

### When not to debride:<sup>2</sup>

Typically when pressure injuries are located on the heel, ear or other site with little subcutaneous or muscle tissue and are covered with a dry, stable eschar, they should not be debrided unless an infection is suspected.

Debridement is also generally not recommended for arterial ulcers and diabetic ulcers with dry eschar or dry gangrene, without infection and with an insufficient vascular supply for healing, unless circulation to the area can be improved.

## Wound Infection

A wound is considered infected when replicating organisms invade the tissues and a host immune response is detected<sup>3</sup>. The only way to definitively determine infection is with a wound culture or tissue biopsy. Two commonly used techniques for obtaining a wound culture (aerobic and anaerobic) by swab are the Z-stroke and Levine techniques. Always first cleanse the wound with normal saline and culture viable tissue; do not culture necrotic tissue, purulent drainage or hard, dry eschar.

### Clinical Signs and Symptoms of Infection<sup>3</sup>

#### Pain:

Increased pain in the peri-wound area.

#### Erythema:

The presence of bright or dark red skin or darkening of normal skin color immediately adjacent to the peri-wound.

#### Edema:

The presence of shiny taut skin or pitting impressions in the skin adjacent to the wound but within 4 cm of the wound margin.

#### Heat/warmth:

A detectable increase in temperature of the skin adjacent to the wound (within 4 cm of the wound margin) as compared to skin 10 cm proximal to the wound.

#### Drainage:

Increase in amount of drainage and purulent drainage.

#### Foul odor:

Assess for odor after cleansing the wound thoroughly.

#### Delayed healing:

An absence of change or an increase in volume or surface area of the wound over the preceding four weeks.

## Types of Tissue Breakdown

There are many types of wounds. Some wounds are caused by pressure and deemed pressure injuries and others such as diabetic/arterial ulcers, venous leg ulcers and skin tears are described as either partial or full thickness to indicate the depth of tissue breakdown.



### Partial Thickness Wound:<sup>1</sup>

Confined to the superficial skin layers; damage does not penetrate below the dermis and may be limited to the epidermal layers only.




### Full Thickness Wound:<sup>1</sup>

Ulceration extending through the dermis to involve subcutaneous tissue and possibly muscle/bone.

## Pressure Injuries

Localized injury to the skin and/or underlying tissue usually over a bony prominence, as a result of pressure or pressure in combination with shear and/or friction. A number of contributing or confounding factors are also associated with pressure injuries; the significance of these factors is yet to be elucidated. Pressure injuries are broken down into stages:

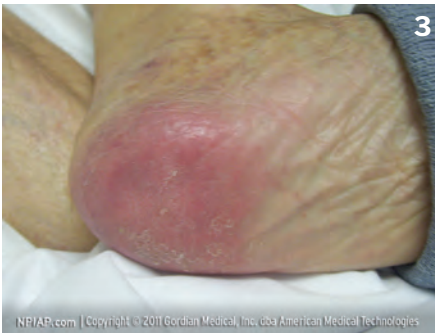

### Stage 1: Pressure Injury: Non-blanchable erythema of intact skin

Intact skin with a localized area of non-blanchable erythema, which may appear differently in darkly pigmented skin. Presence of blanchable erythema or changes in sensation, temperature, or firmness may precede visual changes. Color changes do not include purple or maroon discoloration; these may indicate deep tissue pressure injury.



### Stage 2: Pressure Injury: Partial-thickness skin loss with exposed dermis

Partial-thickness loss of skin with exposed dermis. The wound bed is viable, pink or red, moist, and may also present as an intact or ruptured serum-filled blister. Adipose (fat) is not visible and deeper tissues are not visible. Granulation tissue, slough and eschar are not present. These injuries commonly result from adverse microclimate and shear in the skin over the pelvis and shear in the heel. This stage should not be used to describe moisture associated skin damage (MASD) including incontinence associated dermatitis (IAD), intertriginous dermatitis (ITD), medical skin injury (MARSI), or traumatic wounds (skin tears, burns, abrasions).



### Stage 3: Pressure Injury: Full-thickness skin loss

Full-thickness loss of skin, in which adipose (fat) is visible in the ulcer and granulation tissue and epibole (rolled wound edges) are often present. Slough and/or eschar may be visible. The depth of tissue damage varies by anatomical location; areas of significant adiposity can develop deep wounds. Undermining and tunneling may occur. Fascia, muscle, tendon, ligament, cartilage and/ or bone are not exposed. If slough or eschar obscures the extent of tissue loss this is an Unstageable Pressure Injury.



**Stage 4: Pressure Injury: Full-thickness skin and tissue loss**

Full-thickness skin and tissue loss with exposed or directly palpable fascia, muscle, tendon, ligament, cartilage or bone in the ulcer. Slough and/or eschar may be visible. Epibole (rolled edges), undermining and/or tunneling often occur. Depth varies by anatomical location. If slough or eschar obscures the extent of tissue loss this is an Unstageable Pressure Injury.



**Deep Tissue Pressure Injury: Persistent non-blanchable deep red, maroon or purple discoloration**

Intact or non-intact skin with localized area of persistent non-blanchable deep red, maroon, purple discoloration or epidermal separation revealing a dark wound bed or blood filled blister. Pain and temperature change often precede skin color changes. Discoloration may appear differently in darkly pigmented skin. This injury results from intense and/or prolonged pressure and shear forces at the bone-muscle interface. The wound may evolve rapidly to reveal the actual extent of tissue injury, or may resolve without tissue loss. If necrotic tissue, subcutaneous tissue, granulation tissue, fascia, muscle or other underlying structures are visible, this indicates a full thickness pressure injury (Unstageable, Stage 3 or Stage 4). Do not use DTPI to describe vascular, traumatic, neuropathic, or dermatologic conditions.



**Unstageable Pressure Injury: Obscured full-thickness skin and tissue loss**

Full-thickness skin and tissue loss in which the extent of tissue damage within the ulcer cannot be confirmed because it is obscured by slough or eschar. If slough or eschar is removed, a Stage 3 or Stage 4 pressure injury will be revealed. Stable eschar (i.e. dry, adherent, intact without erythema or fluctuance) on an ischemic limb or the heel(s) should not be removed.

Photos courtesy of AAWC and NPUAP.org® Gordian Medical, Inc. dba American Medical Technologies

1. Ratliff, Catherine R, and Tomaselli, Nancy. Guideline For Prevention and Management of Pressure Ulcers. Mount Laurel: Wound, Ostomy and Continence Nurses Society, 2010. Print.
2. Raymundo J, and Wells J. Wound debridement. In: Ed. Bryant RA. Acute and chronic wounds: nursing management. 2nd ed. St Louis: Mosby, 2000. Print. Bowler P. "Progression toward healing: wound infection and the role of an advanced silver-containing Hydrofiber™ dressing." Ostomy Wound Management. 48.1 (2003): (suppl): S2-S5. Print.
3. NPIAP copyright & used with permission.

**ADDRESSING GOALS – Product Selection Guides**

Connecting the right product to the right application is critical to achieving goals with skin and wound assessment and treatment. Product selection guides for skin care and wound care are available. For more information, contact the Clinical Resource Team at 877.611.0081.

# Advanced Wound Care



For clinical information about our products, call 877.611.0081.  
To order, visit [mms.mckesson.com](https://mms.mckesson.com) or call your McKesson Medical-Surgical Account Manager.



# Collagen Dressings



## McKesson Collagen Dressings

- Advanced, topically-applied wound dressing transforms into a cool, soothing gel upon contact with exudates
- Made from fish-derived collagen, sodium alginate, carboxymethylcellulose (CMC) and ethylenediaminetetraacetic acid (EDTA)
- Alginate and CMC allow dressing to absorb wound drainage, while maintaining an optimal moisture balance to aid in the formation of granulation tissue and epithelialization
- The EDTA in the dressing removes zinc to inhibit the activity of matrix metalloproteinases (MMP), thus creating a suitable environment for wound healing
- Sheets may be trimmed to exact wound size and/or layered for the management of deep wounds
- Collagen with Silver dressings contain silver chloride, an effective antimicrobial intended to prevent colonization of the dressing

**Indications:** Full and partial thickness wounds including: pressure injuries, diabetic ulcers, ulcers caused by mixed vascular etiologies, venous ulcers, donor and graft sites, abrasions, traumatic wounds healing by secondary intention, dehisced surgical wounds, first and second-degree burns.

**Contraindications:** Patients with a known allergy or sensitivity to animal-derived collagen, any other ingredient, or on third-degree burns.

**Change Frequency:** Change dressings daily or as directed by a healthcare professional.

**Recommended Secondary Dressings:** Cover with an appropriate secondary dressing depending on the amount of exudate and frequency of dressing change.

| Item Description           | MFR #   | ITEM #  | QTY                   | HCPCS |
|----------------------------|---------|---------|-----------------------|-------|
| <b>Sterile</b>             |         |         |                       |       |
| 2" x 2", Sheet             | 16-1850 | 1171974 | 1/ea, 10/bx, 10 bx/cs | A6021 |
| 4" x 4", Sheet             | 16-1852 | 1077005 | 1/ea, 10/bx, 10 bx/cs | A6021 |
| 2" x 2", Sheet with Silver | 16-1860 | 1172341 | 1/ea, 10/bx, 10 bx/cs | A6021 |
| 4" x 4" with Silver        | 16-1862 | 1172342 | 1/ea, 10/bx, 10 bx/cs | A6021 |

## McKesson Collagen Dressings Powder

- Powder dressing made from 100% bovine Type 1 collagen - no chemicals added in the manufacturing process
- It maintains a moist wound environment at the wound surface to help promote healing
- Topically applied and forms a biodegradable gel and settles into contact with all areas of the wound as exudate is absorbed
- Application of powder conforms easily to a wound without having to cut and size a sheet dressing

**Indications:** Partial and full thickness wounds including pressure (stage I-IV) and venous ulcers, ulcers caused by mixed vascular etiologies, venous stasis and diabetic ulcers, first and second degree burns, cuts, abrasions, and surgical wounds

**Contraindications:** Patients with a known allergy or sensitivity to animal-derived collagen, any other ingredient, or on third-degree burns. This product has not been studied in pediatric populations and is intended only for use in adult patients.

**Change Frequency:** Dressings should always be changed within seven days. Collagen should only be used under the supervision of a healthcare practitioner, whether short term of four weeks or long term until the wound heals.

**Recommended Secondary Dressings:** Cover with an appropriate secondary dressing depending on the amount of exudate and frequency of dressing change.

| Item Description | MFR # | ITEM #  | QTY                   | HCPCS |
|------------------|-------|---------|-----------------------|-------|
| <b>Sterile</b>   |       |         |                       |       |
| 1 gram           | 1800P | 1171894 | 1/ea, 10/bx, 10 bx/cs | A6010 |

## Compression Bandages

### McKesson Two-Layer Compression Bandage System with Diamond Indicator

- Multilayer compression system to be used on intact skin
- Bandage system simultaneously absorbs, protects, and provides sustained therapeutic compression for up to seven days when applied as directed
- Visual diamond indicator on cohesive bandage helps with correct application
- Short stretch properties yield high working pressure and low resting pressure (in mobile patients); helps reduce oedema
- Air-permeable
- May be worn for up to a week, depending on the condition of the wound and the time for which the wound dressing is worn
- Layer 1: White - 85% polyester, 8% elastane, 7% synthetic rubber
- Layer 2: Brown - 62% polypropylene, 22% elastane, 16% synthetic rubber



| Item Description  | MFR # | ITEM #  | QTY           |
|---|-------|---------|---------------|
| <b>Non-Sterile</b><br>2 Layer, Layer 1: 4" x 7-1/10 yds., Layer 2: 4" x 6-9/10 yds. | 2006  | 1194448 | 1/bg, 8 bg/cs |

### McKesson Spandagrip™ Elastic Tubular Bandage

- Provides sustained compression and support for: strains and sprains, soft tissue injuries, support following cast removal, general edema, post-graft pressure for burn patients and rib cage injuries
- Adjusts to the contours of the body and helps distribute pressure evenly over the surface
- Stays in place without clips, pins or tape
- Full range of sizes to suit most applications
- Washable and reusable



| Item Description  | MFR # | ITEM #     | QTY     | HCPCS                   |
|---|-------|------------|---------|-------------------------|
| <b>Non-Sterile</b><br>Size C, Medium Arm / Small Ankle, 2.75" x 11 yds. |       | 182-13112C | 1112848 | 1/bx, 18 bx/cs<br>A6457 |
| Size D, Large Arm / Medium Ankle / Small Knee, 3" x 11 yds              |       | 182-13113D | 1112851 | 1/bx, 18 bx/cs<br>A6457 |
| Size E, Large Ankle / Medium Knee / Small Thigh, 3.5" x 11 yds.         |       | 182-13114E | 1112849 | 1/bx, 12 bx/cs<br>A6457 |
| Size F, Large Knee / Medium Thigh, 4" x 11 yds.                         |       | 182-13115F | 1112853 | 1/bx, 12 bx/cs<br>A6457 |
| Size G, Large Thigh, 4.5" x 11 yds.                                     |       | 182-13116G | 1112850 | 1/bx, 12 bx/cs<br>A6457 |
| Size J, Small Trunks, 6.75" x 11 yds.                                   |       | 182-13117J | 1112852 | 1/bx, 6 bx/cs<br>A6457  |

# Compression Bandages



## McKesson Unna Boot

- Gauze bandage that has been evenly impregnated with a non-hardening paste of zinc oxide to provide venous ulcer compression therapy
- Stays moist to encourage healing
- Contains no preservatives
- Non-raveling, thread-locked gauze helps prevent loose threads that can contaminate wounds
- Unique, easy-to-open, vacuum seal pouch

**Indications:** Provides venous ulcer compression therapy in actively ambulatory patients. Ideal for managing extremities with conditions such as venous ulcers that require protection and support.

**Contraindications:** Patients with a known hypersensitivity to the product itself or to its components, including zinc oxide. Unna boots are not intended to provide compression therapy in non-ambulatory patients.

**Change Frequency:** Change dressing as directed by a healthcare professional, typically once or twice a week.

**Recommended Secondary Dressings:** Cohesive bandage.

| Item Description   | MFR # | ITEM #  | QTY            | HCPCS |
|--------------------|-------|---------|----------------|-------|
| <b>Non-Sterile</b> |       |         |                |       |
| 3" x 10 yds.       | 2066S | 1220878 | 1/bx, 12 bx/cs | A6456 |
| 4" x 10 yds.       | 2067S | 1220879 | 1/bx, 12 bx/cs | A6456 |

## McKesson Unna Boot With Calamine

- Gauze bandage that has been evenly impregnated with a non-hardening paste of 3% zinc oxide and calamine to provide venous ulcer compression therapy and help ease itching
- Stays moist to encourage healing
- Unique, easy-to-open, vacuum seal pouch

**Indications:** Provides venous ulcer compression therapy in actively ambulatory patients. Dries the oozing and weeping of poison ivy, poison oak, or poison sumac. Ideal for managing extremities with conditions such as venous ulcers that require protection and support.

**Contraindications:** Patients with a known hypersensitivity to the product itself or to its components, including zinc oxide or calamine. Unna boots are not intended to provide compression therapy in non-ambulatory patients.

**Change Frequency:** Change dressing as directed by a healthcare professional, typically once or twice a week.

**Recommended Secondary Dressings:** Cohesive bandage.

| Item Description   | MFR # | ITEM #  | QTY            | HCPCS |
|--------------------|-------|---------|----------------|-------|
| <b>Non-Sterile</b> |       |         |                |       |
| 3" x 10 yds.       | 2068S | 1220880 | 1/bx, 12 bx/cs | A6456 |
| 4" x 10 yds.       | 2069S | 1220881 | 1/bx, 12 bx/cs | A6456 |

## Contact Layers

### McKesson Silicone Wound Contact Layer Dressings

- Versatile dressing to be used on a variety of different wounds when used with a secondary dressing
- Can be used as a protective layer on superficial wounds that are not discharging and on areas of skin that are very fragile
- Prevents an outer dressing from sticking to the wound and peri-wound area
- Open perforated structure allows exudate to pass vertically into the outer absorbent dressing and enables easy delivery of topical treatments

**Indications:** Can be used on many different wounds including skin tears, abrasions, blistering, lacerations, surgical wounds, partial thickness burns, and leg and foot ulcers

**Contraindications:** Patients with a known allergy or sensitivity to silicone or on third-degree burns. Not for surgical implantation.

**Change Frequency:** Dressings can be left in place for several days, depending on the condition of the wound, or as directed by a healthcare professional. The secondary dressings may need to be changed more frequently.

**Recommended Secondary Dressings:** Cover with an appropriate secondary dressing depending on the amount of exudate and frequency of dressing change.



| Item Description | MFR # | ITEM #  | QTY                  | HCPCS |
|------------------|-------|---------|----------------------|-------|
| <b>Sterile</b>   |       |         |                      |       |
| 3" x 4"          | 4816  | 1083095 | 1/ea, 10/bx, 4 bx/cs | A6206 |
| 4" x 7.2"        | 4817  | 1083096 | 1/ea, 10/bx, 7 bx/cs | A6206 |

### McKesson Oil Emulsion Dressings

- Made of high-porosity gauze impregnated with U.S.P. Petrolatum
- Open mesh fabric prevents the pooling of fluid by draining exudate away from the wound site
- Provides a moist environment conducive to wound healing
- Non-adherent to maximize comfort during dressing change or removal

| Item Description | MFR #    | ITEM # | QTY                   | HCPCS |
|------------------|----------|--------|-----------------------|-------|
| <b>Sterile</b>   |          |        |                       |       |
| 3" x 3"          | 61-77041 | 488939 | 1/ea, 50/bx, 12 bx/cs | A6222 |
| 3" x 8"          | 61-77042 | 488940 | 1/ea, 24/bx, 6 bx/cs  | A6223 |



# Cover Dressings and Composites

## McKesson Island Dressings

- Primary or secondary bordered non-adherent gauze dressing
- Excellent alternative to gauze and tape
- Offers the convenience of an all-in-one dressing by combining an absorbent non-adherent central pad surrounded by a breathable adhesive border

**Indications:** Primary or secondary dressing.

**Contraindications:** Patients with a known hypersensitivity to the product itself or to its components.

**Change Frequency:** Change dressing as directed by a healthcare professional.

**Recommended Secondary Dressings:** n/a

| Item Description        | MFR #    | ITEM # | QTY                  | HCPCS |
|-------------------------|----------|--------|----------------------|-------|
| <b>Sterile</b>          |          |        |                      |       |
| 2" x 2" (1" x 1" Pad)   | 16-89022 | 491825 | 1/ea, 25/bx, 4 bx/cs | A6219 |
| 4" x 4" (2" x 2" Pad)   | 16-89044 | 491826 | 1/ea, 25/bx, 4 bx/cs | A6219 |
| 4" x 6" (2" x 4" Pad)   | 16-89046 | 488923 | 1/ea, 25/bx, 4 bx/cs | A6219 |
| 4" x 8" (2" x 6" Pad)   | 16-89048 | 488924 | 1/ea, 25/bx, 4 bx/cs | A6219 |
| 4" x 10" (2" x 8" Pad)  | 16-89041 | 488925 | 1/ea, 25/bx, 4 bx/cs | A6219 |
| 4" x 14" (2" x 12" Pad) | 16-89042 | 488927 | 1/ea, 25/bx, 4 bx/cs | A6220 |
| 6" x 6" (4" x 4" Pad)   | 16-89066 | 491827 | 1/ea, 25/bx, 4 bx/cs | A6219 |
| 6" x 8" (4" x 6" Pad)   | 16-89068 | 488926 | 1/ea, 25/bx, 4 bx/cs | A6220 |
| <b>Non-Sterile</b>      |          |        |                      |       |
| 4" x 4" (2" x 2" Pad)   | 16-89244 | 883053 | 30/bg, 4 bg/cs       | A6413 |
| 6" x 6" (4" x 4" Pad)   | 16-89266 | 883055 | 30/bg, 4 bg/cs       | A6413 |



## McKesson Barrier Island Dressings

- Primary or secondary bordered non-adherent gauze composite dressing with film backing that protects from outside contamination
- Water resistant film layer protects the wound from contamination and acts as an effective barrier to permit showering and bathing
- Offers the convenience of an all-in-one dressing by combining an absorbent non-adherent central pad surrounded by a breathable adhesive border, with an addition of a polyurethane film barrier
- Effective water barrier to permit showering

**Indications:** As a primary or secondary dressing, the Barrier Island Dressing provides an effective barrier to outside contaminants and allows for showering with dressing in place.

**Contraindications:** Patients with a known hypersensitivity to the product itself or to its components.

**Change Frequency:** Change dressing as directed by a healthcare professional.

**Recommended Secondary Dressings:** n/a

| Item Description      | MFR #    | ITEM # | QTY                  | HCPCS |
|-----------------------|----------|--------|----------------------|-------|
| <b>Sterile</b>        |          |        |                      |       |
| 4" x 4" (2" x 2" Pad) | 16-89144 | 488920 | 1/ea, 25/bx, 4 bx/cs | A6203 |
| 6" x 6" (4" x 4" Pad) | 16-89166 | 488921 | 1/ea, 25/bx, 4 bx/cs | A6203 |



## Film Dressings

### McKesson Transparent Film Dressings, Frame Delivery

- Provides a moist environment for granulating or superficial wounds
- Semi-permeable and water-resistant, bacteria or other contaminants cannot penetrate the dressing
- Transparent film allows site to be visually monitored

**Indications:** Protect wounds and IV catheter sites, helps maintain a moist wound surface to assist in autolytic debridement, maintain a moist environment for wound healing and as a secondary dressing.

**Contraindications:** Patients with a known hypersensitivity to the product itself or to its components.

**Change Frequency:** Change dressing as directed by a healthcare professional.

**Recommended Secondary Dressings:** n/a



| Item Description | MFR # | ITEM # | QTY                   | HCPCS |
|------------------|-------|--------|-----------------------|-------|
| <b>Sterile</b>   |       |        |                       |       |
| 2-3/8" x 2-3/4"  | 4985  | 886408 | 1/ea, 100/bx, 4 bx/cs | A6257 |
| 4" x 4-3/4"      | 4986  | 886409 | 1/ea, 50/bx, 4 bx/cs  | A6258 |
| 6" x 8"          | 4987  | 886410 | 1/ea, 10/bx, 8 bx/cs  | A6258 |

## Foam Dressings

### McKesson Hydrocellular Foam Dressings

- Adhesive and non-adhesive foams are used as a primary or secondary dressing for wounds with moderate to heavy exudate
- Highly absorbent central hydrocellular foam layer effectively absorbs wound drainage
- Helps maintain a moist healing environment at the wound surface
- Skin-friendly acrylic adhesive helps ensure that the dressing remains secure
- Soft and cushioning to help provide better comfort and protection
- The adhesive and non-adhesive with film backing dressings feature a waterproof and breathable outer layer that protects the wound from outside contamination and prevents strike-through of exudate

**Indications:** Wounds with moderate to heavy exudate.

**Contraindications:** Patients with a known hypersensitivity to the product itself or to its components.

**Change Frequency:** May be left in place for up to seven days, depending on amount of drainage.

**Recommended Secondary Dressings:** Adhesive foams: n/a. Non-adhesive foams: Cover with an appropriate secondary dressing depending on the amount of exudate and frequency of dressing change or secure with medical tape, roll gauze, or stretch net.



| Item Description                           | MFR #   | ITEM #  | QTY                   | HCPCS |
|--|---------|---------|-----------------------|-------|
| <b>Acrylic Adhesive, Sterile</b>           |         |         |                       |       |
| 3" x 3"                                    | 16-4670 | 1138279 | 1/ea, 10/bx, 10 bx/cs | A6212 |
| 4" x 4"                                    | 16-4671 | 1138281 | 1/ea, 10/bx, 10 bx/cs | A6212 |
| 6" x 6"                                    | 16-4672 | 1138299 | 1/ea, 10/bx, 10 bx/cs | A6213 |
| 7" x 7" Sacral                             | 16-4673 | 1138301 | 1/ea, 10/bx, 10 bx/cs | A6213 |
| <b>Non-Adhesive, Film Backing, Sterile</b> |         |         |                       |       |
| 4" x 4"                                    | 16-4737 | 1138302 | 1/ea, 10/bx, 10 bx/cs | A6209 |
| 6" x 6"                                    | 16-4738 | 1138303 | 1/ea, 10/bx, 10 bx/cs | A6210 |
| <b>Non-Adhesive, Sterile</b>               |         |         |                       |       |
| 4" x 4"                                    | 16-4739 | 1138304 | 1/ea, 10/bx, 10 bx/cs | A6209 |

# Foam Dressings



## McKesson Hydrocellular Foam Dressings, Silicone Adhesive

- Perforated silicone gel adhesive provides gentle but firm adhesion
- Highly conformable and remains intact on skin
- Soft silicone minimizes pain to patients and trauma to wounds upon removal
- Affixes securely to dry, intact skin; seals the wound margins to help prevent exudate from leaking on surrounding tissue, minimizing risk of maceration
- Breathable, yet moisture and bacteria resistant outer film layer
- Absorbent inner hydrocellular foam pad maintains a moist wound environment to help promote healing
- Dressing may be lifted and repositioned without losing its adherent properties

**Indications:** Wide range of acute and chronic wounds like venous leg ulcers, pressure injuries, diabetic ulcers and traumatic wounds.

**Contraindications:** Patients with a known hypersensitivity to the product itself or to its components.

**Change Frequency:** Change dressing as directed by a healthcare professional.

**Recommended Secondary Dressings:** Non-bordered foam dressing may be secured with roll gauze, stretch net or medical tape.

| Item Description  | MFR # | ITEM #  | QTY                   | HCPCS |
|---|-------|---------|-----------------------|-------|
| <b>With Border, Silicone Adhesive, Sterile</b>          |       |         |                       |       |
| 3" x 3"   | 4842  | 886431  | 1/ea, 10/bx, 20 bx/cs | A6212 |
| 4" x 4"   | 4843  | 886432  | 1/ea, 10/bx, 20 bx/cs | A6212 |
| 6" x 6"   | 4844  | 886433  | 1/ea, 10/bx, 20 bx/cs | A6212 |
| 7" x 7" Sacral  | 4845  | 886434  | 1/ea, 10/bx, 20 bx/cs | A6213 |
| 9" x 9" Sacral  | 4841  | 1238306 | 1/ea, 10/bx, 20 bx/cs |       |
| <b>Without Border, Silicone Adhesive, Sterile</b>       |       |         |                       |       |
| 3" x 3"   | 4862  | 1083085 | 1/ea, 10/bx, 20 bx/cs | A6209 |
| 4" x 4"   | 4863  | 1083086 | 1/ea, 10/bx, 20 bx/cs | A6209 |
| 6" x 6"   | 4864  | 1083087 | 1/ea, 10/bx, 20 bx/cs | A6210 |
| 7" x 7" Sacral  | 4865  | 1083088 | 1/ea, 10/bx, 20 bx/cs | A6211 |
| <b>Lite, With Border, Silicone Adhesive, Sterile</b>    |       |         |                       |       |
| 3" x 3"   | 4872  | 1083089 | 1/ea, 10/bx, 20 bx/cs | A6212 |
| 4" x 4"   | 4873  | 1083090 | 1/ea, 10/bx, 20 bx/cs | A6212 |
| 6" x 6"   | 4874  | 1083091 | 1/ea, 10/bx, 20 bx/cs | A6212 |
| <b>Lite, Without Border, Silicone Adhesive, Sterile</b> |       |         |                       |       |
| 3" x 3"   | 4892  | 1083092 | 1/ea, 10/bx, 20 bx/cs | A6209 |
| 4" x 4"   | 4893  | 1083093 | 1/ea, 10/bx, 20 bx/cs | A6209 |
| 6" x 6"   | 4894  | 1083094 | 1/ea, 10/bx, 20 bx/cs | A6210 |

## McKesson PHMB Hydrophilic Foam Dressings

- Sterile wound dressing impregnated with polyhexamethylene biguanide (PHMB) which kills and inhibits the growth of bacteria
- Constructed of waterproof polyurethane bacterial barrier film, hydrophilic PHMB polyurethane foam pad, and perforated wound contact layer
- In-vitro testing demonstrated dressings to be effective against the following three gram positive bacteria, three gram negative bacteria, and two yeast challenge organisms within the dressing: MRSA, MRSE, VRE, Escherichia coli, Klebsiella pneumoniae, Pseudomonas aeruginosa, Candida albicans, and Rhodotorula mucilaginosa
- Semi-occlusive, allowing the exchange of gases such as oxygen and water vapor
- Fast wicking action and excellent absorbency for moderately to heavily exuding wounds

**Indications:** Temporary management of post-surgical incisions, pressure sores, venous stasis ulcers, diabetic ulcers, donor sites, abrasions, lacerations, first and second degree burns, dermatologic disorders, other wounds inflicted by trauma, and as a secondary dressing or cover dressing for packed wounds

**Contraindications:** Not intended as a treatment for infections or third degree burns. Do not use on patients with known sensitivity to any components of the dressing. Do not use with oxidizing solutions such as hypochlorite or hydrogen peroxide, as these can break down the absorbent polyurethane component of the dressing and can also break down the PHMB

**Change Frequency:** Can remain in situ for up to seven days, dependent on patient condition and the level of exudate. Initially, it may be necessary to change the dressing every 24 hours or whenever good wound care practice dictates. Duration of product use is not recommended for longer than 30 days.

**Recommended Secondary Dressings:** Adhesive: n/a. Non-adhesive: tape or compression bandaging, roll gauze, stretch net.



| Item Description   | MFR # | ITEM #  | QTY                   | HCPCS |
|--|-------|---------|-----------------------|-------|
| <b>Non-Adhesive, Anti-Bacterial, Without Border, Sterile</b> |       |         |                       |       |
| 4" x 4"  | 4940  | 1138632 | 1/ea, 10/bx, 10 bx/cs | A6209 |

# Gelling Fiber

## McKesson Gelling Fiber Dressings

- Sterile dressing comprising of an absorbent non-woven fiber sheet or rope composed of carboxymethylcellulose (CMC) and a high G (guluronic acid) calcium alginate
- Manages exudates in in moderately to heavily exuding wounds and creates a favorable environment for effective wound management
- Soft and conformable, highly absorbent and high gelling
- A reinforcement layer allows intact application and removal of the dressing with low risk of maceration

**Indications:** Can be used in the management of moderate to heavily exuding chronic and acute wounds, including: decubitus ulcers (pressure ulcers), venous and arterial leg ulcers, diabetic ulcers, graft and donor sites, post-operative surgical wounds, superficial and partial thickness burns, and cavity wounds

**Contraindications:** Patients with a known sensitivity to the dressing or its components: alginates, carboxymethylcellulose (CMC), or nylon. Not for surgical implantation.

**Change Frequency:** Can remain in situ for up to seven days, dependent on patient condition and the level of exudate. Reapply the dressing when the secondary dressing has reached its absorbent capacity or whenever good wound care practice dictates that the dressing should be changed.

**Recommended Secondary Dressings:** Cover and secure the dressing with an appropriate secondary dressing such as foam or barrier island dressing. Can be used under compression.



| Item Description | MFR # | ITEM #  | QTY                   | HCPCS |
|------------------|-------|---------|-----------------------|-------|
| <b>Sterile</b>   |       |         |                       |       |
| 2" x 2"          | 87200 | 1138625 | 1/ea, 10/bx, 10 bx/cs | A6196 |
| 4" x 4-3/4"      | 87400 | 1138627 | 1/ea, 10/bx, 10 bx/cs | A6197 |
| 6" x 6"          | 87600 | 1138628 | 1/ea, 5/bx, 10 bx/cs  | A6197 |
| 18" Rope         | 87180 | 1138626 | 1/ea, 5/bx, 10 bx/cs  | A6199 |

## McKesson Gelling Fiber Dressings with Silver

- Sterile dressing comprising of an absorbent non-woven fiber sheet or rope containing ionic silver
- Manages exudates in moderate to heavily exuding wounds by absorbing exudate from the wound to form a soft gel that intimately conforms to the wound bed and aids in maintaining a moist wound environment
- Contains ionic silver that effectively manages and suppresses colonization and proliferation of the following: three gram positive bacteria (Vancomycin- resistant enterococcus (VRE), Streptococcus mutans and Staphylococcus aureus), three gram negative bacteria (Enterobacter cloacae, Klebsiella pneumoniae and Serratia marcescens), and yeast (Candida albicans) within the dressing for up to seven days.
- A reinforcement layer allows intact application and removal of the dressing

**Indications:** Can be used in the management of moderate to heavily exuding chronic and acute wounds, including: pressure ulcers (partial and full thickness), leg ulcers (venous stasis ulcers, arterial ulcers and leg ulcers of mixed etiology), diabetic foot ulcers, surgical wounds that heal by primary intent such as dermatological and surgical incisions, surgical wounds left to heal by secondary intention such as dehisced surgical incisions and donor sites, and traumatic wounds.

**Contraindications:** Patients with a known sensitivity to the dressing or its components: silver, alginates, carboxymethylcellulose (CMC), or nylon. Not for surgical implantation.

**Change Frequency:** Can remain in situ for up to seven days, dependent on patient condition and the level of exudate. Reapply the dressing when the secondary dressing has reached its absorbent capacity or whenever good wound care practice dictates that the dressing should be changed.

**Recommended Secondary Dressings:** Cover and secure the dressing with an appropriate secondary dressing, such as foam or super absorbent dressing. Can be used under compression.



| Item Description                | MFR #  | ITEM #  | QTY                   | HCPCS |
|---------------------------------|--------|---------|-----------------------|-------|
| <b>High Absorbency, Sterile</b> |        |         |                       |       |
| 2" x 2"                         | 177200 | 1162304 | 1/ea, 10/bx, 10 bx/cs | A6196 |
| 4" x 4-3/4"                     | 177400 | 1162306 | 1/ea, 10/bx, 10 bx/cs | A6197 |
| 6" x 6"                         | 177600 | 1162307 | 1/ea, 5/bx, 10 bx/cs  | A6197 |
| 8" x 12"                        | 177812 | 1162308 | 1/ea, 5/bx, 10 bx/cs  | A6198 |
| 18" Rope                        | 177180 | 1162310 | 1/ea, 5/bx, 10 bx/cs  | A6199 |

# Honey Dressings



## MEDIHONEY® Gel

- Moist dressing made of 80% Active Leptospermum Honey in combination with natural gelling agents.
- Provides increased stability at the site of the wound due to its natural gelling agents
- Promotes a moist environment conducive to wound healing

**Indications:** Non-draining to moderately exuding wounds such as diabetic foot ulcers, leg ulcers (venous stasis ulcers, arterial ulcers and leg ulcers of mixed etiology), pressure injuries (partial and full thickness), first and second-degree partial thickness burns, donor sites, and traumatic and surgical wounds.

**Contraindications:** Third-degree burns and patients with a known sensitivity to honey.

**Change Frequency:** Dressing change frequency will depend on the condition of the patient as well as the level of wound exudate. Reapply when MEDIHONEY® Gel dressing or the secondary dressing has reached its absorbent capacity or as directed by a wound care professional.

**Recommended Secondary Dressings:** Cover with an appropriate secondary dressing (such as a hydrocolloid, foam, or island/composite).

| Item Description | MFR # | ITEM # | QTY                  | HPCS  |
|------------------|-------|--------|----------------------|-------|
| <b>Sterile</b>   |       |        |                      |       |
| 0.5 fl. oz. Tube | 31805 | 788895 | 1/ea, 10/bx, 4 bx/cs | A4649 |
| 1.5 fl. oz. Tube | 31815 | 788894 | 1/ea, 12/cs          | A4649 |



## MEDIHONEY® Paste

- Moist dressing made of 100% Active Leptospermum Honey
- Assists in maintaining a moist environment for optimal wound healing, aids in autolytic debridement, and can be removed without causing trauma to the wound

**Indications:** For use in hard-to-dress areas including in tunneled wounds or wounds with undermining.

**Contraindications:** Third-degree burns and patients with a known sensitivity to honey.

**Change Frequency:** Dressing change frequency will depend on the condition of the patient as well as the level of wound exudate. The dressing can be left in place under compression and off-loading devices. Reapply when the secondary dressing has reached its absorbent capacity or as directed by a wound care professional.

**Recommended Secondary Dressings:** Cover with an appropriate secondary dressing (such as hydrocolloid, foam, or island/composite).

| Item Description | MFR # | ITEM # | QTY            | HPCS  |
|------------------|-------|--------|----------------|-------|
| <b>Sterile</b>   |       |        |                |       |
| 0.5 fl. oz. Tube | 31505 | 825485 | 10/bx, 4 bx/cs | A4649 |
| 1.5 fl. oz. Tube | 31515 | 699433 | 1/ea, 12/cs    | A4649 |


MEDIHONEY® is a medical-grade honey derived from the pollen and nectar of a specific Leptospermum species of tea tree in New Zealand and Australia. It can be used for wound bed preparation and has demonstrated efficacy from the start of wound treatment through to closure.

MEDIHONEY® is unique among all types of honey in that it maintains its effectiveness to assist in autolytic debridement even in the presence of wound fluid. It has high osmotic potential, which promotes a moist wound environment. It lowers the pH levels within the wound and contributes to an acidic environment that promotes healing.

A range of MEDIHONEY® dressings are available to help in the management of acute, chronic, and stalled wounds and burns.

MEDIHONEY® is a registered trademark of MEDIHONEY Pty Ltd. and licensed to Derma Sciences, Inc. MEDIHONEY® is not a private label product of McKesson but is available for distribution through McKesson's exclusive brand of Advanced Wound Care products.

# Honey Dressings

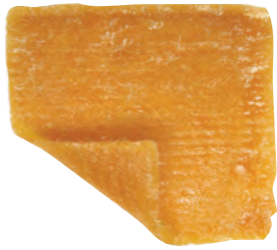

## MEDIHONEY® Calcium Alginate Dressing

- Absorbent primary wound dressing made of Active Leptospermum Honey impregnated into a calcium alginate pad
- As exudate is absorbed, the alginate forms a gel, which assists in maintaining a moist environment for optimal wound healing

**Indications:** Moderate to heavily exuding wounds such as: diabetic foot ulcers, leg ulcers (venous stasis ulcers, arterial ulcers and leg ulcers of mixed etiology), pressure injuries (partial and full thickness), first and second degree partial thickness burns, donor sites, and traumatic and surgical wounds.

**Contraindications:** Third-degree burns and patients with a known sensitivity to alginates or honey.

**Change Frequency:** Change frequency will depend on the condition of the patient as well as the level of wound exudate. Reapply when dressing has reached its absorbent capacity or as directed by a wound care professional.

**Recommended Secondary Dressings:** Cover with an appropriate secondary dressing (such as a super absorbent dressing, foam, or island/composite).

| Item Description | MFR # | ITEM # | QTY                   | HCPCS |
|------------------|-------|--------|-----------------------|-------|
| <b>Sterile</b>   |       |        |                       |       |
| 2" x 2", Sheet   | 31022 | 683998 | 1/ea, 10/bx, 10 bx/cs | A4649 |
| 4" x 5", Sheet   | 31045 | 702990 | 1/ea, 10/bx, 5 bx/cs  | A4649 |
| 3/4" x 12" Rope  | 31012 | 688573 | 1/ea, 5/bx, 4 bx/cs   | A4649 |



## MEDIHONEY® Hydrogel Dressing

- Super absorbent polymer technology provides absorption and fluid handling capability
- Delivers soothing, cooling effect on application
- Transparent dressing offers visibility to the wound

**Indications:** To cover and/or fill partial-to-full thickness wound with light-to-moderate drainage

**Contraindications:** Third-degree burns and patients with a known sensitivity to honey.

**Change Frequency:** Change frequency will depend on the condition of the patient as well as the level of wound exudate. Reapply when dressing has reached its absorbent capacity or as directed by a wound care professional.

**Recommended Secondary Dressings:** Cover the non-bordered Hydrogel Dressing with appropriate secondary dressing such as transparent film, roll gauze or stretch net. (Adhesive Border needs no secondary dressing.)

| Item Description                           | MFR # | ITEM #  | QTY                  | HCPCS |
|--|-------|---------|----------------------|-------|
| <b>Non-Bordered, Non-Adhesive, Sterile</b> |       |         |                      |       |
| 2.4" x 2.4"                                | 31620 | 1030998 | 1/ea, 10/bx, 5 bx/cs | A6242 |
| 4.3" x 4.3", Sheet                         | 31640 | 1030999 | 1/ea, 10/bx, 5 bx/cs | A6243 |
| <b>Adhesive Border, Sterile</b>            |       |         |                      |       |
| 4.3" x 4.3" with 2.8" x 2.8" Pad           | 31720 | 1031000 | 1/ea, 10/bx, 5 bx/cs | A4625 |
| 6" x 6" with 4.5" x 4.5" Pad               | 31740 | 1031001 | 10/bx, 5 bx/cs       | A4626 |

# Hydrocolloids



## McKesson Hydrocolloid Dressings

- Primary or secondary dressing for wounds with light to moderate exudate
- Pliable, absorbent, and waterproof wound dressings that can provide an effective barrier to microorganisms
- Hydrocolloid layer reacts with cellular material from the wound to form a gel like covering to help maintain a moist wound environment
- Cohesive dressing removal system does not break down or leave residue in the wound and facilitates easy removal
- Moisture resistant outer barrier film helps prevent urine, feces and/or moisture from penetrating the dressing reducing the risk of contamination to the wound
- Thin and film back versions feature smooth, tapered design to help minimize friction and prevent dressing edges from rolling up
- Thin hydrocolloid is semi-transparent and allows visualization of the wound

**Indications:** Management of partial and full thickness wounds such as leg ulcers and pressure injuries. Help maintain a moist wound healing environment to assist in autolytic debridement and protect the wound from external contamination.

**Contraindications:** Patients with known hypersensitivities to the components of the dressing or third-degree burns. Conditions that lead to the formation of the wound (pressure, venous insufficiency, etc.) must be considered and addressed as part of the total wound care treatment plan.

**Change Frequency:** Dressing change frequency is dependent upon the amount of drainage. May be left in place for up to seven days or as directed by a healthcare professional.

**Recommended Secondary Dressings:** n/a

| Item Description             | MFR # | ITEM # | QTY                   | HCPCS |
|------------------------------|-------|--------|-----------------------|-------|
| <b>Thin, Sterile</b>         |       |        |                       |       |
| 2" x 2", Sheet               | 1882  | 882980 | 1/ea, 20/bx, 20 bx/cs | A6234 |
| 4" x 4", Sheet               | 1883  | 882982 | 1/ea, 10/bx, 20 bx/cs | A6234 |
| 6" x 6", Sheet               | 1884  | 882983 | 1/ea, 10/bx, 20 bx/cs | A6235 |
| 6" x 7" Sacral               | 1885  | 882986 | 1/ea, 10/bx, 20 bx/cs | A6236 |
| <b>Film Backing, Sterile</b> |       |        |                       |       |
| 2" x 2", Sheet               | 1886  | 882991 | 1/ea, 20/bx, 20 bx/cs | A6237 |
| 4" x 4", Sheet               | 1887  | 882992 | 1/ea, 10/bx, 20 bx/cs | A6237 |
| 6" x 7" Sacral               | 1888  | 882993 | 1/ea, 5/bx, 16 bx/cs  | A6238 |
| <b>Foam Backing, Sterile</b> |       |        |                       |       |
| 4" x 4", Sheet               | 1889  | 882995 | 1/ea, 10/bx, 10 bx/cs | A6234 |
| 6" x 6", Sheet               | 1890  | 883000 | 1/ea, 10/bx, 10 bx/cs | A6235 |

# Hydrogels



## McKesson Hydrogel Amorphous Wound Dressings

- Clear, amorphous, non-greasy hydrogel
- Helps maintain a moist wound healing environment to assist in autolytic debridement
- To protect peri-wound skin from maceration, apply a moisture barrier ointment or barrier film

**Indications:** Provide moist wound healing environment and help promote autolytic debridement.

**Contraindications:** Patients with a known hypersensitivity to the product itself or to its components.

**Change Frequency:** Change daily or as directed by a healthcare professional.

**Recommended Secondary Dressings:** Cover with an appropriate secondary dressing depending on the amount of exudate and frequency of dressing change.

| Item Description                     | MFR # | ITEM # | QTY         | HCPCS |
|--------------------------------------|-------|--------|-------------|-------|
| <b>Non-Sterile</b>                   |       |        |             |       |
| 1 fl. oz. Accordion Tube             | 1721  | 949422 | 1/ea, 30/cs | A6248 |
| 3 fl. oz. Multiple-use Standard Tube | 1722  | 949423 | 1/ea, 12/cs | A6248 |



## McKesson Hydrogel Sheet Dressings

- Primary dressing that provides a cooling and soothing effect on superficial wounds
- Helps maintain a moist wound healing environment to assist in autolytic debridement
- To protect peri-wound skin from maceration, apply a moisture barrier ointment or barrier film

**Indications:** Provide moist wound healing environment and help promote autolytic debridement.

**Contraindications:** Patients with a known hypersensitivity to the product itself or to its components.

**Change Frequency:** Change as directed by a healthcare professional.

**Recommended Secondary Dressings:** Cover with an appropriate secondary dressing depending on the amount of exudate and frequency of dressing change.

| Item Description | MFR #    | ITEM # | QTY                  | HCPCS |
|------------------|----------|--------|----------------------|-------|
| <b>Sterile</b>   |          |        |                      |       |
| 2" x 2", Sheet   | 61-82022 | 494192 | 1/ea, 10/bx, 4 bx/cs | A6242 |
| 4" x 4", Sheet   | 61-82044 | 494194 | 1/ea, 10/bx, 4 bx/cs | A6242 |

## McKesson Puracyn® Plus Professional Formula Antimicrobial Hydrogel

- Designed to add and maintain moisture in the wound bed to facilitate autolytic debridement and the wound healing process
- Won't harm healthy tissues
- Puracyn Plus contains hypochlorous, a known antimicrobial. Intended for wounds with little to no drainage. Shake the hydrogel well before use. Cover with appropriate secondary dressing such as Barrier Island Dressing. Change dressing as directed by a healthcare professional.

| Item Description   | MFR #    | ITEM #  | QTY        | HCPCS |
|--------------------|----------|---------|------------|-------|
| <b>Non-Sterile</b> |          |         |            |       |
| 3 fl. oz.          | 186-6542 | 1113215 | 1/ea, 6/cs | A6248 |



## Hydrogels

### McKesson Silver Antimicrobial Wound Gel

- Water-based gel with silver for use in wound healing
- Nominal silver content: 0.11% (1100µg/gram)
- Opaque, amorphous hydrogel containing a high (>80%) water content and water-soluble polymer chains
- Water donation which makes the gel effective in assisting the debridement and desloughing process in dry necrotic wounds, while maintaining a moist wound environment for optimal wound healing
- Contains an antimicrobial silver compound (silver carbonate) creating an effective barrier to bacterial penetration by inhibiting the growth of broad spectrum of microorganisms which come into contact with the gel
- For the management of dry to moderate exuding partial and full thickness wounds such as: pressure ulcers, leg ulcers, diabetic ulcers, graft and donor sites, post-operative surgical wounds, trauma wounds, first- and second-degree burns, abrasions, and lacerations
- In-vitro testing has shown that silver antimicrobial wound gel is effective against the following broad spectrum of microorganisms; Staphylococcus aureus, including MRSA, Staphylococcus epidermidis, including MRSE, Streptococcus pyogenes, Pseudomonas aeruginosa, Escherichia coli and fungi such as Candida albicans and Aspergillus brasiliensis when they come into contact with the gel. Clinical implications of the in-vitro findings are unknown.
- Provides protection against a range of bacteria and fungi



| Item Description   | MFR # | ITEM #  | QTY            | HCPCS |
|--------------------|-------|---------|----------------|-------|
| <b>Non-Sterile</b> |       |         |                |       |
| 1.575-ounce tube   | 1726  | 1241462 | 1/ea, 10 bx/cs | A6244 |

## Hydrophilic Dressings

### McKesson Hydrophilic Wound Dressing with Zinc, Vitamins A and B6, Calcium and Magnesium

- Formulated with a patented zinc-nutrient formulation, and along with its balanced pH technology, provides a moist, mildly acidic environment conducive to wound healing
- Can be applied directly to the wound bed and provides filler for the wounds

**Indications:** Pressure injuries Stages 1-4, surgical incisions, tissue trauma, partial thickness wounds, thermal burns, diabetic and venous stasis ulcers

**Contraindications:** Patients with a known hypersensitivity to the product itself or to its components.

**Change Frequency:** Change dressing daily or as directed by a physician.

**Recommended Secondary Dressings:** Cover with an appropriate secondary dressing depending on the amount of exudate and frequency of dressing change.



| Item Description   | MFR #    | ITEM # | QTY         | HCPCS |
|--------------------|----------|--------|-------------|-------|
| <b>Non-Sterile</b> |          |        |             |       |
| 3 fl. oz. Tube     | 61-SPD03 | 488915 | 1/ea, 12/cs | A6261 |

# Impregnated Dressings

## McKesson Xeroform Petrolatum Dressings

- Contains 3% bismuth tribromophenate to provide bacteriostatic properties and help to reduce risk of infection and deodorize the wound
- Helps maintain a moist wound healing environment to assist in autolytic debridement
- Primary dressing for wounds with light or no exudate

**Indications:** Open wounds, cancer resections, burns, donor sites and surgical incisions.

**Contraindications:** Patients with a known hypersensitivity to the product itself or to its components.

**Change Frequency:** Change dressing as directed by a healthcare professional.

**Recommended Secondary Dressings:** Cover with an appropriate secondary dressing depending on the amount of exudate and frequency of dressing change.



| Item Description | MFR # | ITEM #  | QTY                  | HCPCS |
|------------------|-------|---------|----------------------|-------|
| <b>Sterile</b>   |       |         |                      |       |
| 2" x 2"          | 2202  | 865270  | 1/ea, 25/bx, 6 bx/cs | A6222 |
| 4" x 4"          | 2206  | 865271  | 1/ea, 25/bx, 6 bx/cs | A6222 |
| 5" x 9"          | 2207  | 864640  | 1/ea, 50/bx, 4 bx/cs | A6222 |
| 1" x 8"          | 2201  | 864638  | 1/ea, 50/bx, 4 bx/cs | A6222 |
| 4" x 3 yds.      | 2243R | 1086958 | 1/rl, 6/bx, 6 bx/cs  | A6266 |



## McKesson Petrolatum Gauze Non-Adhering Dressings

- Non-adhering occlusive dressing consisting of fine-mesh absorbent gauze impregnated with white petrolatum

| Item Description | MFR #    | ITEM # | QTY                     | HCPCS |
|------------------|----------|--------|-------------------------|-------|
| <b>Sterile</b>   |          |        |                         |       |
| 3" x 9"          | 61-20056 | 488942 | 1/pk, 12 pk/bx, 6 bx/cs | A6223 |



## McKesson Hydrogel Impregnated Gauze Dressings

- Primary dressing for wounds with light or no exudate that add or maintain moisture
- Helps maintain a moist wound healing environment to assist in autolytic debridement
- To protect peri-wound skin from maceration, apply a moisture barrier ointment or barrier film

**Indications:** Provide moist wound healing environment and help promote autolytic debridement.

**Contraindications:** Patients with a known hypersensitivity to the product itself or to its components.

**Change Frequency:** Change daily or as directed by a healthcare professional.

**Recommended Secondary Dressings:** Cover with an appropriate secondary dressing depending on the amount of exudate and frequency of dressing change.

| Item Description | MFR #    | ITEM # | QTY            | HCPCS |
|------------------|----------|--------|----------------|-------|
| <b>Sterile</b>   |          |        |                |       |
| 2" x 2"          | 61-53022 | 488935 | 10/bx, 4 bx/cs | A6231 |
| 4" x 4"          | 61-53044 | 488936 | 10/bx, 4 bx/cs | A6231 |

# Skin Fold Management

## McKesson Moisture Wicking Fabric with Antimicrobial Silver

- Non-sterile skin protectant which is made of a stretch polyester fiber impregnated with antimicrobial metallic silver
- Designed to provide moisture transportation to keep skin dry whilst reducing odor
- Reduces colonization of bacteria and yeasts such as Staphylococcus aureus, Staphylococcus epidermidis, Pseudomonas aeruginosa and Candida albicans within the fabric
- Acts as a lubrication aid thereby reducing skin-to-skin friction and provides a protective environment for the skin.

**Indications:** Indications: Management of skin folds and other skin-on-skin contact areas

**Contraindications:** should not be placed directly onto an open wound or breached skin. Do not use on individuals with known sensitivity to polyester, nylon and silver.

**Change Frequency:** Each piece of may be used up to a maximum of 5 days, depending on fabric soiling, odor, amount of moisture and general skin condition.

**Recommended Secondary Dressings:** Secure the fabric with the skin fold, with a small amount of medical tape, and/or tucking under clothing



| Item Description | MFR #     | ITEM #  | QTY                  | HCPCS |
|------------------|-----------|---------|----------------------|-------|
| <b>Sterile</b>   |           |         |                      |       |
| 10" x 144", Roll | 16-10144R | 1152160 | 1/ea, 12/cs          | A6250 |
| 10" x 36", Sheet | 16-1036S  | 1152161 | 1/ea, 10/bx, 6 bx/cs | A6250 |

# Super Absorbent Dressings

## McKesson Super Absorbent Dressings

- These dressings hold more fluid; lock fluid inside the dressing, helping to decrease the risk of maceration and keep the harmful wound fluid components away from the wound and peri-wound skin
- Super absorbent polymers bind fluid and convert it to a gel
- Contains protective outer layer and non-adherent wound contact layer for easy removal

**Indications:** Primary or secondary dressing for partial thickness or full thickness wounds. Use when there is moderate to large amounts of drainage and extra absorbency is needed to help better manage the fluid and decrease the frequency of dressing changes. Excellent dressing for use under compression for heavily draining venous leg ulcers.

**Contraindications:** Patients with a known hypersensitivity to the product itself or to its components, including the sodium polyacrylate super absorbent polymer. Not for use in tunneling wound pockets as the dressing can expand considerably when wound exudates are absorbed. Not for use on wounds with little to no exudates, as this may cause drying out of the wound.

**Change Frequency:** Change dressing as directed by a healthcare professional.

**Recommended Secondary Dressings:** Elastic net retainer dressing or conforming bandage. Medical adhesive tape should not be used due to expansion of the dressing as it absorbs wound fluid.



| Item Description | MFR #    | ITEM # | QTY                   | HCPCS |
|------------------|----------|--------|-----------------------|-------|
| <b>Sterile</b>   |          |        |                       |       |
| 4" x 5"          | 61-89545 | 682970 | 1/ea, 10/bx, 10 bx/cs | A6252 |
| 6" x 9"          | 61-89569 | 682971 | 1/ea, 10/bx, 5 bx/cs  | A6253 |


## Wound Measuring Devices



### McKesson Wound Measuring Guide, Paper Ruler

- Paper ruler measures up to 6" or 15 cm, graduated in metric and English
- Measuring the wound provides objective data that is critical in assessing progression of healing
- Calibrated measuring scales

| Item Description   | MFR #     | ITEM # | QTY                       |
|--------------------|-----------|--------|---------------------------|
| <b>Non-Sterile</b> |           |        |                           |
| 6" Paper Ruler     | 533-LR/50 | 871649 | 50/pd, 12 pd/bg, 24 bg/cs |

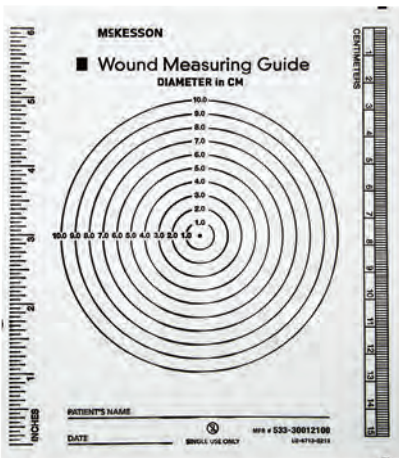

### McKesson Wound Measuring Guide, Circular Grid

- Clear plastic sheet type displaying circular grid, graduated in metric and English
- Measuring the wound provides objective data that is critical in assessing progression of healing
- Calibrated measuring scales

| Item Description      | MFR #        | ITEM # | QTY              |
|-----------------------|--------------|--------|------------------|
| <b>Non-Sterile</b>    |              |        |                  |
| 5" x 7" Circular Grid | 533-30012100 | 580718 | 100/pk, 60 pk/cs |

# Wound Closure



For clinical information about our products, call 877.611.0081.  
To order, visit [mms.mckesson.com](https://mms.mckesson.com) or call your McKesson Medical-Surgical Account Manager.

## WOUND CLOSURE

# Skin Closure Strips



## McKesson Skin Closure Strips

- Ideal for closing small cuts and wounds
- Unique microporous construction is designed to permit skin ventilation and prevent maceration
- Easy to apply and comfortable to wear
- Flexible nonwoven polyamide material conforms to the skin
- Skin-friendly adhesive

| Item Description                    | MFR # | ITEM # | QTY                      |
|-------------------------------------|-------|--------|--------------------------|
| <b>Non-Reinforced, Tan, Sterile</b> |       |        |                          |
| 1/8" x 3"                           | 3000  | 876299 | 5/pk, 50 pk/bx, 4 bx/cs  |
| 1/4" x 3"                           | 3001  | 876300 | 3/pk, 50 pk/bx, 4 bx/cs  |
| 1/4" x 1-1/2"                       | 3002  | 876301 | 6/pk, 50 pk/bx, 4 bx/cs  |
| 1/4" x 4"                           | 3003  | 876302 | 10/pk, 50 pk/bx, 4 bx/cs |
| 1/2" x 4"                           | 3004  | 876303 | 6/pk, 50 pk/bx, 4 bx/cs  |
| 1" x 5"                             | 3005  | 876304 | 4/pk, 25 pk/bx, 4 bx/cs  |
| <b>Non-Reinforced, Tan, Sterile</b> |       |        |                          |
| 1/8" x 3"                           | 3006  | 876305 | 5/pk, 50 pk/bx, 4 bx/cs  |
| 1/4" x 3"                           | 3007  | 876306 | 3/pk, 50 pk/bx, 4 bx/cs  |
| 1/4" x 1-1/2"                       | 3008  | 876307 | 6/pk, 50 pk/bx, 4 bx/cs  |
| 1/4" x 4"                           | 3009  | 876308 | 10/pk, 50 pk/bx, 4 bx/cs |
| 1/2" x 4"                           | 3010  | 876309 | 6/pk, 50 pk/bx, 4 bx/cs  |
| 1" x 5"                             | 3011  | 876310 | 4/pk, 25 pk/bx, 4 bx/cs  |

## Topical Skin Adhesive

### McKesson LIQUIBAND® Topical Adhesive

- McKesson LiquiBand® is indicated for the closure of topical skin incisions including laparoscopic incisions, and trauma induced lacerations in areas of low skin tension that are simple, thoroughly cleansed, and have easily approximated edges.
- No-clog applicator facilitates simple and precise dosage
- Preassembled; ready for immediate use
- Saves time—no dressings; no suture/staple removal
- Antibacterial film helps protect the wound
- May be used in conjunction with but not in place of deep dermal stitches
- Flexible nonwoven polyamide material conforms to the skin
- Skin-friendly adhesive



| Item Description  | MFR #    | ITEM #  | QTY                  |
|---|----------|---------|----------------------|
| McKesson LiquiBand® Exceed™ XS Topical Skin Adhesive, 0.4 mL Applicator | 122-LBXS | 1106591 | 1/ea, 10/bx, 6 bx/cs |
| McKesson LiquiBand® Flow Control, 0.5 Gram Applicator                   | 122-LFC  | 728085  | 1/ea, 12/bx, 6 bx/cs |
| McKesson LiquiBand® Exceed, 0.8 Gram Applicator                         | 122-LBX  | 948660  | 1/ea, 10/bx, 6 bx/cs |

# Kits and Trays



**For clinical information about our products, call 877.611.0081.  
To order, visit [mms.mckesson.com](https://mms.mckesson.com) or call your McKesson Medical-Surgical Account Manager.**

# Dressing Change Trays

## McKesson Dressing Change Tray


- Tray components packaged in the proper order for maximum efficiency and effectiveness
- Helps contain costs and promote standardized procedures
- Encourages consistent product usage and adherence to protocol
- Eliminates the need to search for missing procedural components or the waste of extra, unused items

**Contents:**

- (1) Drape, 17" x 19" (white)
- (1) Mask
- (1) Pair Gloves Vinyl Yellow Stretch Powder Free
- (1) 4" x 4" 4 Ply Non-woven gauze
- (1) 2" x 2" 4 Ply Non-woven gauze
- (1) 2" x 2" Non adherent pad
- (1) 2" x 2" Non adherent split pad
- (1) ChloraPrep® 3mL applicator (with DFU)
- (1) PVP swabsticks 3-pack
- (1) 4" x 4.75" Tegaderm™ dressing
- (1) Dressing change label
- (1) Roll of Transpore™ tape



| Item Description                       | MFR #   | ITEM # | QTY         |
|--|---------|--------|-------------|
| <b>Sterile</b><br>Dressing Change Tray | 25-2835 | 471087 | 1/ea, 30/cs |

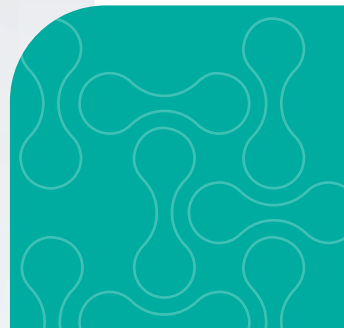

## Dressing Change Trays



### McKesson Dressing Change Tray

- Tray components packaged in the proper order for maximum efficiency and effectiveness
- Helps contain costs and promote standardized procedures
- Encourages consistent product usage and adherence to protocol
- Eliminates the need to search for missing procedural components or the waste of extra, unused items

**Contents:**

- (1) Drape, 17" x 19" (white)
- (1) Mask
- (1) Pair Gloves Vinyl Yellow Stretch Powder Free
- (1) 3" x 3" 8 Ply Non-woven gauze
- (1) 2" x 2" Non adherent pad
- (1) PVP swabsticks 3 pack
- (1) BZK swabstick
- (1) PVP ointment, 1g
- (1) Wire scissors sharp/sharp (4.5")
- (1) Alcohol swabstick 3 pack
- (1) 6" Cotton tipped applicator
- (1) 4" x 4.75" Tegaderm™ dressing
- (1) Dressing change label
- (1) Roll of Transpore™ tape

| Item Description | MFR # | ITEM # | QTY |
|------------------|-------|--------|-----|
|------------------|-------|--------|-----|

**Non-Sterile**

|                      |         |        |             |
|----------------------|---------|--------|-------------|
| Dressing Change Tray | 25-2026 | 221151 | 1/ea, 30/cs |
|----------------------|---------|--------|-------------|



### McKesson Dressing Change Tray

- Tray components packaged in the proper order for maximum efficiency and effectiveness
- Helps contain costs and promote standardized procedures
- Encourages consistent product usage and adherence to protocol
- Eliminates the need to search for missing procedural components or the waste of extra, unused items

**Contents:**

- (1) Poly-lined sterile towel
- (1) Abdominal pad
- (1) Safety pin
- (1) Iodophor PVP swabstick
- (4) 4" x 4" Gauze sponges
- (1) Sharp/blunt scissors
- (1) Polybag with twist tie
- (1) Roll medical tape
- (1) Transfer forceps
- (1) Posi-grip plastic forceps
- (1) Tray, latex-free

| Item Description | MFR # | ITEM # | QTY |
|------------------|-------|--------|-----|
|------------------|-------|--------|-----|

**Non-Sterile**

|                      |      |         |             |
|----------------------|------|---------|-------------|
| Dressing Change Tray | 7598 | 1089517 | 1/ea, 24/cs |
|----------------------|------|---------|-------------|

## NEVER COMPROMISE ON CARE™

With McKesson Brands, you'll never have to compromise. We deliver quality products at competitive prices, backed by our 100% satisfaction guarantee. We have decades of experience in the market, and we offer you support and work to help ensure you have the products you need, when you need them — making it even easier for you to provide the best care for your patients.



### Satisfaction Guaranteed

If you are not completely satisfied with any McKesson Brands product, you may return it for a full refund or credit within 12 months from the purchase date. McKesson Brands satisfaction guarantee is limited to products purchased directly from McKesson. If a McKesson Brands product is not purchased directly from McKesson, any return request or request for refund must be directed to the selling party.



McKesson Medical-Surgical  
9954 Mayland Drive  
Richmond, VA 23233

The product information contained in this document, including the product images and additional product materials, was collected from various supplier sources. All product claims and specifications are those of the product suppliers, not McKesson Medical-Surgical or its affiliates ("McKesson") and have not been independently verified by McKesson. McKesson is not responsible for errors or omissions in the product information.

The properties of a product may change or be inaccurate following the posting or printing of the product information in the document, either in the print or online version. Caution should be exercised when using or purchasing any products from McKesson's online or print documents by closely examining the product packaging and the labeling prior to use.

Due to product changes, information listed in this document is subject to change without notice. This information is placed solely for your convenience in ordering and McKesson disclaims all responsibility for its completeness and accuracy, whether or not the inaccuracy or incompleteness is due to fault or error by McKesson.

Clinical Disclaimer: Be advised that information contained herein is intended to serve as a useful reference for informational purposes only and is not complete clinical information. This information is intended for use only by competent healthcare professionals exercising judgment in providing care. McKesson Medical-Surgical cannot be held responsible for the continued accuracy of or for any errors or omissions in the information.

All trademarks and registered trademarks are the property of their respective owners.

© 2025 McKesson Medical-Surgical Inc. All rights reserved.  
2025\_McKBWound