



COMPRECARES[®] by SIGVARIS

Compression Garment with Liner Socks

Application and Removal Instructions

1. The Liner:

Pull the liner on like a sock, without bunching or rolling it. (figure 1)



2. The Garment:

Pull on the garment over the liner just below your knee. The black panel should be in the front and the tag should be in the back at the top of the garment. (figure 2)




Pull the straps across the front of your leg. Close the straps without stretching or applying pressure. Avoid leaving any gaps between them or creating overlap. Press the hook ends of the strap lightly to the outside of the garment to secure it. (figure 3)



3. ACCUTAB Placement:

Place an ACCUTAB at the end of each strap. The ACCUTAB should be parallel to the strap with the words closest to the strap end. Then pull the strap to the pressure level line your doctor prescribed. These tabs can remain in place when worn and be used as a guide when you reapply the garment.* (figure 4)

*NOTE: If your calf size changes, please redo steps 1 and 2 again.

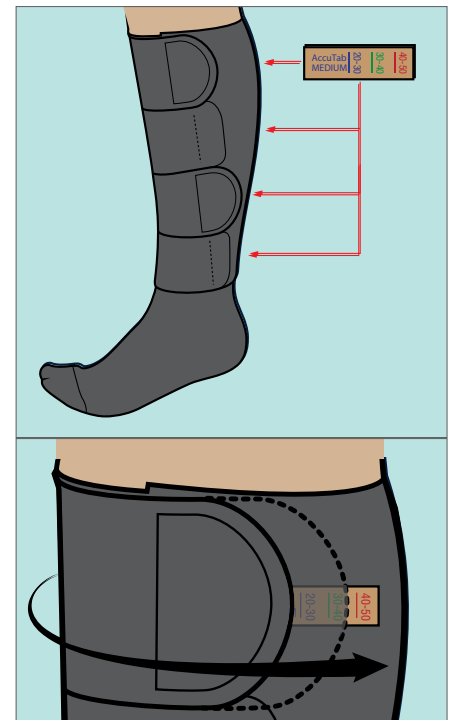

4. Garment Removal:

Fold each strap back on itself to create a loop. Then slide the garment down and off the leg. Now remove the liner sock. (figure 5)



Machine Washable

Garment can be laundered and air dried

For more information, visit medline.com
or contact your Medline Representative.
1-800-MEDLINE (1-800-633-5463)