




COMPRECARES®

Compression Garment Measuring Guide

Get the right fit. COMPRECARES compression garment comes in ten sizes to make sure it provides maximum therapeutic benefits.

How to choose the right size.

- Using a measuring tape with centimeters, measure the leg height from the center of outer ankle bone (point Ø) to the fibular head, as shown on the illustration. Record this measurement in the space provided for point **G**.
- Measure around each part of the leg at the letters shown in the illustration:
 - C1**—30 cm above point Ø
 - C**—25 cm above point Ø
 - B**—15 cm above point Ø
 - A**—5 cm above point Ø
- Use measurements **A** and **C1** to determine the correct sizing for COMPRECARES Transition Liners.

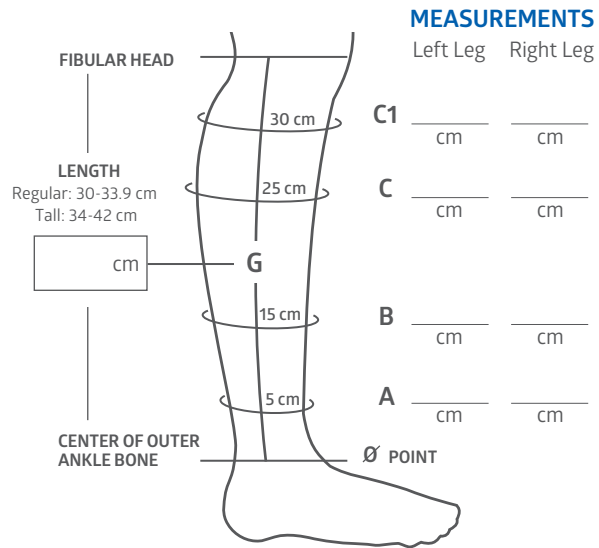

COMPRECARES Compression Garment Sizing Chart

	Small	Medium	Large	X-Large	XX-Large	
C1	29–39 cm	34–44 cm	40–50 cm	46–55 cm	54–64 cm	
C	29–39 cm	34–44 cm	40–50 cm	46–55 cm	54–64 cm	
B	24–34 cm	29–39 cm	34–44 cm	39–49 cm	44–55 cm	
A	19–26 cm	21–30 cm	26–36 cm	31–41 cm	36–46 cm	
G	Regular 30–33.9 cm	MSC1401	MSC1402	MSC1403	MSC1404	MSC1405
	Tall 34–42 cm	MSC1401T	MSC1402T	MSC1403T	MSC1404T	MSC1405T

Compare your measurements to the chart and select the size that matches.*

*If measurements C1, C, B and A align to multiple sizes, select the smaller garment and use a strap extender (**MSC1401SE**) for the larger areas of the leg.

COMPRECARES Transition Liner Sizing Chart

Item No.	Size	A	C1	Women's Shoe Size	Men's Shoe Size
MSC1401LINER	Small	19–26 cm	29–39 cm	<7.5	<6.5
MSC1402LINER	Medium	21–30 cm	34–44 cm	7.5–9.5	6.5–8.5
MSC1403LINER	Large	26–36 cm	40–50 cm	10–12	9–11
MSC1404LINER	X-Large	31–41 cm	46–56 cm	12.5–15	11.5–14
MSC1405LINER	XX-Large	36–46 cm	54–64 cm	12.5–15	11.5–14