

Medigrip™ sizing chart

For compression, use a double layer of Medigrip



Medigrip is indicated for the management of chronic venous insufficiency and edema, for dislocations, sprains, and for use as a retention dressing. Available in Standard and Latex Free (LF).

Ordering information

Size	Description	Standard SKU	Latex Free (LF) SKU
A	Small extremities	MSC9500	MSC9500LF
B	Hands and small arms	MSC9501	MSC9501LF
C	Medium arms, small ankles	MSC9502	MSC9502LF
D	Large arms, medium ankles, small knees	MSC9503	MSC9503LF
E	Large ankles, medium knees, small thighs	MSC9504	MSC9504LF
F	Large knees, medium thighs	MSC9505	MSC9505LF
G	Large thighs	MSC9506	MSC9506LF
J	Small trunks	MSC9507	MSC9507LF
K	Medium/Large trunks	MSC9508	MSC9508LF



Medigrip standard sizing chart for compression

Limb Measurement*				Compression Level for Double layer		
Min Inches	Max Inches	Min cm	Max cm	Low (5-10 mmHg)	Medium (10-20 mmHg)	High (20-30 mmHg)
5	6	13	15	B	-	-
6	7	15	18	B	-	-
7	8	18	20	C	B	-
8	9	20	23	D	B	-
9	10	23	25	E	C	-
10	12	25	30	E	D	-
12	15	30	38	F	E	B
15	18	38	46	G	E	C
18	23	46	58	G	E	D
23	28	58	71	J	F	E
28	38	71	97	K	G	-

Medigrip latex free (LF) sizing chart for compression

Limb Measurement*				Compression Level for Double layer		
Min Inches	Max Inches	Min cm	Max cm	Low (5-10 mmHg)	Medium (10-20 mmHg)	High (20-30 mmHg)
5	5.5	13	14	A	-	-
5.5	6	14	15	B	A	-
6	7	15	18	C	B	A
7	8	18	20	D	B	A
8	9	20	23	E	C	B
9	10	23	25	F	D	B
10	12	25	30	F	E	C
12	15	30	38	G	F	D
15	18	38	46	G	F	E
18	23	46	58	J	G	F
23	28	58	71	J	G	-
28	38	71	97	K	J	-

*For full arm coverage, measure the largest part of the forearm. For full leg or below the knee coverage, measure the widest point of the calf. And, for the hand, measure around metacarpophalangeal joint.