


Digital Floor Scale



Instructions for Use



REF MDR440FD | Product is not made with natural rubber latex.

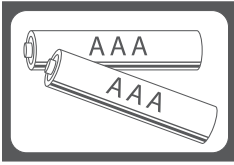


Battery

Remove the isolation sheet from the battery contact before use.

Note: Wait for scale to turn off after initial power on and then step on scale.

Requires 2 x 1.5V AAA batteries (included).

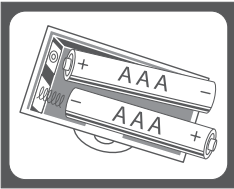

To Replace Batteries

Step 1. Open the battery case cover on the back of the scale.

Step 2. Take out used batteries with the aid of a pointed object.

Step 3. Install new batteries.

Step 4. Batteries should be recycled.




Operation

Step 1: To ensure accuracy, place the scale on a hard flat surface. Make sure there are no objects beneath the scale before weighing.


Step 2. Gently step on the scale, the scale will automatically turn on. Stand evenly on the scale without moving and wait until your weight shown on the display is stable and locked. The display will flash three times when this happens.



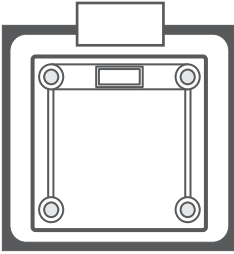

Automatic Switch Off

When you step off the scale, it will automatically switch off. Auto Shut-off occurs approximately 8 seconds after you step off the scale.

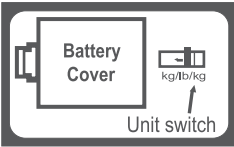

Unit Conversion

To convert between kilogram (kg) and pound (lb) units:

- Step 1. Find the unit switch on the back of the scale.
- Step 2. Pull the unit switch up or down to convert scale unit between kg-lb.



Overload Indication

Scale overload (max. weight capacity is indicated on the back of the scale). Step off to avoid damage.



Low battery Indication

Battery power is low. Install new batteries.



Re-Weigh

Weight reading error. Wait for the scale to turn off and repeat the weighing process.

Warnings!

1. The glass platform will be slippery when wet. **DO NOT USE WHEN SURFACE IS WET!**
2. Stand still during the weighing.
3. **DO NOT** drop the scale.
4. Clean the scale with a damp cloth and avoid getting water inside the scale. **DO NOT** use chemical/abrasive cleaning agents.
5. Keep the scale in a cool and dry place.
6. Always keep the scale in horizontal position.
7. If scale fails to turn on, check whether batteries are installed, or if battery power is low. Insert or replace batteries.
8. If there is an error on display or the scale will not turn off for a long time, remove the batteries for about 3 seconds, then install it again. If you can not resolve the problem, please contact your dealer for help.

Warranty

One year limited warranty against manufacturers defects.



Medline Industries, LP, Three Lakes Drive, Northfield, IL 60093 USA

Medline United States
1-800-MEDLINE (633-5463)
medline.com | info@medline.com

Medline Canada

1-800-268-2848 | medline.ca | canada@medline.com

Medline Mexico

01-800-831-0898 | medlinemexico.com | mexico@medline.com

Follow us    

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this instruction sheet. ©2023 Medline Industries, LP. Medline is a registered trademark of Medline Industries, LP.

MDR440FD | PIC-00380 (Rev: V2 083023)