

# Disposable Stethoscopes

Maintain superior acoustic sound with proper use and care.




## Instructions for use

### Assembly Instructions

Attach the binaural to the yellow plastic tubing by pushing the two ends of the binaural into the two branches of the tubing.

### Adjust the headset

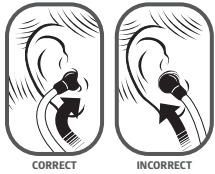

Easily find your most comfortable fit.

- Provides a comfortable, acoustically sealed ear fit
- Ear tubes are shaped to fit right in the ear canal

- Ear tips should point in a forward direction as you insert them into the ear canal



### To reduce spring tension

- Hold each ear tube at the bend near the ear tip and gradually pull apart until fully extended



### To increase spring tension

- Grasp the headset with one hand where the ear tubes enter the tubing
- Squeeze until the tubing on one ear tube touches the other. Repeat as necessary

### For optimal acoustic performance

- Use soft-sealing ear tips provided with your stethoscope
- Ear tip attaches to the ear tube through a unique design
- Ear tips are pushed onto the end of the ear tube and snapped firmly into place
- To remove, pull firmly on the ear tip

**Learn more.** Contact your Medline Representative or call 1-800-MEDLINE for more information.