Elongated Aluminum Commode with Wheels

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General Warnings

To ensure the safe assembly and operation of the product, these instructions must be followed. FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT. If you are unable to understand these warnings, cautions and/or instructions, contact a healthcare professional, dealer or Medline before attempting to use this product.

- DO NOT assemble and use this product without first reading and understanding this user guide, otherwise injury or damage may occur.
- DO NOT use if components are missing or are damaged, contact Medline at 1-800-MEDLINE (633-5463) | 1-800-268-2848.
- DO NOT exceed weight capacity of 300 lb (136 kg).
- DO NOT make any adjustments while using the product.
- DO NOT attempt to use the arm supports to support full body weight.
- DO NOT stand on commode.
- WARNING: Ensure weight is evenly distributed between the arm support when using the commode.
- DO NOT "plop" or suddenly drop body weight onto the commode seat. Ease gently onto the seat when attempting to sit.

Fully Assembled Rolling Aluminum Elongated Commode

Cautions

Users with limited physical strength and/or mobility should be supervised and/or assisted while using the commode. Prior to use, ensure the following:

1. Toilet seat must be in the down position before use. Failure to do so may result in possible injury and/or damage.
2. There are no loose, broken or missing components.
3. All screws, nuts and/or bolts are tightened.
4. When using the backrest, ensure the commode remains balanced. Excessive twisting or turning while seated on the commode may cause loss of balance and/or damage to the product.
5. The wheels and wheel locks are not worn, broken or missing. Replace any missing or damaged parts before use.
6. User evenly distributes weight between the arm supports when transferring in and out of the commode.
7. All four legs touch the floor simultaneously at all times. Always ensure the wheel locks are properly engaged during use.
8. All push buttons are fully protruded and are properly engaged.

9. Armrests are dry. Arms are unlatched and dropped to assist in transfer on or off the commode.
10. The tips are not worn, broken or missing. Replace any missing or damaged tips before use.
11. The wheel locks are properly engaged during use.
12. All parts are secure.

Assembly and use

Frame is shipped partially assembled. If exposed to cold temperatures, allow contents to come to room temperature before completing assembly. Remove contents from box, making sure you have all parts described here.

NOTE: If any of the parts are missing, DO NOT use, contact Medline at 1-800-MEDLINE (633-5463)

| 1-800-268-2848

- | | |
|--------------------------------|--------------------|
| ■ Seat Frame Assembly (1) | ■ M6 Nut (12) |
| ■ Armrest Frame Assembly L (1) | ■ Seat (1) |
| ■ Armrest Frame Assembly R (1) | ■ Lid (1) |
| ■ Backrest Assembly (1) | ■ Bucket Lid (1) |
| ■ M6 Screw (12) | ■ Splash Guard (1) |
| ■ Curved Washer (12) | ■ Bucket (1) |

(Figure 1)

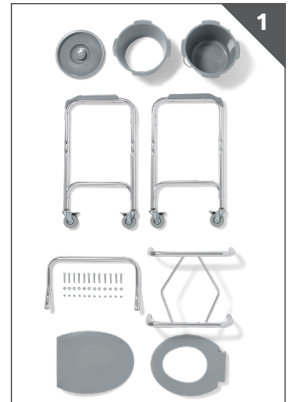
Figure 1

1. Place Seat Frame Assembly between the Armrest Frame Assembly L and Armrest Frame Assembly R. Insert one M6 screw through the frame and secure with a curved washer and M6 nut. **(Figure 2A)** Repeat until the Seat Frame Assembly is fully secured onto the Armrest Frame Assemblies; a total of 8 screws, 8 curved washers and 8 nuts will be utilized. **(Figure 2B)**

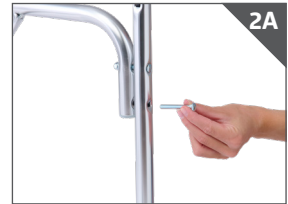
Figure 2

Figure 2

2. Place the Backrest Assembly behind the Armrest Frame Assemblies. Insert one M6 screw through the Armrest Frame Assembly and secure with a curved washer and M6 nut. Repeat until the Backrest Assembly is fully secured onto the Armrest Frame Assemblies; a total of 4 screws, 4 curved washers and 4 nuts will be utilized. **(Figure 3)**

Figure 3

3. Place the Bucket, Splashguard and Bucket Lid on the Seat Frame Assembly. **(Figure 4)**
4. Place the Seat and Lid onto the Seat Frame Assembly. Ensure each clip snaps into place. **(Figure 5)**
5. Before use, lock the wheels by pushing down on the brake tab on each wheel. Pull up on the brake tab to release when needed. **(Figure 6)**

Figure 4

Figure 5

Figure 6

Maintenance

Periodic visual inspection of this product is recommended to ensure that all parts and hardware are secure, that components are in good working order and not cracked, broken or fractured, and that there are no obstructions or impediments to normal operation.

- If any part or mechanism appears to be unsafe or damaged, DO NOT use the product.
- If you experience problems, please contact Medline at 1-800-MEDLINE (633-5463) | 1-800-268-2848.

Cleaning

1. Clean with regular household cleaning products. Use a non-abrasive cleaner with warm water. Wipe dry after cleaning.
2. DO NOT use abrasive powders, cleaners or scouring pads.

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Limited Warranty

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This limited warranty is an expression of our confidence in the materials and workmanship of our products. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as tip accessories, casters and grips, which are subject to normal wear and replacement.

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Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2025 Medline Industries, LP. Medline is a registered trademark of Medline Industries, LP.

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Medline Industries, LP, Three Lakes Drive, Northfield, IL 60093 USA

Medline United States
1-800-MEDLINE (633-5463)
medline.com | info@medline.com

Medline Canada

1-800-268-2848 | medline.ca | canada@medline.com

Medline México

01-800-831-0898 | medlinemexico.com | mexico@medline.com

©2025 Medline Industries, LP. All rights reserved/tous droits réservés/todos los derechos reservados. Medline is a registered trademark of/est une marque déposée de/es una marca comercial registrada de Medline Industries, LP, Three Lakes Drive, Northfield, IL 60093 USA.

G1-501LWX1 (QIM-00244) | (Rev: V1 111325)