

## Elastomeric infusion pump

**Filling Guidelines:** These filling guidelines reflect delivery times for partial or overflow volumes. To find partial or overflow volume for a particular model, locate the desired delivery time in the far left column and slide to the right to the column referencing the item code to find fill volume.

Please note the delivery times for partial or overflow volume are approximate values.

### Short Duration Pumps

Item Code	1250521	1250522	1250523	1250524	1250525	1250526	1250527	1250528	1250529
Catalog Number	16-EP5050	16-EP10050	16-EP100100	16-EP400100	16-EP250175	16-EP100200	16-EP400200	16-EP250250	16-EP500250
Nominal Flow Rate	50mL/H	50mL/H	100mL/H	100mL/H	175mL/H	200mL/H	200mL/H	250mL/H	250mL/H
Nominal Fill Volume	50mL	100mL	100mL	400mL	250mL	100mL	400mL	250mL	500mL
Min Fill Volume	40mL	80mL	80mL	240mL	200mL	80mL	240mL	200mL	400mL
Max Fill Volume	85mL	150mL	150mL	400mL	250mL	150mL	400mL	250mL	500mL
Residual Volume	2mL	2mL	2mL	5mL	3.5mL	2mL	5mL	3.5mL	5mL

### Approximate Delivery Time

Hours: Min									
0:15						50mL			
0:30			50mL			100mL		150mL	
0:45			75mL		140mL			200mL	
1:00	50mL		100mL		175mL		200mL	250mL	
1:15	60mL		120mL		220mL		250mL	300mL	360mL
1:30		75mL			250mL		300mL		400mL
1:45					300mL		350mL		475mL
2:00		100mL					400mL		500mL
2:15		110mL					450mL		550mL
2:30							500mL		
2:45				275mL					
3:00				300mL					
3:30				360mL					
4:00				400mL					
4:30				450mL					
5:00				500mL					
5:30				550mL					

### Long Duration Pumps

Item Code	1250530	1250532	1250531
Catalog Number	16-EP1255	16-EP2705	16-EP27010
Nominal Flow Rate	5mL/H	5mL/H	10mL/H
Nominal Fill Volume	125mL	270mL	270mL
Min Fill Volume	90mL	250mL	230ml
Max Fill Volume	150mL	360mL	360mL
Residual Volume	1.2mL	3mL	3mL

### Approximate Delivery Time

Hours	Days		
18		90mL	
20		100mL	230mL
25	1.00	125mL	250mL
27	1.00		270mL
30	1.25	150mL	300mL
36	1.50		330mL
40	1.50		
46	2.00	240mL	
50	2.00	250mL	
54	2.25	270mL	
72	3.00	330mL	

The product information contained in this document, including the product images and additional product materials, was collected from various supplier sources. All product claims and specifications are those of the product suppliers and have not been independently verified by McKesson Medical-Surgical or its affiliates ("McKesson"). McKesson is not responsible for errors or omissions in the product information.

The properties of a product may change or be inaccurate following the posting or printing of the product information in the document, either in the print or online version. Caution should be exercised when using or purchasing any products from McKesson's online or print documents by closely examining the product packaging and the labeling prior to use.

Due to product changes, information listed in this document is subject to change without notice. This information is placed solely for your convenience in ordering and McKesson disclaims all responsibility for its completeness and accuracy, whether or not the inaccuracy or incompleteness is due to fault or error by McKesson.

© 2025 McKesson Medical-Surgical Inc.