Foldable Toilet Safety Rail

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Do not install this product without first reading and understanding this instruction manual.

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Failure to follow instructions may result in serious injuries.

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General Warnings

To ensure the safe assembly and operation of this product, these instructions must be followed. **FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT.** If you are unable to understand these warnings, cautions and/or instructions, contact a healthcare professional, dealer or Medline before attempting to use this product.

- **DO NOT** assemble and use this product without first reading and understanding this user's guide, otherwise injury or damage may occur.
- **DO NOT** use if components are missing or are damaged. Contact Medline at 1-800-MEDLINE (633-5463) | 1-800-268-2848.
- **DO NOT** exceed weight capacity of 250 lb (113 kg).
- **DO NOT** make any adjustments while using the product
- **DO NOT** stand on the product.
- **DO NOT** attempt to use the arms to support full body weight
- **DO NOT** use product for anything other than its intended use.
- **DO NOT** use product if wobbly, unstable, or not level.
- **DO NOT** use product on uneven surfaces.

Cautions

Users with limited physical strength and/or mobility should be supervised and/or assisted while using the product. Prior to use, ensure the following:

- All push buttons are fully protruded and are properly engaged.
- There are no loose, broken or missing components.
- All parts are secure.
- Ensure both rails are at the same width adjustment position before use.
- Excessive twisting or turning while using the safety rails may cause a loss of balance and/or damage to the product.
- Both legs touch the floor simultaneously at all times.
- User evenly distributes weight between the arm supports when transferring in and out of the safety rails.
- The tips are not worn, broken or missing. Replace any missing or damaged tips before use.
- Ensure all legs are at the same height adjustment level before use.

Assembly Instructions

Before beginning assembly, identify all parts (Figure 1)

- 1 - Bracket
- 2 - Toilet rails
- 2 - Thumbscrews
- 2 - Legs with tips

Figure 1

1. Remove existing toilet seat assembly, including bolts and nuts.
2. Place toilet safety rail bracket between toilet fixture and seat assembly with rounded part of bracket closest to the back of toilet (Figure 2).
3. Replace toilet seat assembly and reinsert the original mounting bolts through front holes of the bracket and toilet fixture (Figure 3). Tighten the nuts.
4. Side rail arms will connect to each side of the bracket. To position the arms, loosen the locking collar and press the push button. Rotate the tubing in the direction in which the tubing end will be inserted into the rail bracket until the push button clicks. Hand tighten the locking collar. (Figure 4).

NOTE: Arms **ARE NOT** left and right specific. Follow step 4 to change the tubing direction if both arms are in the same direction.

Figure 2

Figure 3

Figure 4

5. Slide each side rail into desired width position. Insert black adjustment hand knobs into the bracket and tighten to secure into place (Figure 5).
6. Adjust foot pieces to desired height (Figure 6). Make sure both foot pieces are fully engaged and adjusted at equal heights.
7. Make sure mounting hardware is tightened and all snap buttons are fully secure before use.

Figure 5

Figure 6

Product fully assembled

Figure 7

Maintenance

Periodic visual inspection of this product is recommended to ensure that all parts and hardware are secure, that components are in good working order and not cracked, broken or fractured, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or damaged, do not use the product. If you experience problems, please contact Medline at 1-800-MEDLINE (633-5463) | 1-800-268-2848.

Cleaning

1. Clean with a non-abrasive cleaner or household cleaner with warm water.
2. Rinse with warm water and wipe dry after cleaning.
3. **DO NOT** use abrasive powders, cleaners, cloths, or scouring pads.

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Limited Warranty

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as tip accessories, casters and grips, which are subject to normal wear and replacement.

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Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual.

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©2024 Medline Industries, LP. All rights reserved/tous droits réservés/todos los derechos reservados. Medline is a registered trademark of/est une marque déposée de/es una marca comercial registrada de Medline Industries, LP, Three Lakes Drive, Northfield, IL 60093 USA. **1-800-MEDLINE**