GUARDIAN[®]

Family-owned. Trusted by
hospitals for over 50 years.

FOREARM CRUTCHES USER INSTRUCTIONS & WARRANTY

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REF G05163C

WARNING

For safety, these precautions must be taken when using this product.

Before each use make sure:

- You are correctly fitted.
- All spring buttons are engaged and all screws are tightened.
- Tips are in good condition. Replace tips immediately when worn.
- Hand grips are attached securely and do not slip.
- Nothing is loose or uneven and there are no rattles or cracks in the metal, plastic, etc. or any other irregularities.
- Product is used only with the manufacturer's parts and accessories— have seller install.
- User does not have any handicaps, disabilities or is on medication, etc. that a doctor, therapist, and/or seller would deem to interfere with the safe usage of the product.

Do not exceed the maximum weight capacity of your crutch. Listed below please find the model you own to ensure that you are within the weight capacity. Failure to comply may result in injury.

Model #	Weight Capacity
G05160	300 lbs. (136 kg.)
G05161	300 lbs. (136 kg.)
G05162Y	300 lbs. (136 kg.)
G05163C	200 lbs. (90 kg.)

Use only as instructed by your healthcare provider.

Always use extreme care when using a forearm crutch. Avoid all potential safety hazards including slippery, uneven or soft surfaces and objects in path.

Failure to follow the above instructions may result in serious injury.

Proper Height

When using forearm crutches, ensure that both crutches are adjusted to an equal height and that all spring lock detent buttons are properly seated.

- Adjust so that while the user stands as upright as possible, the handgrips are at the level of the user's hip joint, or break of the wrist. The user's elbows should bend 30°, and the wrists should be straight as the user grasps the hand grips.
- For cuff height, adjust so that the cuff is 1" to 2" below the bend on the elbow.

Proper Usage

- User's weight should be carried on the handgrips with the wrists straight.
- User should consult physician or therapist to be shown proper walking gait.
- Do not subject crutch to sudden impact or jolt.

Grip Cleaning Instructions

Clean the grips with a mild soap, detergent or household cleaner. **Do not** use an organic solvent (such as acetone, MEK, lacquer thinner, lighter fluid, dry cleaning fluid, gasoline or turpentine) that may dissolve the plastic and cause the grip to loosen.

Tips on this product contain dry natural rubber.

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LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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GUARDIAN®

Medline Industries, LP,
Three Lakes Drive, Northfield, IL 60093

Medline United States
1-800-MEDLINE (633-5463)
www.medline.com | info@medline.com

Medline Canada
1-800-396-6996
www.medline.ca | Canada@medline.com

Medline Mexico
01-800-831-0898
www.medlinemexico.com | mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2018 Medline Industries, LP. Guardian is a registered trademark of Medline Industries, LP.

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