# Full Body Slings

## Instructions for use

### Description

Full Body slings are made of durable materials and are ideal for patient transferring and toileting activities.

### Model SKUs

MDSMR110	MDSMR114	MDSMR112
MDSMR111	MDSMR115	MDSMR113
MDSMR140	MDSMR116	MDSMR117
	MDSMR141	MDS88200DSL

### Cautions

- Do not exceed the maximum weight capacity of sling or patient lift.
- Always confirm the weight capacity of the sling against the patient's weight and ensure proper placement before performing a lift.
- Always inspect slings prior to each use. Signs of rips, tears, or frays indicate sling wear which is unsafe and could result in injury. Signs of color fading, bleached areas, or permanent wrinkles on the straps indicate improper laundering which is unsafe and could result in injury. Any slings with signs of wear or improper laundering should be immediately removed from use.
- Always confirm compatibility between the connection style of the patient sling and the patient lift before use. Medline loop-style 6-point cradles are compatible with these slings. If other lift cradles are being used, the user should confirm compatibility between the hoist and the sling. Reach out to your Medline Representative with questions.
- Do not remove sling labels. If sling labels are removed or no longer legible, sling must be immediately removed from use.
- Never launder a disposable patient sling.
- Always follow your facility's Safe Patient Handling policy regarding how many staff members are needed to perform a patient lift.
- Always refer to the manufacturer's manual for operation of lift equipment.

### Instructions


#### Sizing

Always refer to weight capacity of the sling when selecting a size. Use clinical judgement for body shape and patient comfort when selecting between sizes.

**Note:** For amputees, use an amputee sling or repositioning sling, and use the discretion of your clinical professionals.

#### Positioning

With the product label outwards, position the full body sling underneath the patient with the top positioned just above the head, and the bottom just behind the patient's knees. Make sure the patient is centered on the sling.

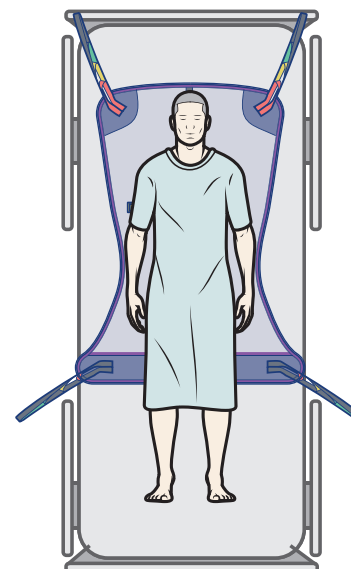### Application

Scan QR code to view application video, or go to <https://vimeo.com/356271549>

1. Ensure the patient is centered on the sling and the head is supported.
2. Lock the wheels on all equipment the patient is on and/or will be transferred to.
3. Starting at the head, make sure each side has the same colored loop attached to the left and right side of the cradle before lifting.
  - a. Use different loop lengths on the straps for the desired patient positioning:
    - Shorter straps towards the head, and longer straps towards legs, will create a seated position.
    - Longer towards the head, and shorter towards the legs, will create a reclined position.
4. When all straps are attached to the patient lift cradle, do a final visual check that all loop lengths match from side-to-side.
5. When performing the lift, reassure and calm the patient. Patient should not hold any part of the lift or straps during transfer.
6. Lift the patient and proceed with repositioning or transfer.
7. After lift or transfer, remove all loops of the patient sling from the lift and remove sling from behind patient.

### Reusable Slings

- Reusable slings are recommended for use with the same patient unless washed before using for a different patient.
- Frequency of laundering should follow facility guidelines, or when the sling is soiled. Refer to the sling's tag for laundering instructions and follow all wash instructions.

### Disposable Slings

- Disposable slings are for single-patient use and may only be re-used with the same patient.
- Disposable slings should be discarded when soiled or when patient is discharged.

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