GUARDIAN[®]

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GENERAL WARNINGS

Be sure to read all instructions and safety information before using this chair. If you have questions about using this product safely or transferring to and from the bath, please consult with a healthcare professional. FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT.

Users with limited physical strength and/or mobility should be supervised and/or assisted while using the commode.

- DO NOT assemble and use this product without first reading and understanding this user guide, otherwise injury and/or damage may occur.
- DO NOT use if components are missing or damaged. Contact Medline at 1-800-MEDLINE (633-5463) |1-800-268-2868.
- DO NOT exceed the weight capacity of 275 lbs (124 kg).
- DO NOT make any adjustments while using the product.
- DO NOT attempt to use the arms to support full body weight.
- DO NOT stand on the frame of the product.
- DO NOT stand or apply excess pressure on the footrests. When getting in or out of the chair, ensure the footplates are in the UP position.
- DO NOT attempt to lift the product by the armrests or any detachable component.
- DO NOT place hands or fingers near any moving part.
- DO NOT transfer in or out of the chair on your own unless you have sufficient upper body strength, balance and agility.
- DO NOT lean forward out of the chair any farther than the length of the armrests.
- DO NOT attempt to reach for objects if doing so means you move forward in the seat.
- DO NOT shift your weight or sitting position toward the direction you are reaching as the chair may tip over.
- DO NOT attempt to lean over the top of the backrest cushion to reach for objects.
- DO NOT tip the chair without assistance.
- DO NOT lean the chair back as this may cause the chair to tip over. If trying to reach objects, reach back only as far as your arm will extend without altering your seating position.
- DO NOT modify the chair. Examples; drilling holes, grinding, welding parts, using non-authorized/non-approved parts. Such modifications may change the chair's structure and center of balance which may increase the risk of serious injury to the user and void the warranty.
- DO NOT use this chair in an outdoor setting.

ASSEMBLY

READ ALL INSTRUCTIONS CAREFULLY BEFORE USING THIS CHAIR.

1. Remove contents from box, making sure you have all parts listed.
2. Mounting the backrest assembly: Unscrew both locking knobs from the backrest frame assembly. Insert the backrest frame assembly into the frame and secure with the locking knobs. Lower the armrests into position for use. **(Figure 1)**
3. Mounting the seat: Place the seat onto the seat frame. Firmly press down until each clip snaps into place. Note: Make sure the recessed portion is located at the rear. **(Figure 2)**
4. Mounting the footrests: Insert the footrests into the frame tubes and swivel the footrest forward. The footrest should click into place. **(Figure 3)**

- | | |
|-----------------|----------------------|
| 1. Push handle | 5. Footrests |
| 2. Back cushion | 6. Steering castors |
| 3. Armrest | 7. Locking knobs (2) |
| 4. Seat | |

ADJUSTMENTS

Adjusting the castors

1. Remove the front castor bolt using the allen key and tool provided.
2. Adjust the castor to the desired height.
3. Lock the castor in place using the bolt and nut. Repeat until all four castors have been adjusted to the same preferred height. **(Figure 4)**

Adjusting the footrests

1. Unscrew the bolt and nut from the rear side of the leg rest frame.
2. Set to desired height and screw on bolt and nut.
3. Repeat until both footrests have been adjusted to the desired height. Make sure the screws are fully tightened. **(Figure 5)**

Figure 1

Figure 2

Figure 3

Figure 4

Figure 5

INSTRUCTIONS FOR USE

Locking Castors

1. Step down on the tab to lock the castor in place.
2. To release, lift up from underneath the tab. **(Figure 6)**

Commode Usage

1. Always lock the locking castors prior to using the chair.
2. Remove the lid from the bucket and push the bucket into place from the rear side of the chair.
3. Remove the padded seat lid cutout.
4. After use, pull bucket out from the rear side. Empty the contents and clean immediately.
5. Replace the padded seat lid cutout.

Over the Toilet Usage

1. Remove the bucket and lid from the rear side.
2. Remove the padded seat lid cutout.
3. Raise the toilet seat and lid.
4. Roll the commode over the toilet and lock the castors.

Shower Chair Usage

1. Pull the bucket and lid out from the rear side.
2. Remove padded seat lid cutout.
3. When using the chair in the shower, make sure the locking castors are engaged.

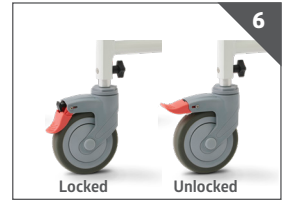
Figure 6

SAFETY INSTRUCTIONS

Reaching/leaning/bending forward or sideways

- Position the chair as close as possible to the object.
- When reaching forward make sure the front castors are pointing in a forward position **(Figure 7)**. This extends the wheel base and will help stabilize the chair.
- When reaching sideways, position the castors to provide the greatest stability side to side. **(Figure 8)**
- Make sure the locking castors are engaged before sitting down, standing up, reaching, or using as a commode.
- DO NOT attempt to reach for objects if doing so means you must move forward in the seat. DO NOT shift your weight toward the direction you are reaching.
- DO NOT attempt to reach objects if you have to pick them up from the floor by reaching down between your knees.
- DO NOT shift your weight or seat position toward the direction you are reaching.

Figure 7

Figure 8

SAFETY INSTRUCTIONS (CONTINUED)

Reaching/leaning backwards

- Position the chair as close as possible to the object. Make sure the front casters are pointing in a forward position. This extends the wheel base and will help stabilize the chair. **(Figure 9)**
- DO NOT lean back over the top of the back cushion. Reach back only as far as your arm will extend without changing your position in the seat. **(Figure 9)**

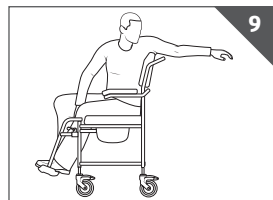
Figure 9

Transferring into and out of the chair

Transferring to and from the chair is a significant task that requires balance, agility and upper body strength. Practice with assistance prior to attempting transfers.

- Position the chair as close as possible to the user.
- Make sure all locking castors are engaged.
- Make sure the armrests and footrests are moved out of the way. **(Figure 10, 11)**

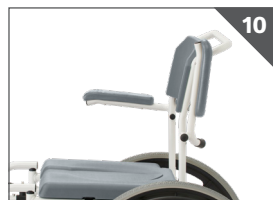
Figure 10

Obstacles

Obstacles such as floor threshold strips, uneven floors, cords or items on the floor, or cracks in pavement can damage the chair and may cause loss of control or tipping over. Scan your path and avoid any areas that may cause instability.

Figure 11

SAFETY INSPECTION AND MAINTENANCE

Prior to use, make sure:

- There are no loose, broken or missing components.
- All screws, nuts and/or bolts are tightened.
- The castors and castor locks are not worn, broken or missing. Replace any missing or damaged parts before use.
- The locking castors are always properly engaged during use.
- User evenly distributes weight between the arm supports when transferring in and out of the commode.

CLEANING

Clean after each use.

- Clean with regular household cleaning products. Use a non-abrasive cleanser with warm water. Wipe dry after cleaning.
- DO NOT use abrasive powders, cleansers or scouring pads.

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Limited Warranty

Medline guarantees for the original purchaser, that the frame of the chair will be free from defects in materials for the life of the chair.

Medline also guarantees the parts on the chair to be free from defect in materials and workmanship for a period of one (1) year from the date of purchase by the original purchaser.

This warranty does not cover device failure due to owner misuse, negligence or normal wear and tear. All warranty claims must include the product in questions unless otherwise authorized by Medline. For more information about the product or warranty, call Medline at 1-800-MEDLINE (633-5463) | 1-800-268-2868.

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Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual.

©2021 Medline Industries, LP. Guardian and Medline are registered trademarks of Medline Industries, LP.

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