# GUARDIAN<sup>®</sup>

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**Adjustable Height | Locking  
Casters**

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REF G1-506DWX1

## General Warnings

To ensure the safe assembly and operation of the product, these instructions must be followed. FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT.

If you are unable to understand these warnings, cautions and/or instructions, contact a healthcare professional, dealer or Medline before attempting to use this product.

- DO NOT assemble and use this product without first reading and understanding this user guide, otherwise injury or damage may occur.
- DO NOT use if components are missing or are damaged, contact Medline at 1-800-MEDLINE | 1-800-268-2848.
- DO NOT exceed weight capacity of 300 lbs (136 kg).
- DO NOT make any adjustments while using the product.
- DO NOT attempt to use the arms to support full body weight.
- DO NOT stand on the commode.

**Fully Assembled Drop-arm Commode**

## Cautions

Users with limited physical strength and/or mobility should be supervised and/or assisted while using the product. Prior to use, ensure the following:

1. Toilet seat must be in the down position before use. Failure to do so may result in possible injury or damage.
2. All push buttons are fully protruded and are properly engaged.
3. There are no loose, broken or missing components and ensure all parts are secure.
4. When using the backrest, ensure the commode remains balanced. Excessive twisting or turning while seated on the commode may cause a loss of balance and/or damage to the product.
5. The wheels and wheel locks are not worn, broken or missing. Replace any missing or damaged parts before use.
6. All four legs touch the floor simultaneously at all times. Always ensure the wheel locks are properly engaged during use.
7. User evenly distributes weight between the arm supports when transferring in and out of the commode.
8. Arms should be unlatched and dropped to assist in transfer on or off the device.

## Assembly and use

Frame is shipped partially assembled. If exposed to cold temperatures, allow contents

to come to room temperature before completing assembly. Remove contents from box, making sure you have all parts described here:

- Seat Frame with left and right drop arms assembled (1)
- Backrest Assembly (1)
- Splash Guard (1)
- Toilet Seat (1)
- Toilet Lid (1)
- Bucket Lid (1)
- Bucket (1)

1. Place the arm assembly (L) and arm assembly (R) into seat frame by pushing in pin and lifting arm up. **(Figure 1)**
2. Place backrest assembly between the arm assemblies.
3. Snap clips on toilet seat into place on the seat frame. Lower the splash guard bucket, bucket and bucket lid into seat frame assembly. **(Figure 2)**
4. Lock wheels before use by pushing down brake tab on each wheel. Pull up on tab to release brakes to roll the commode when needed. **(Figure 3)**

## Adjusting leg height

Before using the commode, make sure all legs are set to the same preferred height.

1. To adjust leg height, depress pushbutton and slide each leg to desired height by aligning with proper holes and letting the pushbutton snap into the correct hole to lock leg in place. **(Figure 4)**
2. Make sure legs are locked into place by pushing down and pulling up on each leg commode.

## Maintenance

Periodic visual inspection of this commode is recommended to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, or loose, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or damaged, do not use the product. If you experience problems, please contact Medline at 1-800-MEDLINE | 1-800-268-2848.

## Cleaning

1. Clean with regular household cleaning products. Use a non-abrasive cleaner with warm water. Wipe dry after cleaning.
2. DO NOT use abrasive powders, cleaners or scouring pads.

Figure 1

Figure 2

Figure 3

Figure 4

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## Limited Warranty

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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Medline Industries, LP, Three Lakes Drive, Northfield, IL 60093

Medline United States  
1-800-MEDLINE (633-5463)  
medline.com | info@medline.com

Medline Canada

1-800-268-2848 | medline.ca | canada@medline.com

Medline Mexico

01-800-831-0898 | medlinemexico.com | mexico@medline.com

Follow us

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2021 Medline Industries, LP. Guardian and Medline are registered trademarks of Medline Industries, LP.

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