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General Warnings

To ensure the proper assembly and operation of this product, these instructions must be followed. **FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT.** If you are unable to understand these warnings, cautions and/or instructions, contact a healthcare professional, dealer or Medline before attempting to use this product.

- **DO NOT** assemble and use this product without first reading and understanding this user's guide, otherwise injury or damage may occur.
- **DO NOT** use if components are missing or are damaged. Contact Medline at **1-800-MEDLINE (633-5463) | 1-800-268-2848**.
- **DO NOT** exceed weight capacity of **400 lb (181 kg)**.
- **DO NOT** make any adjustments while using the product.
- **DO NOT** stand on the product.
- **DO NOT** excessively twist or turn while seated on the chair as this may cause a loss of balance and/or damage to the product.
- **DO NOT** attempt to use the arms to support full body weight.
- **DO NOT** attempt to use the backrest to support full body weight.
- **DO NOT** use product if wobbly, unstable, or not level.
- **DO NOT** use product on uneven surfaces.
- **DO NOT** "plop" or suddenly drop body weight onto the shower chair seat. Ease gently onto the seat when attempting to sit.
- **WARNING:** Shower chair can be slippery when wet. Use caution when attempting to sit or transfer.
- **WARNING:** Ensure weight is evenly distributed between the arm supports, if applicable, when using the shower chair.

Cautions

Users with limited physical strength and/or mobility should be supervised and/or assisted while using the product. Prior to use, ensure the following:

1. Ensure chair is completely unfolded before use. Failure to do so may result in possible injury or damage.
2. All push buttons are fully protruded and are properly engaged.
3. There are no loose, broken or missing components.
4. All parts are secure.
5. Ensure the chair remains balanced.
6. All four legs touch the floor simultaneously at all times.
7. User evenly distributes weight between the arm supports when transferring in and out of the chair.
8. The tips are not worn, broken or missing. Replace any missing or damaged tips before use.
9. Ensure all legs are at the same height adjustment level before use.

Assembly

If exposed to cold temperatures, allow contents to come to room temperature before completing assembly. Remove contents from box, ensuring all parts listed are included:

- Backrest (1 pc)
- Backrest support tubes (2 pcs)
- Seat (1 pc)
- Seat support tubes (2 pcs)
- Leg tubes (4 pcs)
- Tips (4 pcs)
- Short Thumbscrews (5pcs)
- Long Thumbscrews (4pcs)
- Washers (9pcs)

Use

1. Place seat upside down on a flat surface.
2. Place the leg section marked with a "B" into the groove of the bottom of the seat marked "B". Place the leg section marked with an "A" on top of the "B" tube and into the groove of the seat marked "A". Insert the shorter thumb screw with curved washer through the middle hole but do not fully tighten the screw. **(Figure 1)**
3. Next, assemble the backrest by taking the "L" shaped backrest supports and securing them to the backrest using the shorter thumb screws with curved washer as shown. **(Figure 2)**
4. Attach backrest to the seat by aligning holes on "L" shaped frame supports with the holes on the leg section. Secure the legs of the frame marked "B" in place by using the longer thumbs screws with curved washer. Make sure you attach backrest to "rear" of the seat as marked on the bottom. **(Figure 3)** **NOTE:** Seat depth can be adjusted by choosing one of the two holes on the frame supports.

Figure 1

Figure 2

Figure 3

5. Secure the legs of the frame marked "A" in place with the remaining two longer thumb screws with curved washers.
6. Fully tighten the shorter middle screw with curved washers and check to make sure all screws are fully tightened.
7. Insert legs into leg frame by depressing pushbutton, sliding legs over the receptacle and setting to desired height. Make sure all the legs are set to the same height. (**Figure 4**)

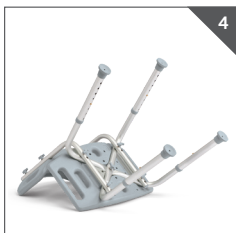
Figure 4

Leg Height Adjustment

1. Before use, adjust the height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes (**Figure 5**).
2. Ensure that the legs are locked into place by pushing down and pulling up on each leg.

Figure 5

IMPORTANT NOTE: Ensure the user's legs do not dangle while sitting on the bench as this could potentially cause a loss of circulation in the legs.

Maintenance

Periodic visual inspection of this product is recommended to ensure that all parts and hardware are secure, that components are in good working order and not cracked, broken or fractured, and that there are no obstructions or impediments to normal operation. Drain any water that has accumulated in the seat or backrest. If any part or mechanism appears to be unsafe or damaged, do not use the product. If you experience problems, please contact Medline at **1-800-MEDLINE (633-5463) | 1-800-268-2848**.

Cleaning

1. Clean with a non-abrasive cleaner or household cleaner with warm water.
2. Rinse with warm water and wipe dry after cleaning.
3. **DO NOT** use abrasive powders, cleaners, cloths, or scouring pads.

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Limited Warranty

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This limited warranty is an expression of our confidence in the materials and workmanship of our products. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as tip accessories, casters and grips, which are subject to normal wear and replacement.

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Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2025 Medline Industries, LP. Medline is a registered trademark of Medline Industries, LP.

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