GUARDIAN[®]

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General Warnings

To ensure the safe assembly and operation of this product, these instructions must be followed. **FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT.** If you are unable to understand these warnings, cautions and/or instructions, contact a healthcare professional, dealer or Medline before attempting to use this product.

- DO NOT assemble and use this product without first reading and understanding this user's guide, otherwise injury or damage may occur.
- DO NOT use if components are missing or are damaged. Contact Medline at 1-800-MEDLINE (633-5463) | 1-800-268-2848.
- DO NOT exceed weight capacity of 300 lb (136 kg).
- DO NOT make any adjustments while using the product.

Cautions

- Users with limited physical strength and/or mobility should be supervised and/or assisted while using the product.
- Ensure that suitable attachment hardware is used and is compatible with wall construction materials.

Prior to use, ensure the following:

- There are no loose, broken or missing components and ensure all parts are secure.
- All hardware is securely tightened.

Components

- 1 - Grab Bar
- 6 - Stainless steel screws

Installation Instructions*

Tools Required for Grab Bar Installation:

- 3/8" or 1/4" electric drill
 - Extension cord
 - 5/32" Drill bit for drilling pilot hole in wood stud
 - 1/4" Drill bit for clearance hole with tile if appropriate
 - Sharp center punch
 - Level
 - Soft grease pencil
 - Hammer
 - Screwdriver
 - Stud finder
 - Toggle bolts (if not able to install into wall stud)
1. Locate wall studs. Slightly loosen rear screws on back of flanges. Rotate flanges until desired position is determined.
 2. Position grab bar at angle which best accommodates user's needs and also allows at

least one end to be mounted into the wall stud. Mark hole locations with soft grease pencil. When locating vertical or horizontal positions, use your level. If one end cannot be mounted into a wall stud, you must use a butterfly toggle fastener (not supplied) - tighten rear flange screws.

3. Mounting directly into wood studs:

- Drill 5/32" pilot hole. Using the screws provided, insert the screw until it is flushed with the grab bar.

Mounting through tile: (Caution: Avoid drilling into water lines)

- At the mark for your hole, take your sharp center punch and gently chip the glaze away. The chipped area need be no larger than the hole you are going to drill.
- Once the glaze is chipped, take the 1/4" tile drill bit and drill into the wall. If you strike wood, stop drilling. Switch to the 5/32" drill bit and drill 1" into the wood. Insert the appropriate screw type ensuring it is flushed with the grab bar. If you do not penetrate wood, see instructions for mounting with a toggle bolt.

MOUNTING USING A TOGGLE FASTENER (not supplied)

- Drill hole using drill bit large enough to receive the toggle fastener.
 - Insert the toggle bolt through the flange of the grab bar, spin butterfly tip on just enough to be secure, then feed bolt through large hole drilled.
 - Once the butterfly is inserted it will fly open on the inside of the wall. Pull out on the grab bar to make sure the butterfly has opened properly (reference illustration). Drive screw down until you feel the bar snug to the wall using screwdriver.
4. Test bar for tightness by applying full body force to the bar to make sure that it is solidly anchored.

*Manufacturer bears no responsibility for improper installation.

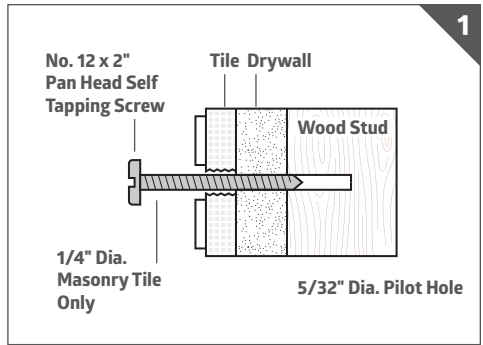
Figure 1

User Instructions

Firmly grasp bar. Pull up firmly on the bar to move into a comfortable position in the bathtub, shower or toilet area. Afterward, grasp the bar firmly to transfer back into your original position.

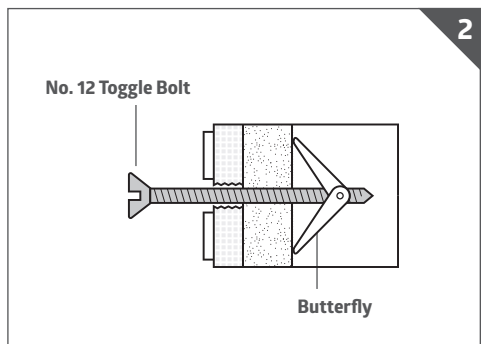
Figure 2

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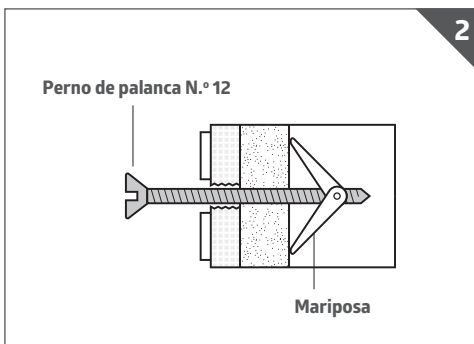
Figura 2

Limited Warranty

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair, or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as tip accessories, casters and grips, which are subject to normal wear and replacement.

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Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual.

©2022 Medline Industries, LP. Guardian and Medline are registered trademarks of Medline Industries, LP.

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