GUARDIAN[®]

Family-owned. Trusted by hospitals for over 50 years.

WALKER SEAT USER INSTRUCTIONS & WARRANTY

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REF G07872

LIFETIME
LIMITED
WARRANTY

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Save these instructions for future reference.

Safety Instructions:

To ensure your safety in using the Medline Walker Seat, the following safety information and all instructions must be followed.

Warning

Failure to follow these instructions could result in injury.

Warning

If your walker has front leg wheels, the rear legs must be equipped with rubber tips, glide brakes, or wheel brakes.

Warning

Use assistance when sitting on the seat to prevent the walker from moving away from the user.

Warning

Before sitting down make sure of the following:

- Push down to engage the rear rubber tips to the floor, retracting the brakes. Failure to do so may cause your walker to move resulting in a fall.
- Make sure that the seat is completely attached to the support bar.
- Make sure the seat supports are unfolded and completely inserted into the main tube.
- Make sure the seat supports are firmly resting on the walker cross braces.
- Be sure the walker is stable.

Warning

DO NOT sit on the seat when on a sloped or uneven surface.

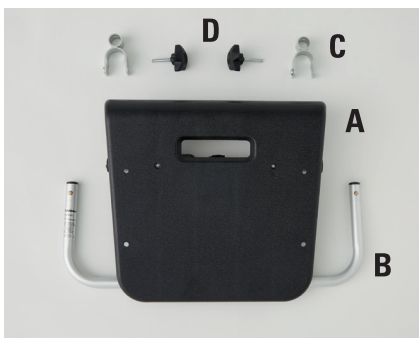
Warning

DO NOT exceed the **350 lb.** weight limit.

Warning

DOES NOT FIT Medline Junior walkers.

- A.** Seat (1)
- B.** Seat with support bar.
Comes pre-attached to the bottom of the seat.
- C.** Frame Clips (2)
- D.** Knobs (2)

Assembly Instructions

Unpacking the Walker Seat

1. Carefully remove the walker seat components from the shipping carton.
2. Examine all parts for shipping damage. **DO NOT** use the walker seat if there is apparent damage or if any parts are missing. If there are missing or damaged parts, please call Medline Customer Care toll free at **1-800-MEDLINE**.

Set up the Walker

1. Unfold the walker and make sure the walker is in a standing position before attaching the seat.

Attach the Walker Seat to the walker (Refer to Fig 1 for parts identification)

1. Take one Frame Clip and place the clip above the lower horizontal cross bar on the front of the walker. Push the clip into the frame of the walker until it snaps firmly around the frame. **(Figure 1 & 2)**

Figure 1

Figure 2

2. Repeat this step for the other frame clip on the other side of the walker.

3. Insert the knobs into the 2 frame clips. Push through until the threads on the knob engage the silver button on the opposite side of the frame clip. Hand tighten. **(Figure 3 & 4)**

Figure 3

Figure 4

4. Slide the support bar into the 2 frame clips. Push in until the brass push button snaps through the slot. **(Figure 5 & 6)**

Figure 5

Figure 6

5. Turn the walker around as if you were going to walk with it.

6. On the bottom of the seat, find the Seat Support Assembly. Notice the 2 end parts, which resemble handles, are folded under and held in place with 2 white clips. Unsnap the handles, rotate them outward so the assembly is straight. Make sure the end that is connected with the bungee cord is inserted firmly into the main tube. **(Figure 7 & 8)**

Figure 7

Figure 8

7. Now lower the seat so that the Seat Support Assembly rests on the side braces of the walker.

Figure 9

Storing the Walker Seat on the Walker (Refer to Fig 1 for parts identification)

NOTE: This assumes the walker seat has already been installed on the walker.

1. Lift the seat up and fold the seat support assembly.
2. Snap the handles into the white clips.
3. Leave the seat connected to the support bar.
4. Position the seat on the outside of the frame and connect the hook and loop strap around the frame so that it is secure. **(Figure 10 & 11)**

Figure 10

Figure 11

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Figura 11

LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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Warning: This product contains a chemical known to the State of California to cause cancer.

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GUARDIAN®

Medline Industries, LP, Three Lakes Drive,
Northfield, IL 60093 USA

Medline United States
1-800-MEDLINE (633-5463)
www.medline.com | info@medline.com

Medline Canada
1-800-396-6996
www.medline.ca | Canada@medline.com

Medline Mexico
01-800-831-0898
www.medlinemexico.com | mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2017 Medline Industries, LP. Guardian and Medline are registered trademarks of Medline Industries, LP.

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