





HeelMedix[®] Advanced

Application instructions

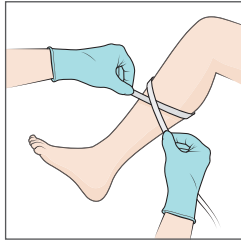


1. Measure calf circumference using measuring tape.


PETITE	5-11 inches (12.7-27.9 cm)
STANDARD	11-19 inches (27.9-48.2 cm)
X-LARGE	19-25 inches (48.2-63.5 cm)

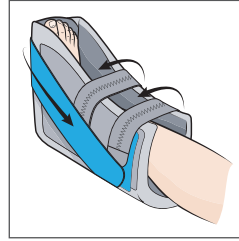


4. Secure black straps across the leg and the blue and green straps along the sides of the boot to the corresponding color strips.

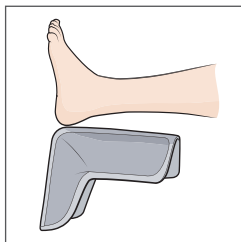


2. Open all straps and flip the boot inside out. Place the lower leg on the long side of the boot, with the heel in the hole.

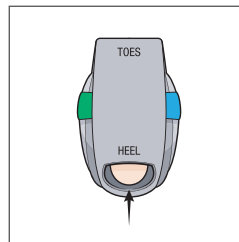


5. Make sure the heel is elevated.

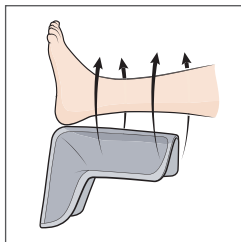

3. Pull sides up around the foot.

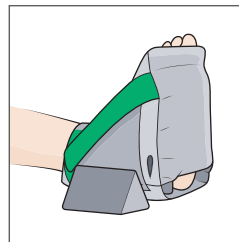

6. Apply 1 or 2 optional wedges to inside/outside of the boot for extra stabilization.

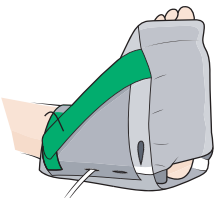

Compression device compatible

If using an Intermittent Compression Device, there are three ports on each side of the boot where tubing can be run through. If using a Sequential Compression Device, run tubing between the top two straps of the boot. Make sure no tubing is in contact with the patient's skin.



[Scan here for a step-by-step video.](#)

HeelMedix heel protector is single patient use only. Product is used on patients while in bed to help prevent pressure ulcers by offloading the heel. Patient must not walk or stand while wearing the HeelMedix heel protector. To clean, wipe down with damp cloth.

Medline Industries, Inc. Three Lakes Drive, Northfield, IL 60093 | 1-800-MEDLINE (633-5463)

© 2020 Medline Industries, Inc. All rights reserved. Medline is a registered trademark of Medline Industries, Inc. MKT19W1244595 / LIT040R / 80