

How to Use the Safety Syringe

With Sheath Technology



1 Draw up the medication following the established protocol.



2 Administer the medication. When finished, extend the safety shield forward until you hear a click. The needle is now covered.



3 To lock the safety shield using one hand, twist the safety shield until you hear another click. The needle is now locked in place and fully protected.



4 Place the syringe into the nearest sharps container for disposal.

Safety Travel Mode—when preparing the syringe in one area and transporting to the patient:

- Extend the safety shield forward until you hear a click. The needle is now covered.
- When you are ready to administer the medication, pull the safety shield back.

View How-To Video at <https://vimeo.com/65624755>

