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# Warnings & Safety Instructions

## **Warnings:**

A caregiver / transporter must be present at all times while the transport chair is in use. Never leave the occupant unattended. This transport chair design is not intended to allow a patient to maneuver the transport chair.

To ensure your safety in using this medline transport chair, all warnings and safety information and all instructions must be followed. Failure to do so may result in serious bodily injury or damage to the transport chair.

**Do not** use this product without first reading and understanding the instructions contained in this booklet. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, durable medical dealer or technical personnel, or contact medline at 1-800-medline | 1-800-268-2848 before use - otherwise serious bodily injury or product damage may occur.

**Do not** repair or make any changes to the transport chair without first reading and understanding this owner's manual or serious injury may occur. Technicians: do not service the transport chair without first reading and understanding this owner's manual. If you are unable to understand the warnings, cautions, or instructions, contact medline at 1-800-medline | 1-800-268-2848.

**Do not** use accessories, parts, or adapters other than those authorized by medline.

Consult with your doctor, physical or occupational therapist, or other qualified professional regarding the selection of the type of transport chair and accessories best suited to your individual needs and capabilities.

Save these instructions for future reference!

## **General Use:**

**DO NOT** stand on the frame of the transport chair.

**DO NOT** stand or put pressure on the footrests. When getting in or out of the transport chair, make sure the footplates are in the up position.

**DO NOT** attempt to lift the transport chair by any detachable components.

There are potential finger pinch point areas at moving parts and between the seat frame and the seat cradles on some models. When opening the transport chair, take care not to place your hand or fingers between the seat frame and the seat cradles (if present) or on any other moving parts.

ALWAYS keep hands and fingers away from moving parts. **DO NOT** sit in the transport chair until the seat is completely in the open position. Risk of injury may occur if care is not taken.

There is no folding mechanism for the push canes.

The transport chair should be on a stable, level surface with the wheel locks engaged before transferring the user to or from the transport chair.

Completely engage the wheel locks when entering and leaving the transport chair or using the transport chair in an elevator or a wheelchair lift.

**DO NOT** transfer in or out of the transport chair on your own unless you have sufficient upper body strength, balance and agility and have been instructed in transfer techniques by a health professional. Every precaution should be taken to reduce the gap distance between the transport chair and the target seat or bed. Turn both casters parallel to the object you are transferring to.

**DO NOT** use the wheel locks (or hand grip locks if present) to stop the transport chair from moving. Wheel locks are not brakes. **DO NOT** use the wheel locks to slow your descent down an incline.

Changing direction when going down an incline could cause instability.

**DO NOT** lean forward out of the transport chair any farther than the length of the armrests.

**DO NOT** attempt to reach for objects if doing so means you must move forward in the seat.

**DO NOT** attempt to reach objects if you have to pick them up from the floor by reaching down between your knees.

**DO NOT** shift your weight or sitting position toward the direction you are reaching as the transport chair may tip over.

**DO NOT** attempt to lean over the top of the back upholstery to reach objects.

Before leaning or reaching forward, make sure to rotate the casters straight ahead toward the front of the transport chair and engage the wheel locks to prevent the transport chair from tipping over or moving. **DO NOT** tip the transport chair without assistance. **DO NOT** lean the transport chair back since this can cause the transport chair to tip over. If trying to reach objects, reach back only as far as your arm will extend without altering your seating position.

Any person assisting the rider in maneuvering or lifting of the transport chair must have sufficient strength and be able to see obstacles in the path of travel. When learning a new assistance technique, have an experienced assistant help you before attempting it alone.

Remember to use good body mechanics.

Contact with water or excess moisture may cause the non-aluminum transport chairs to rust or corrode.

**DO NOT** use the transport chair in a shower, pool or other body of water. Avoid excess moisture (example: do not leave the transport chair in a damp bathroom while taking a shower). Dry the transport chair immediately if exposed to moisture.

When cleaning the hand grips, use only a clean towel lightly dampened with cool water. Ensure the hand grips are dry before use. Use of soap or ammonia-based cleaning solutions may result in the hand grips sliding off.

If the transport chair is exposed to extreme temperatures, high humidity and/or becomes wet, check the hand grips to ensure they are not loose and do not twist on the handle.

Avoid storing or using the transport chair near open flames or combustible products.

### **Adjustments**

**DO NOT** operate, maintain, or adjust the transport chair without reading and understanding all instructions in the manual.

**DO NOT** adjust the footrests any closer than 2 ½ inches (6.3 cm) from the floor.

**DO NOT** modify the transport chair. Examples: drilling holes, grinding, welding parts, using non-authorized / non-approved parts. Such modifications may change the transport chair's structure and center of balance, which may increase the risk of serious injury to the user and void the warranty.

### **Before Each Use**

- √ Make sure the transport chair rolls easily and that all parts work smoothly.
- √ Check for noise, vibration, or any change in use. Changes may indicate loose parts or transport chair damage.
- √ Check to ensure wheel locks are in good working order.
- √ Make sure hand grips are not loose and cannot be moved or twisted.
- √ Make sure the seatback is locked in the upright position.
- √ Have any problem found repaired promptly with authorized parts.

# Guidelines: Safety & Handling of Transport Chairs

## Center of Gravity, Balance and Stability

The transport chair has been designed to be safe and stable during normal transport activity as long as proper balance is maintained. The point at which a transport chair will tip forward, backward, or to the side depends on its center of balance. To assure stability and proper operation of the transport chair, the center of gravity must be maintained in the transport chair. Center of gravity is the point at which all the weight of an object can be considered to be concentrated and represents the balance point.

Most activities such as transferring in and out of the transport chair, bending, and reaching affect the center of gravity and weight distribution of the transport chair. Addition of accessories such as a backpack will also re-distribute weight. Loss of proper balance may cause the transport chair to tip over. Use of the seat belt is advised to help maintain stability by minimizing movement within the transport chair seat. The use of anti-tip devices especially when maneuvering on ramps or inclines will also enhance safety.

Transport chairs are designed for safe and efficient transport with someone pushing the transport chair. Transport chairs are not intended for independent use by transport chair-bound individuals. The design does not allow maneuvering by the occupant.

To determine safety limits of the transport chair, practice with activities such as transferring, bending, and reaching with a healthcare professional that is knowledgeable about transport chair use.

To assure safe operation of the transport chair, ALL of the Warnings and Instructions in this manual must be followed. Failure to do so may result in tipping the transport chair over or causing the user to fall which may result in serious bodily injury or damage to the transport chair.

Some basic transport chair safety guidelines follow. The guidelines are critical for standard transport chair use. If any of the following activities are required while using the transport chairs, these guidelines must be followed to ensure safety and to avoid potential injury to the user or the transporter and to avoid damage to the transport chair.

## Reaching / Leaning / Bending – Forward or Sideways

- Position the transport chair as close as possible to the object.
- Make sure the front casters are pointing in a forward position. This extends the wheel base and will help stabilize the transport chair.
- Make sure the wheel locks are engaged before activity.
- **DO NOT** attempt to reach for objects if doing so means you must move forward in the seat. **DO NOT** shift your weight toward the direction you are reaching.

- **DO NOT** attempt to reach objects if you have to pick them up from the floor by reaching down between your knees.
- **DO NOT** shift your weight or seat position toward the direction you are reaching.

## **Reaching / Leaning – BACKWARD**

- Position the transport chair as close as possible to the object.
- Make sure the front casters are pointing in a forward position. This extends the wheel base and will help stabilize the transport chair.
- **DO NOT** engage the wheel locks – if your weight shifts while reaching backwards it is better for the transport chair to roll than to tip.
- **DO NOT** lean back over the top of the back upholstery.
- Reach back only as far as your arm will extend without changing your position in the seat.

## **Transferring into and out of the transport chair**

- Position the transport chair as close as possible.
- Make sure both wheel locks are engaged.
- Remove or swing away the footrests that are in the transfer path.
- **DO NOT** put weight on the foot rests.
- Transfer as far back into the seat or onto the bed/transport chair as possible.

## **Obstacles**

Obstacles such as floor threshold strips, uneven floors, cords or items on the floor, or cracks in pavements can damage your transport chair and may cause loss of control or a tip over. Be alert! Scan your path and avoid any items that may cause instability.

## **Curbs, Steps and Stairs**

Do not attempt to negotiate curbs or single steps without assistance. Ask your healthcare worker to provide instructions regarding safe transport chair use for curbs, steps and stairs. Proper body mechanics and awareness of the center of gravity is crucial for these maneuvers. The step tubes at the back of the transport chair are utilized to tilt the transport chair backwards until the balance point is achieved and the front casters clear the curb/step. Gently lower the front wheels to the ground and slowly roll the transport chair forward until the rear wheels roll up over the curb/step.

Extreme caution is required if it is absolutely necessary to maneuver an occupied transport chair up or down stairs. Two assistants with adequate body strength and endurance must be present. **DO NOT** lift the transport chair by any removable parts (i.e. swing-away footrests). Make sure the hand grips are not loose. Techniques for going up or down steps should be received from a health care professional familiar with transport chair use and practiced before attempting to transport on stairs. If at all possible, an alternate means of transport up or down stairs is recommended.

## Ramps / Inclines

The assistant must know their capabilities and limitations in terms of strength and endurance before attempting to go up or down a ramp. Slopes will change the center of gravity. **DO NOT** attempt to negotiate steep inclines greater than 10% (6 degrees, or one foot of rise or fall for every ten feet of ramp length) without an experienced assistant. Use of anti-tip devices are encouraged however, they may not prevent a tip-over. Sometimes anti-tip devices may catch at the beginning of a ramp. If this occurs, they will need to be disengaged and extra care should be taken. Backpacks or other gear on the back of the transport chair will cause it to tip backward more easily. Training and practice negotiating ramps with a healthcare professional is recommended.

- Go as straight up or down the incline as possible.
- Check the ramp for any obstacles, bumps, depressions, or wetness before beginning. Ask for additional assistance if necessary.
- Stay in the center of the ramp if possible.
- Avoid changing direction while on the ramp as this may cause loss of control of the transport chair.
- Avoid stopping on a ramp. Stopping may cause loss of control of the transport chair. If stopping is necessary, avoid sudden, abrupt forward movement when you resume propelling the transport chair as that could cause instability.
- When going up an incline, the occupant should lean forward. This will adjust the center of gravity to counteract the tendency of the transport chair to tip backward.
- When going down a ramp, the occupant should face forward and lean backward into the seat. Descent should always be done slowly. **DO NOT** use wheel locks as brakes to slow down or stop the transport chair. Attempting to use the wheel locks could result in accidental locking, veering or tipping to one side, or suddenly stopping.
- When going down a steep incline, the assistant should turn the transport chair and go down backwards to avoid tipping the occupant out of the transport chair.

## Basic Information for Transport Chair Assistants / Transporters

Work with the transport chair user's healthcare professional or your institution to learn safe ways to assist that are within your capability and comfort level. To prevent injury to yourself or the transport chair occupant use good body mechanics:

- Always tell the rider what you intend to do so he/she is ready for your actions and can assist with maintaining a balanced center of gravity for stability.
- Bend at the knees, not at the waist. Keep knees bent, not locked straight.
- Use legs for strength, not the weaker muscles of the back or arms. This will help prevent back strain.
- Do not twist at the waist. Keep the torso facing the same direction as the hips and move the feet to turn. This will help prevent back strain.
- Maintain a straight back. Hunching or rounding the shoulders can cause back strain.
- Do not hesitate to call for additional assistance if needed.
- If you do not feel comfortable with an activity you are requested to assist with (such as negotiating a step or a steep incline), you may decline to assist and should help find appropriate assistance for the occupant.

# List of Contents

1. Transport chair
2. Footrests (pair)
3. Manual

Typical Product Specifications for the 22" (55.9 cm) Bariatric Transport Chair:

Weight	48 lb (22 kg) (excluding foot riggings)
Weight Capacity	450 lb (204 kg)
Seat Width	22" (55.9 cm)
Seat Depth	18" (45.7 cm)
Armrests	Removable desk length
Footrests	Swing-away detachable footrest
Tires	Solid front and rear with tread
Warranty on Frame & Crossbar	Limited Lifetime
Warranty on Upholstery Parts & Components	1 year

## General Use Instructions

Opening and Folding the Transport Chair

<b>Important Notes:</b>
<p>There are potential finger pinch point areas at moving parts and between the seat frame and the seat cradles on some models. When opening the transport chair, take care not to place your hand or fingers between the seat frame and the seat cradles (if present) or on any other moving parts.</p> <p>ALWAYS keep hands and fingers away from moving parts. DO NOT sit in the transport chair until the seat is completely in the open position. Risk of injury may occur if care is not taken.</p>

## To Open the Transport Chair

1. Unfold the chair by pressing firmly on the ends of the seat – keep your fingers flat for safety.

## To Close / Fold the Transport Chair

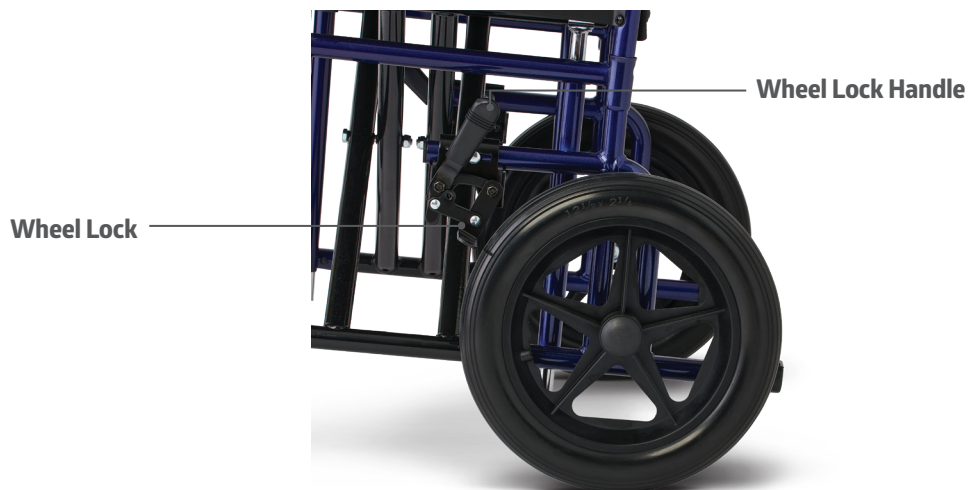
1. Ensure the swing-away legrests are in the forward and locked position.
2. Fold the footplates up to the vertical position and swing back the legrests.
3. Using both hands, grasp the two handles on the seat upholstery and lift the seat up.

# General Use Instructions

## Wheel Locks

- Pushing forward will lock the brake
- Pulling back will release the brake.

# General Use Instructions

## Wheel Lock Warnings

- Check the wheel locks to ensure proper function before using the transport chair. Wheel locks should embed into the wheel at least 1/8 inch (3 mm) when engaged.
- Engage the wheel locks on both wheels before entering the transport chair, leaving the transport chair, leaning forward in the transport chair or while being transported in an elevator or a transport chair lift.
- Check to make sure that the legrests do not unlock the wheel locks when they are swung back.
- DO NOT use the wheel locks (or hand grip locks if present) to stop the transport chair from moving. Wheel locks are not brakes. DO NOT use the wheel locks to slow your descent down an incline.

# Installing / Removing the Swing-Away Leg Rest / Footrest Riggings

## To Install the Riggings:

1. Hold the leg rest rigging to the side of the transport chair.
2. Install the leg rest assembly hinge plates onto the hinge pins of the transport chair frame.
3. Push the leg rest rigging to the front until it locks securely into place.

## To Swing-Away / Remove the Legrest Assembly:

1. Fold the footplate up to the vertical position.
2. Release the legrest lock by pushing on the legrest lever.

## To Remove the Armrests

1. Press firmly the push button located on the bottom front of the armrest.
2. Slowly removing the armrest to the upward position while still pressing the push button until the whole armrest is completely out from the frame.

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## MEDLINE LIMITED WARRANTY

Medline has great confidence in the quality of our transport chairs. That is why Medline warranties for the original purchaser, that the frame of the transport chair will be free from defects in materials and workmanship for the life of the transport chair.

Medline also warranties the parts and upholstery on the transport chair to be free from defect in materials and workmanship for a period of one (1) year from date of purchase by the original purchaser.

If within the warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Medline's option.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear.

All warranty claims must include the product in question unless otherwise authorized by Medline.

For more information about your transport chair, warranty or to reorder parts call 1-800-MEDLINE | 1-800-268-2848.

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Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2025 Medline Industries, LP. Medline is a registered trademark of Medline Industries, LP.