
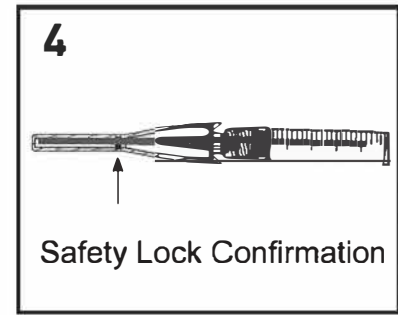
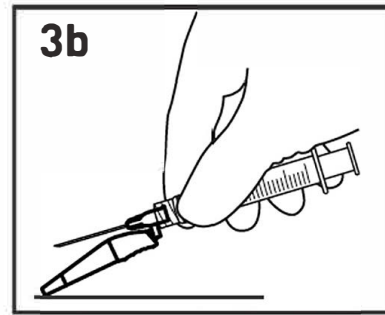
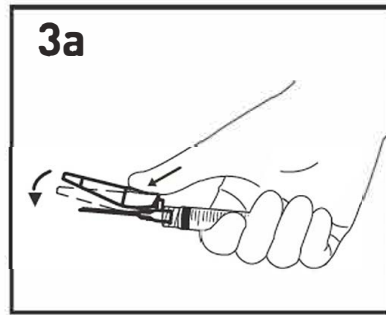
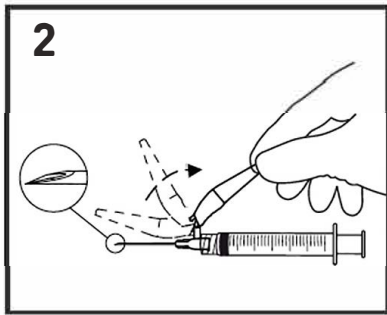


INSTRUCTIONS FOR USE

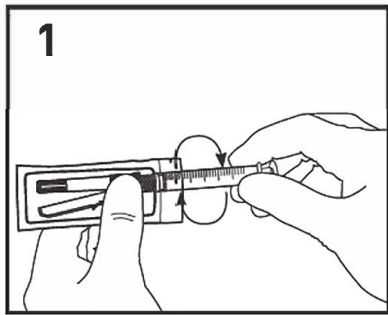

1. Tighten a syringe to the needle using aseptic technique. Grip the base of the needle, not the safety hinge, and twist the syringe clockwise.
2. Flip the safety hinge back, away from the needle and toward the syringe barrel. Then, carefully remove the needle cover. Perform injection procedure following facility protocol. **NOTE:** The needle's bevel is oriented up when facing the protective hinge.
3. Activation: After the injection procedure, use a one-handed technique to activate the hinged safety mechanism by firmly pressing down using a thumb or hard surface. Activation is verified by an audible and tactile "click."
4. Visually confirm that the safety hinge is fully engaged. Dispose of used needles and materials following the policies and procedures of your facility as well as federal and local regulations for sharps disposal.