





Disposable Blood Pressure Cuff

Instructions For Use


Rx only. Intended for use with non-invasive blood pressure monitoring systems.
If electronic, attach all tubes to monitor and follow manufacturer's instructions.
If manual, attach one tube to aneroid and one tube to bulb and valve.

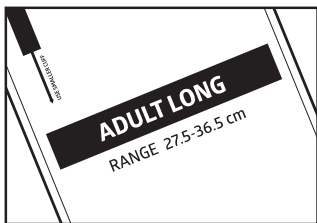


1 Select the cuff that is the appropriate size based on limb circumference.

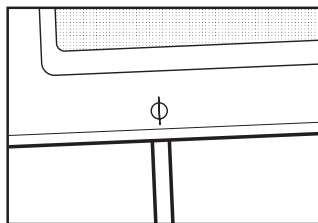


2 Locate the artery line indicator on the cuff.

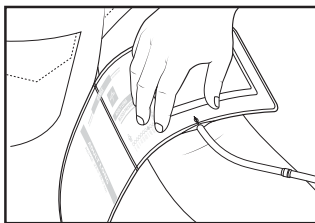

3 Locate the brachial artery and align the artery line indicator along brachial artery.

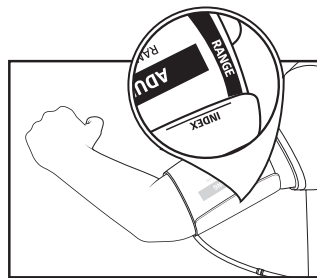

4 Apply the blood pressure cuff to the bare upper arm, ensuring the index line falls within the range printed on the cuff. Patient should be seated with their arm resting at the same level as their heart. Ensure tubing is directed down away from cuff. Take blood pressure.

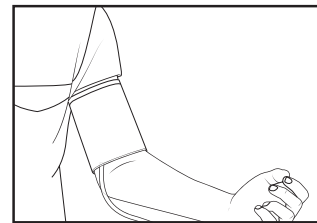

Precautions/Warnings:

Accuracy is determined by using the appropriate size cuff. If the index line falls outside of the range indicator, please use a smaller/larger cuff. The blood pressure cuff should be secured to the arm to a snug fit. If using thigh cuff for a non-upper arm reading, please position on the leg. Make sure to align arrow marked ARTERY over the patient's brachial artery (or popliteal artery for thigh measurement). Press the rough and soft sides of the closure together. Discard BP cuff if it becomes soiled.