

BD Surgiphor™

Antimicrobial Irrigation System

Ready-to-use

REF 910110



DESCRIPTION:

An antimicrobial irrigation system containing 0.5% povidone iodine (PVP-I) in phosphate-buffered saline, potassium iodide and Vitamin E TPGS. PVP-I acts as a preservative to help inhibit microbial growth in the irrigation solution.

- Sterile solution
- Non-pyrogenic
- Not made with natural rubber latex
- Not made with di(2-ethylhexyl) phthalate (DEHP)

SYSTEM CONTENTS:

(1) Cap

(1) BD Surgiphor™ 450 mL bottle

- 0.5% Povidone iodine
- Phosphate-buffered saline
- Potassium iodide
- Vitamin E TPGS

INDICATIONS FOR USE:

BD Surgiphor™ Antimicrobial Irrigation System is intended to mechanically loosen and remove debris, and foreign materials, including microorganisms, from wounds.

CONTRAINDICATIONS:

- Not for use in patients with known allergic reaction to any of the ingredients in the solution.

CONTRAINDICATIONS (continued):

- Do not combine with other antiseptic solutions, such as octenidine, due to potential reactions and reduction in the effectiveness of the solution.
- Not for use in neonates (less than 28 days after birth).

WARNINGS:

- Do not use or mix with other cleansers, soaps, lotions, or ointments.
- Do not use if packaging is damaged or if seal integrity is compromised.
- Anaphylaxis may occur in patients with severe iodine allergy.
- May cause allergic reactions such as rash or skin irritation in patients with iodine allergy. Discontinue use immediately if irritation or an allergic reaction occurs.
- Do not use for injection or infusion.
- Do not swallow.
- Do not use and avoid contact with eyes and ear canals.
- Do not use more than 24 hours after puncturing the seal.

PRECAUTIONS:

- May cause a temporary irritation and/or burning sensation on exposed skin in very rare cases.
- Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.
- Single patient use only.
- Do not use past the expiration date.
- Not for at-home use.


INSTRUCTIONS FOR USE:

- Use only as directed.
- Do not dilute the solution before use.


See other side for further instructions

INSTRUCTIONS FOR USE (continued):

1. Remove bottle and cap from tray using aseptic technique.
2. Screw the cap on to the bottle and fully tighten, cap will puncture the seal as it is tightened.

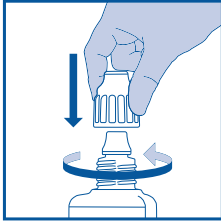

3. Invert the bottle, compress the bottle and direct the fluid into the wound to irrigate the entirety of the wound area.

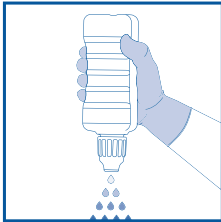

4. Immediately after irrigation remove BD Surgiphor™ solution from the wound, and rinse thoroughly with sterile saline solution.
5. Remove excess saline from the wound area.
6. If wound requires additional irrigation, remaining BD Surgiphor™ solution may be used until the solution has been exhausted, for up to 24 hours after puncturing the seal. After 24 hours, dispose of unused solution and device in accordance with local regulations.

STORAGE CONDITIONS:

- Store at room temperature
- Avoid freezing or heating above 40°C (104°F)

SYMBOL & MARK KEY:

R_x Only

Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.



Consult instructions for use



Do not reuse



Catalog number



Batch code



Sterilized using irradiation



Do not use if package is damaged



Not made with natural rubber latex



Manufacturer



Solution is sterile

 Becton, Dickinson and Company™
75 North Fairway Drive,
Vernon Hills, IL 60061
1-844-823-5433

bd.com • bd.com/surgeryifu
bd.com/symbols-glossary
7-910101-B • Made in USA

BD, the BD Logo, the Sterile Solution design and Surgiphor are the property of Becton, Dickinson and Company or its affiliates. ©2022 BD. All rights reserved.