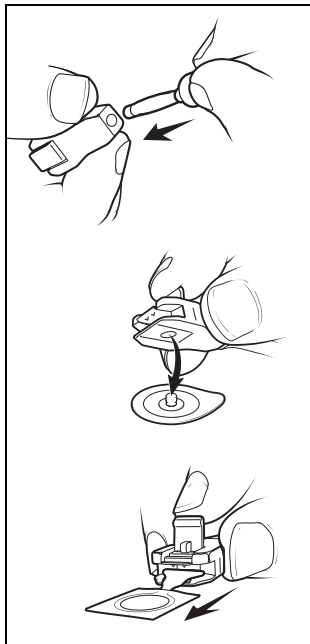
Reusable Electrode Adapter

Instructions for Use

REF 989803166031
Clear Tab/Snap Adapter
Banana Plug 3-4 mm, 10/bag

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



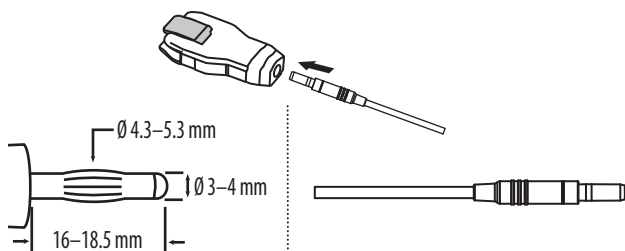
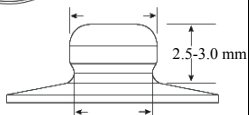
$\varnothing 3.6-3.8$ mm

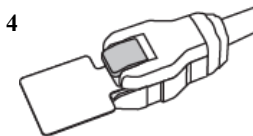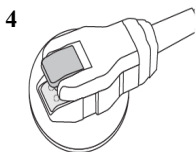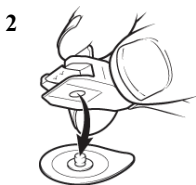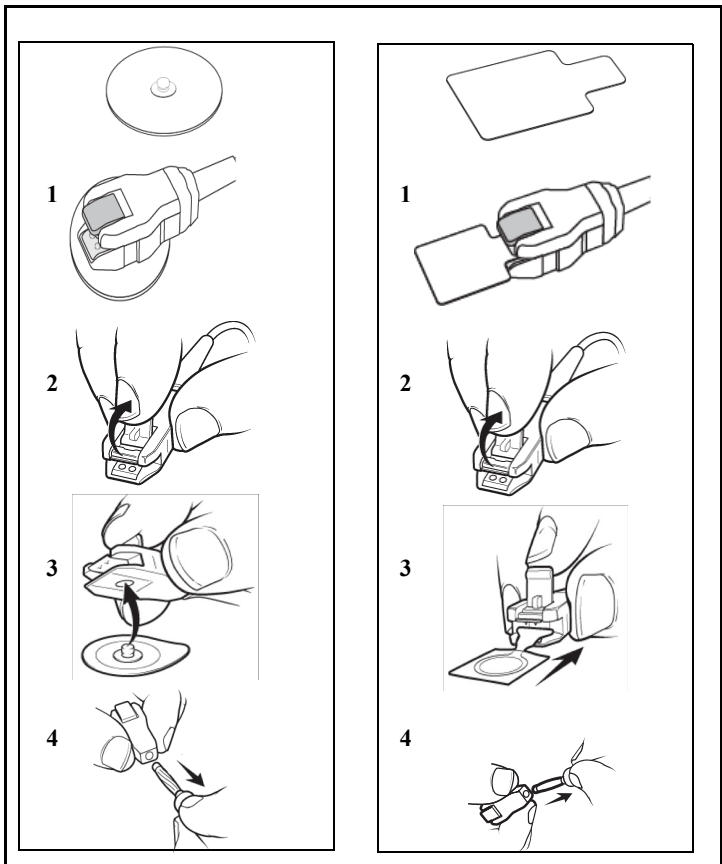Intended Use

The Philips Reusable Electrode Adapter is intended for use by or under the supervision of a licensed physician or other healthcare provider. This adapter provides a connection between lead wires and ECG electrodes.

Compatibility

The Electrode Adapter 989803166031 accepts any standard monitoring lead that has a 3-4 mm (0.12 - 0.16 inches) banana plug.

The Electrode Adapter 989803193821 accepts any standard monitoring lead that comes with a female Safety DIN (42802) plug.

The adapters may be locked onto both snap and tab electrodes. (*Note: electrodes with neck diameters less than 3.6 mm (0.141 inches) or larger than 3.8 mm should not be used.*)

Disposal

Immediately discard any adapter showing visual signs of damage or deterioration. Follow approved medical waste disposal methods specified by your patient care facility or local regulations.

Warnings

- Do not use the adapter with MRI equipment.
- Always inspect the adapter before use. Do not use if visual inspection reveals damage, contamination or if adapter shows signs of corrosion of the metal spring or deterioration of the plastic housing.
- When disconnecting a lead from an adapter, be sure to pull on the banana/DIN plug. Do not pull on the cable or lead.
- Never immerse the adapter in any liquid. Doing so may damage the adapter, including corrosion.
- Do not sterilize the adapter by autoclave, radiation, or steam methods. See the *Cleaning and Sterilization* sections in this document.

Symbols and Definitions

Page i provides definitions of symbols used on product packaging.

Directions for Use

To attach or remove the adapter, follow the steps on pages ii-iv at the beginning of this manual.

Cleaning

Note: Cleaning with any compounds or methods other than those indicated below may damage the product.

1. Disconnect the patient cable and/or leadwire from the monitoring device.
2. Dampen a clean cloth or gauze pad with one of the following appropriate cleaning solutions and wipe all exposed surfaces..

Cleaning/Disinfecting Agents			
Isopropanol	<input checked="" type="checkbox"/>	Carpe Diem ^{TM/MC} Tb	<input checked="" type="checkbox"/>
Bacillo [®] AF	<input checked="" type="checkbox"/>	Super Sani-Cloth	<input checked="" type="checkbox"/>
Bacillo [®] 25	<input checked="" type="checkbox"/>	SANI-CLOTH [®] PLUS Germicidal Disposable Wipes	<input checked="" type="checkbox"/>
Melisep [®]	<input checked="" type="checkbox"/>	SANI-CLOTH [®] HB Germicidal Disposable Wipes	<input checked="" type="checkbox"/>
Accel TB RTU	<input checked="" type="checkbox"/>	Hexanios G+R	<input type="checkbox"/>
Oxivir [®] Tb Cleaner Disinfectant	<input checked="" type="checkbox"/>	Sodium hypochlorite, bleach	<input type="checkbox"/>
Oxivir [®] Tb Wipes	<input checked="" type="checkbox"/>	Surfanios Citro	<input type="checkbox"/>

Notes: = approved cleaner/disinfectant.

= cleaning/disinfecting agent is known to cause damage to plastic material and should be avoided, if possible.

After cleaning/disinfecting, ensure that all residue is removed by wiping the surface using a clean cloth dampened with clean water.

® and TM/MC denote name is a trademark of the respective manufacturer.

3. Dampen a clean cloth or gauze pad with sterile or distilled water and again wipe all exposed surfaces.
4. Dry all exposed surfaces with a clean, dry cloth or gauze pad.

Sterilization

The adapter may be subjected to 10 sterilization cycles by ethylene oxide method.

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Copyright © 2021
All rights reserved
Böblingen GmbH
Limited