

CAUTERY

Store At Room Temperature

WARNINGS:

DO NOT USE IN THE PRESENCE OF FLAMMABLE MATERIALS OR IN OXYGEN-ENRICHED ENVIRONMENTS. FIRE COULD RESULT.

THIS MEDICAL DEVICE CANNOT BE EFFECTIVELY CLEANED AND/OR STERILIZED BY THE USER AND THEREFORE CANNOT BE SAFELY REUSED. IT IS INTENDED FOR SINGLE USE ONLY. ANY ATTEMPT BY THE USER TO CLEAN AND RESTERILIZE THIS DEVICE MAY RESULT IN BIOINCOMPATIBILITY, INFECTION, OR OTHER RISKS OF DEVICE FAILURE TO THE PATIENT.

ALWAYS RETAIN THE COVER CAP AND REPLACE IT ON THE CAUTERY WHEN NOT IN USE.

DEACTIVATE THE CAUTERY BEFORE DISCARDING THE DEVICE IN A BIOHAZARD OR "SHARPS" DISPOSAL CONTAINER. FIRST BREAK OR CUT OFF THE CAUTERY WIRE TIP USING HEMOSTATS OR SCISSORS AND REPLACE THE COVER CAP. FAILURE TO FOLLOW THIS INSTRUCTION COULD RESULT IN UNINTENDED ACTIVATION OF THE CAUTERY AND MAY CAUSE A FIRE IN THE DISPOSAL CONTAINER.

CAUTIONS:

R_x ONLY



Read Instructions Before Use



Do Not Use if Packaging is Damaged or Opened

STERILE EO



Do Not Reuse. This is a Single Use Device




Do Not Resterilize

MC-18189 Rev. 4

I



II

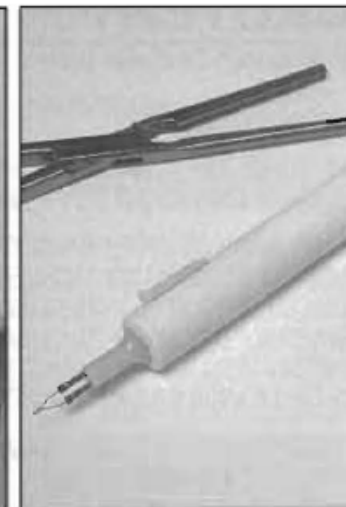

III



IV

