

## Forearm Crutches adult

ADJUSTABLE | VINYL CONTOURED GRIP

Patient Height: 5 ft–6 ft 2 in

Provides a total weight bearing capacity of 300 lbs (136kg).

For use with MFR # 146-RTL10403.

### ADJUSTING LEG HEIGHT

1. Before use, adjust height of leg to your specific needs by depressing the push button and loosening the anti rattle collar (Figure 1).
2. When desired height is reached, lock leg into place by aligning button into proper hole, (Figure 2).
3. To be sure leg is locked into place by pushing down on or by pulling leg.

### CLEANING INSTRUCTIONS

1. Use a non-abrasive mild detergent or a household cleaner with warm water.
2. Rinse with warm water and wipe dry.
3. **DO NOT USE AN ABRASIVE CLEANER OR CLOTH** with this product as damage may ensue.

FIGURE 1


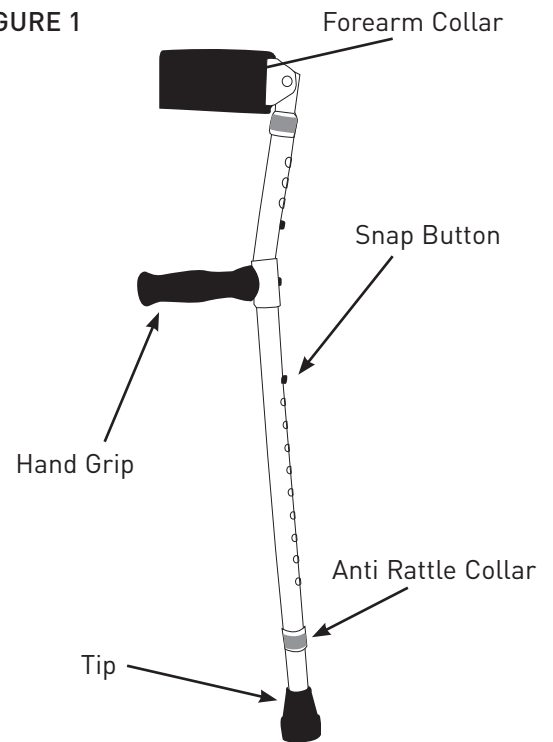


FIGURE 2

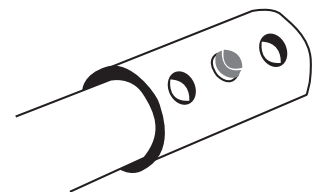

**WARNING:** This product can expose you to bisphenol A (BPA), which is known to the State of California to cause cancer and birth defects or other reproductive harm. For more information, go to [www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov).

Not made with natural rubber latex.

Technical support, replacement parts and warranty information? Call 1-866-694-5085

General product questions? Call 1-800-777-4908

### Satisfaction Guaranteed

For complete details, please visit  
[mms.mckesson.com/mckesson-brands](http://mms.mckesson.com/mckesson-brands).

Distributed by McKesson Medical-Surgical Inc.  
Richmond, VA 23233  
PVN B0521 Made in China