

MCKESSON

Ear Wash System FLEXIBLE TUBE

1

KIT PER BAG

CONTENTS: 1 Spray bottle, 3 single use ear tips

DIRECTIONS

1. Fill washer bottle with warm (not hot or cold) water.
2. Twist a disposable tip onto the end of the nozzle. Your ear wash system comes with 3 tips. Additional tips are available and fit all McKesson Ear Wash Systems.
3. If you have an ear basin, place under the ear to catch fluid or lean over sink. Rapidly pump solution into the ear to flush out wax. For heavy build up, vigorous washing may be necessary. You may need to refill the bottle.

Not made with natural rubber latex.

Questions? Call 1-800-777-4908

Satisfaction Guaranteed

If you are not completely satisfied with any McKesson Brands product, you may return it for a full refund or credit.

Distributed by
McKesson Medical-Surgical Inc.
Richmond, VA 23233
PVN A0517 9000-088
Made in USA

LOT



(01)10612479224602

MFR # 140-2

MCKESSON

WARNING

1. Do not use this device in the presence of infection, ruptured tympanic membranes or tympanic tubes.
2. Do not use on individuals that have ear pain or bloody discharge. Ear washing should not hurt. Pressure is normal but if there is pain, stop and consult a physician.
3. Make sure to install a tip prior to use. Ensure that the tip end is not inserted too far into the ear canal as to avoid injury to the tympanic membrane.
4. To prevent injury, use this Ear Wash System on a seated individual. Never use a solution that is cooler than body temperature or dizziness may occur.
5. Do not pull used tip off end; make sure to unscrew after each use. Keep tip tubing fairly straight while using to allow for good flow. If flow is impeded due to bending or plugging of the tip tubing, built-up pressure may cause the tubing to separate from the tip during use.
6. Do not use this device for any other purpose except for which it is intended.
7. Always re-read the instructions printed on the bottle prior to each use.

Do not autoclave.

CAUTION: Always examine the individual's ear before and after cleaning. Discard tip after use. Do not re-use tips. You may disinfect the exterior of the white hose/nozzle by wiping off using a dilute disinfectant solution. You may disinfect the interior with the same solution by spraying through the trigger mechanism. Allow Ear Wash System to dry before storing.