# Universal Thumb Splint

For use with MFR #'s 155-BH82710.

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

**INDICATIONS**

Thumb sprains or strains. Hyperextension, arthritis of the first carpal metacarpal (Basal Joint Arthritis), and thumb tendonitis.

**CONTRAINDICATIONS**

Phlebitis patients should not wear neoprene products.

**PRECAUTIONS**

- Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene by-products. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately. Never wear the

**DIRECTIONS FOR USE**

1. Place the thumb in the support.
2. Using the contact closure on one side of the wrist support, wrap and secure the wrist wrap.
3. Secure thumb wrap.
4. Radial, palmar and dorsal stay pockets contain a malleable aluminum stay and a spiral stay which can be removed for desired support

- NOTE: Wrist and thumb strap may be trimmed as needed.

**CLEANING INSTRUCTIONS**

Hand wash liner in warm water using mild soap, rinse thoroughly. AIR DRY. NOTE: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

**FOR SINGLE PATIENT USE ONLY.**





Not made with natural rubber latex.

Questions? Call 1-800-777-4908  
 Technical Support? Call 1-800-336-6569

Distributed by McKesson Medical-Surgical Inc.  
 Richmond, VA 23233  
 PVN A0520  
 PN 13-5509 Rev A  
 Made in China

Symbol Definitions	
Symbol	Definition
	Consult instructions for use
	Caution, consult accompanying documents

**NOTICE:** While every effort has been made in state-of-the-art techniques to obtain the maximum compatibility of function, strength, durability and comfort, this device is only one element in the overall treatment program administered by a medical professional. There is no guarantee that injury will be prevented through the use of this product.

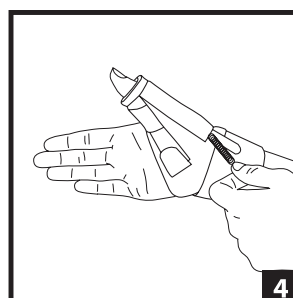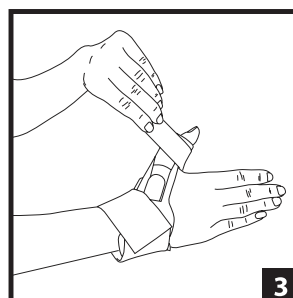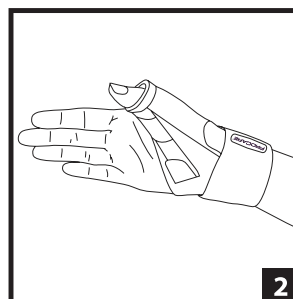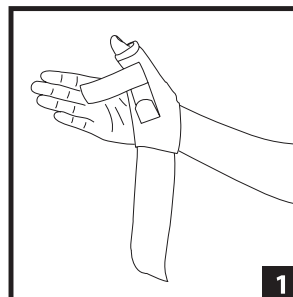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Distribuido por McKesson Medical-Surgical Inc.