MOBILITY  
MOVILIDAD  
MOBILITÉ

McKESSON

## Walker Glides tennis balls

WITH REPLACEMENT GLIDE PADS  
EASY TO INSTALL | DURABLE

Provides a quiet, smooth glide experience  
when used with a walker.

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
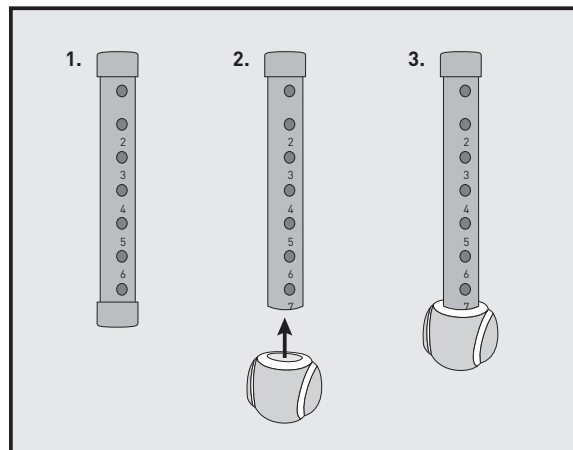
## INSTALLATION INSTRUCTIONS


1. Remove tips from both rear legs of walker
2. Slide Tennis Ball Glide over tubing. Push Tennis Ball Glide up until tubing is secure inside Tennis Ball Glide

## TO REPLACE GLIDE PADS

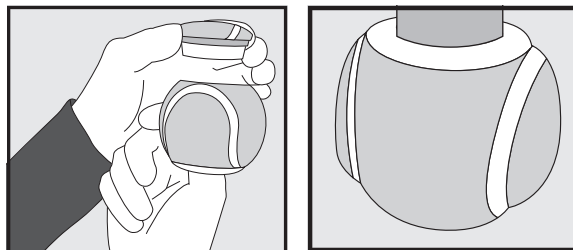
1. Pull Tennis Ball Glide off extension leg.
2. Grasp tennis ball with one hand and insert thumb into Tennis Ball Glide. Push replacement tennis ball pad until it is removed from Tennis Ball Glide.
3. Push replacement glide pad into tennis ball glide until it is secured into position.
4. Replace pads when felt is worn.

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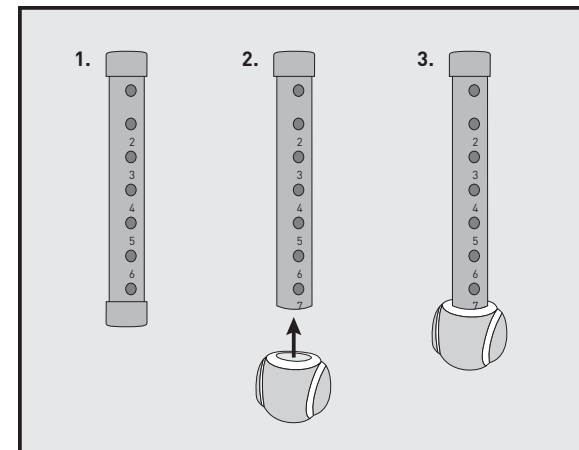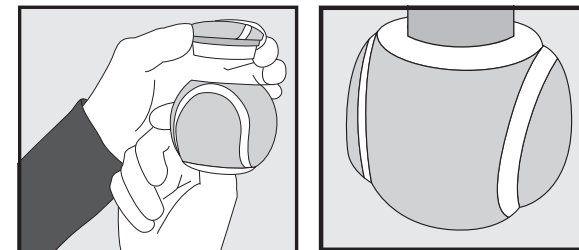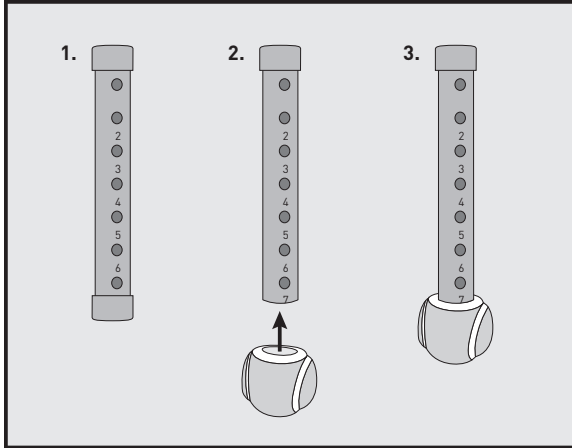
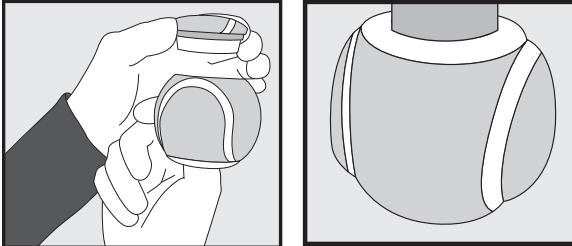
McKesson carries a comprehensive selection of durable medical equipment products for bath safety, mobility and aids to daily living, including wheelchairs, canes, bath benches, and more. Choose McKesson for quality, selection, and value.

Technical support, replacement parts and warranty information? Call 1-866-694-5085  
General product questions? Call 1-800-777-4908

**Satisfaction Guaranteed**

Distributed by McKesson Medical-Surgical Inc.  
Richmond, VA 23233  
PVN A0618  
Made in China