
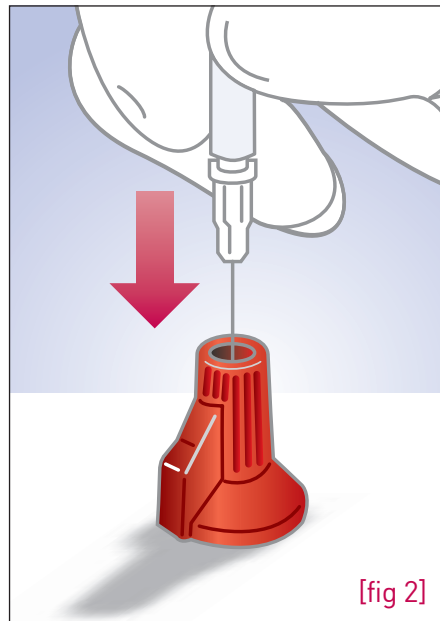


Point-Lok® Sharps Safety Device

Step-by-Step Guidelines

The Point-Lok® device provides a standardized sharps safety management program for a wide range of healthcare professionals using needles from 16-30 gauge that do not have a built-in engineered sharps injury protection device.

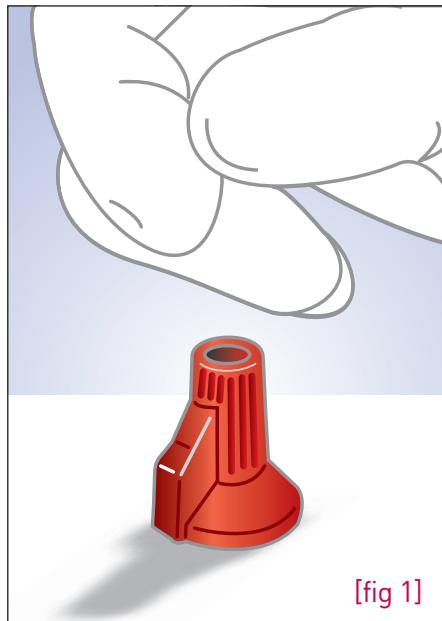


1 Place

- Place the Point-Lok® device on a flat, sturdy surface — **NOT ON YOUR HAND.** [fig 1]

2 Insert



- Insert the pointed tip of an exposed needle into the open end of the Point-Lok® device and gently push down. [fig 2]

NOTE: A patented mechanism securely locks the needle within the Point-Lok® device. A layer of soft plastic in the bottom of the device provides a leak-resistant seal.

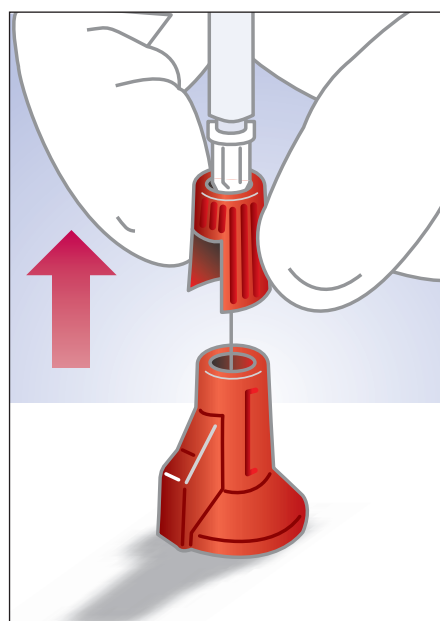

3 If you need to remove the needle from the syringe

- To remove the needle from the syringe, slide the unwinder off the Point-Lok® device and engage it with the needle Luer.
- If you do not need to remove the needle from the syringe, dispose of the needle and attached Point-Lok® device in an approved sharps container.

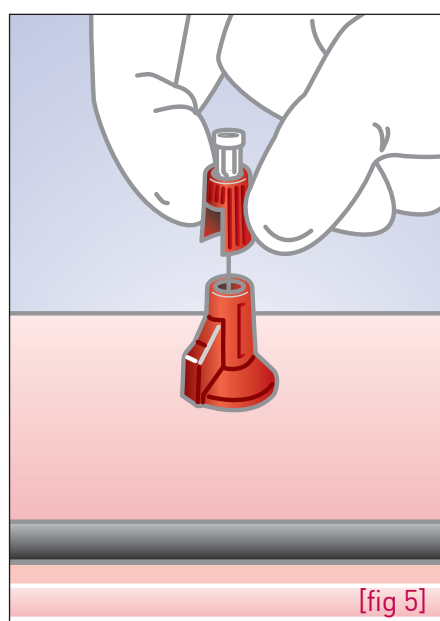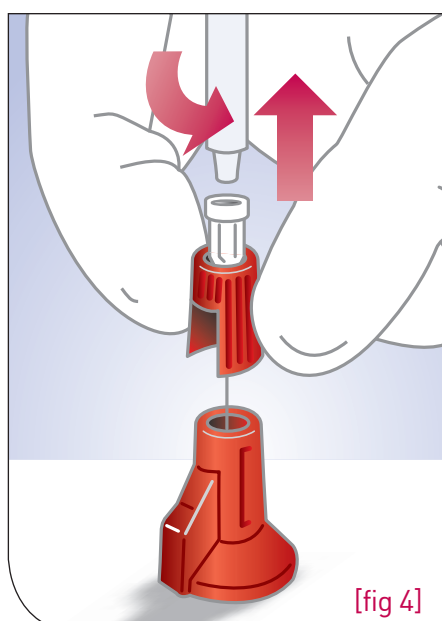

4 Unscrew

- Unscrew the engaged needle from the syringe. [fig 4]

5 Dispose

- Dispose of the needle and attached Point-Lok® device in an approved sharps container. [fig 5]

Smiths Medical ASD, Inc.

160 Weymouth Street, Rockland, MA 02370 USA

T: 800 848 1757 F: 800 621 2590 www.smiths-medical.com

Smiths Medical, part of the global technology business Smiths Group

Caution: Federal law (U.S.A.) restricts this devices to sale by or on the order of a physician. Please read the Instructions for use supplied with the product for detailed instructions, warnings, and cautions on the safe use of this device.

Smiths Medical and JELCO design marks and Point-Lok are trademarks of the Smiths Medical family of companies. The symbol © indicates it is registered in the U.S. Patent and Trademark Office and certain other countries.

©2007 Smiths Medical family of companies. All rights reserved. SS300P - SS195210EN 10.2007