Tennis Elbow Strap

For use with MFR #'s 155-BH-194

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INDICATIONS

- Medial epicondylitis (golfer's elbow) • Lateral epicondylitis (tennis elbow)

PRECAUTIONS

- If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.
- Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene by products.

DIRECTIONS FOR USE

1. Wrap the support around the forearm at least two finger widths below crease of the elbow.
2. The support should fit snugly, but it should not be tight enough to impair circulation.
3. Loop end through D-ring and secure. For the tennis elbow strap, secure hook and loop closures. Support should fit snugly, but it should not be tight enough to impair circulation.

CLEANING INSTRUCTIONS

Hand wash in cold water using mild soap, rinse thoroughly. AIR DRY. Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

FOR SINGLE PATIENT USE ONLY.

Not made with natural rubber latex.

Questions? Call 1-800-777-4908

Technical Support? Call 1-800-336-6569


Distributed by McKesson Medical-Surgical Inc.

Richmond, VA 23233

PVN A0520

PN 13-5502 Rev A

Made in Mexico

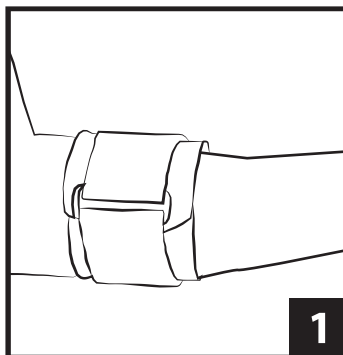Symbol Definitions	
Symbol	Definition
	Consult instructions for use
	Caution, consult accompanying documents

NOTICE: While every effort has been made in state-of-the-art techniques to obtain the maximum compatibility of function, strength, durability and comfort, this device is only one element in the overall treatment program administered by a medical professional. There is no guarantee that injury will be prevented through the use of this product.

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Distribuido por McKesson Medical-Surgical Inc.