

**SMALL/
MEDIUM**

McKESSON

Enteral Feeding Tube Securement Device

GRIP PEEL TAB | HOOK-AND-LOOP CLOSURE | SINGLE PATIENT USE



S/M
SMALL/
MEDIUM

WAIST SIZE
30 in - 45 in
(76.2 cm - 114.3 cm)

1
PER
PACK

MFR #
NEL-1920-MMS

INSTRUCTIONS FOR USE

DESCRIPTION


The McKesson Enteral Feeding Tube Securement Device comfortably and discreetly secures a G-tube, J-tube, or PEG-tube beneath clothing. Its innovative design eliminates the need for pins and tape, reducing skin irritation and discomfort.

Use under the supervision of a health care professional.

If you have questions about the use of this product consult your health care professional.

WASHING INSTRUCTIONS

Machine wash frequently, with fasteners closed, using a mild disinfectant. Hang dry.

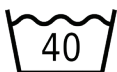

Machine washing temperature 104°F (40°C), normal process.




Do not bleach.



Do not tumble dry.




Hang to dry.

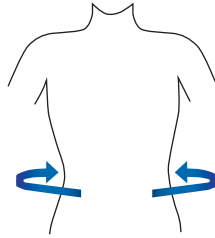


SIZE & FIT GUIDE

Measure around the center of your waist.


It is recommended that a sterile I.V. split drain dressing be placed against the skin as a buffer (See Figure 1).

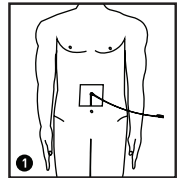


Wrap strap around the waist and secure it using the hook-and-loop closure, adjusting for comfort.

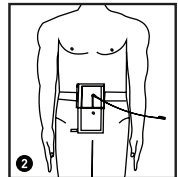

Slide tube through slit on pouch and position pouch over tube (See Figure 2).

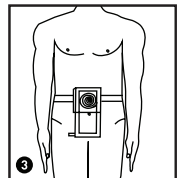

While not in use, the tube can be secured by coiling it and then closing the pouch's bottom flap over the coiled tubing (See Figures 3 and 4).

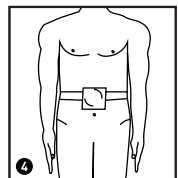

McKESSON

WARNING: To avoid danger of suffocation, keep this plastic bag away from babies and children. Do not use this bag in cribs, beds, carriages, or play pens. This bag is not a toy.

Not made with natural rubber latex.

Questions? Call 1-800-777-4908

■ Satisfaction Guaranteed

If you are not completely satisfied with any McKesson Brands product, you may return it for a full refund or credit within 12 months from the purchase date. McKesson Brands satisfaction guarantee is limited to products purchased directly from McKesson. If a McKesson Brands product is not purchased directly from McKesson, any return request or request for refund must be directed to the selling party.

Distributed by
McKesson Medical-Surgical Inc.
Richmond, VA 23233
PVN A0421
Made in USA

MFR #

NEL-1920-MMS

