McKESSON

Night Splint Dorsal

For use with MFR #'s 155-14040S-M, 155-14040L-XL.

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INDICATIONS

Helps reduce pain associated with plantar fasciitis.

PRECAUTIONS

- If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

CONTRAINDICATIONS

- Not recommended for everyday walking.

DIRECTIONS FOR USE

1. Apply splint while seated.
2. Loosen all straps.
3. Slide foot into splint. Ensure toes and ball of the foot are comfortably positioned in the forefoot section of the splint.
4. Secure the forefoot straps first, then the calf strap. Make sure all straps are secure but not too tight.

CLEANING INSTRUCTIONS

Wipe clean with warm water. Line dry. Avoid heat or sun.

FOR SINGLE PATIENT USE ONLY.

Not made with natural rubber latex.

Questions? Call 1-800-777-4908

Technical Support? Call 1-800-336-6569

Distributed by McKesson Medical-Surgical Inc.

Richmond, VA 23233

PVN A0520

PN 13-5504 Rev A

Made in China

Symbol Definitions	
Symbol	Definition
	Consult instructions for use
	Caution, consult accompanying documents

NOTICE: While every effort has been made in state-of-the-art techniques to obtain the maximum compatibility of function, strength, durability and comfort, this device is only one element in the overall treatment program administered by a medical professional. There is no guarantee that injury will be prevented through the use of this product.

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Distribuido por McKesson Medical-Surgical Inc.