# Wrist Splints with Abducted Thumb Universal

For use with MFR #'s 155-81-87480, 155-81-87490.

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

**INDICATIONS**

Treatment of deQuervains syndrome; scaphoid injuries; samekeeper's thumb; and wrist and thumb sprains and strains.

**PRECAUTIONS**

- If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

**DIRECTIONS FOR USE**

1. Release single pull strap and thumb straps. Position hand in brace with cotton stockinet liner on top of hand. Thumb should rest on the abduction stay. Make sure the Palmer stay is positioned comfortably below the palmer crease and aligned with forearm.
2. Wrap single pull strap around wrist area and secure on brace. Strap should be snug but not constrict.
3. Thumb straps should wrap in opposing directions and secure on brace. Straps should be snug but not constrict.

**CLEANING INSTRUCTIONS**

Hand wash liner in warm water using mild soap, rinse thoroughly. AIR DRY. NOTE: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

**FOR SINGLE PATIENT USE ONLY.**

Not made with natural rubber latex.

Questions? Call 1-800-777-4908

Technical Support? Call 1-800-336-6569




Distributed by McKesson Medical-Surgical Inc.

Richmond, VA 23233

PVN A0520

PN 13-5511 Rev A

Made in China

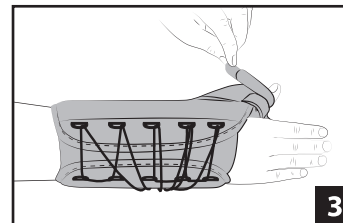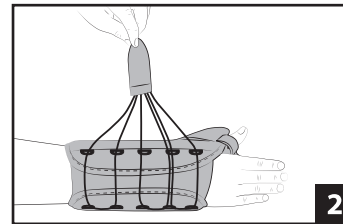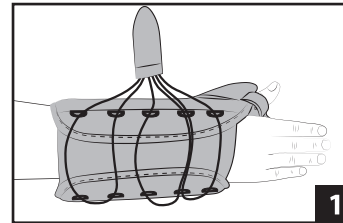Symbol Definitions	
Symbol	Definition
	Consult instructions for use
	Caution, consult accompanying documents

**NOTICE:** While every effort has been made in state-of-the-art techniques to obtain the maximum compatibility of function, strength, durability and comfort, this device is only one element in the overall treatment program administered by a medical professional. There is no guarantee that injury will be prevented through the use of this product.

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Distribuido por McKesson Medical-Surgical Inc.